

CLEANING INSTRUCTIONS

Hand Wash ONLY and allow the product to air dry. This product is NOT Machine Washer or Dryer safe.



LIMITED LIFETIME WARRANTY

Your Medic Therapeutics product is backed by a limited lifetime manufacturer's warranty. Medic Therapeutics will repair or replace your device at any time should it fail due to a defect in material or workmanship, subject to the certain limitations.

This limited warranty does not cover any damage that results from unauthorized or improper use, service, or repair. Further, it does not cover damage caused by accident, impact, negligence, or normal wear and tear. Should you discover your Medic Therapeutics product is not functioning properly, please send your device to our repair center for evaluation. If your product cannot be repaired or serviced, we will reserve the right to change it for a similar or newer model.

Please note that a flat rate of \$8.00 will be charged to cover a service evaluation fee and return shipping of your device. All warranty claims must be accompanied by a copy of your proof of purchase from an authorized retailer. Please send your device, proof of purchase, and a check or money order in the amount of \$8.00 made out to Medic Therapeutics to:

Address:

Medic Therapeutics Service Center
3069 Taft Street
Hollywood, FL 33021

Contact:

warranty@medictherapeutics.com



POSTURE CORRECTOR



User Manual


FOLDING AND STORAGE

This product helps improve posture and minimize back, shoulder and neck pain. Please keep this user manual for future reference.

PUTTING THE PRODUCT TOGETHER



STEP 1:
Your product will arrive without the straps attached and will look like this image.

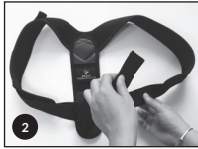

STEP 2:
Detach the Velcro end of each strap and slide the end (Velcro side out) through the same side of the plastic buckle.



STEP 3:
Pull the strap through 1st elastic hook (this elastic hook will keep the strap in place).



STEP 4:
Pass it through the 2nd elastic hook as well to keep it in place firmly and reattach originally. Repeat the same process for the 2nd strap.



STEP 5:
Your Posture Corrector is now ready to wear. Make sure the straps are not twisted.


HOW TO WEAR



STEP 1:
Adjust the length of the straps.



STEP 2:
Put one strap over each arm.

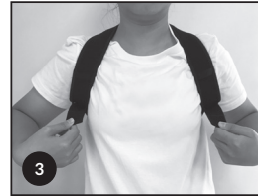

STEP 3:
Adjust the Velcro straps until the Posture Corrector is tightened and it fits comfortably.



STEP 4:
Ready to wear!

SAFETY INFORMATION

- Do not wear the Posture Corrector brace longer than 3 hours in one session or 12 hours daily. We recommend to use it for 30 minutes every day.
- Do not pull the straps too tight, it may disturb proper blood circulation.
- You may wear the posture corrector under clothing and it can make contact directly to your skin, but it is recommended to wear it over a cotton clothing material to help avoid any possible negative skin reactions/irritation.
- Do not wear while doing high-intensity physical exercise.
- Do not wear while sleeping.