



Tel:(852) 3105 2821 Fax:(852) 3105 0197 E-mail: admin@synemed.info

Bath in bed wipes

A complete bathing solution in one simple step:

✓ Cleanse
 ✓ Remove harmful bacteria

✓ Moisturise
✓ Control odour

Reynard Bath in bed wipes are ideal for fast, effective and gentle waterless cleansing of the body & face. Fragrance free, rinse free and dermatologically tested, Bath in bed wipes offer the perfect solution when water is not available or not an option. Our wipes eliminate the need for heating water, bowls, towels, cleaning products and other reusable accessories that can harbour harmful bacteria. Upgrading to Reynard Bath in bed wipes can play an important role in reducing cross contamination and preventing patient infection.

Convenient and pleasant to use, our high-quality wipes contain no harsh chemicals or perfumes that could dry the skin or alter its acid mantle, making them ideal for use on sensitive skin. Reducing patient bathing time and removing the need for materials and laundering, Reynard Bath in bed wipes present a cost-effective solution for saving time and money.

Available in various wipe quantities and cloth substrates to suit every need.

Product name	Bath in bed wipes
SKU	RHS102
Wipe size	33 x 23cm
Wipes per pack	8
Packs per carton	24

Product name	Everyday Mini Pack Bath in bed wipes
SKU	RHS116
Wipe size	30 x 20cm
Wipes per pack	4
Packs per carton	48

Product name	Everyday Bath in bed wipes
SKU	RHS117
Wipe size	30 x 20cm
Wipes per pack	8
Packs per carton	24









How to use Reynard Bed in bath wipes

These wipes can be used heated, cooled or at room temperature for optimal comfort.

Remove one cloth at a time and then reseal pack. Use each cloth for specific areas of the

body as follows (see diagrams below):

Instructions for 8 wipe packs

How to use:

1 Cloth 1: Face, neck and chest

2 Cloth 2: Right arm and axilla

Cloth 3: Left arm and axilla


4 Cloth 4: Perineum

Cloth 5: Right leg

6 Cloth 6: Left leg

🔂 Cloth 7: Back

(B) Cloth 8: Buttocks

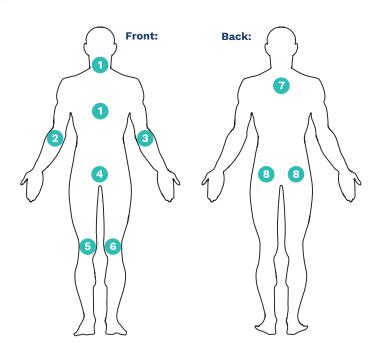

Instructions for 4 wipe packs

How to use:


Oloth 1: Hands, face, neck and arms

Cloth 2: Armpits, chest area and sides of body

3 Cloth 3: Groin and buttocks area

4: Back and both legs



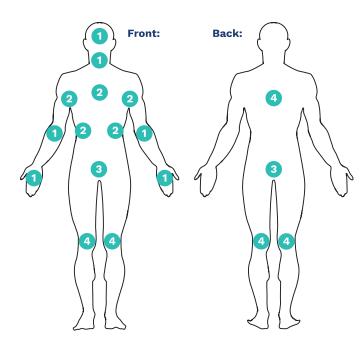

For more information on how to use Reynard Bath in bed Wipes you can view our educational video HERE