

Shampoo Cap

A single step cleansing and conditioning solution that leaves hair smelling fresh and feeling great. The rinse-free cap is impregnated with a gentle, conditioning shampoo that is designed to release when massaged into the scalp. The used solution is then absorbed back into the cap, along with any dirt, EEG gel, debris or blood.

When used in conjunction with Reynard Bath in bed wipes, we offer a complete in-bed bathing method that is less stressful and time consuming on both patient and carer. A simplified approach to hair washing that offers a pragmatic solution for frail, ill or critical care residents and patients.

Product name	Shampoo Cap
SKU	RHS104
Caps per pack	1
Packs per carton	24



How to use a Reynard Shampoo Cap


Preparation:

Heating: Heat the unopened pack in a microwave for 10 to 20 seconds (always check the temperature of cap before use).


Chilling: Place sealed pack in the refrigerator for 10 minutes.

Other: Use cap directly from pack.


Step 1: Place cap on the patients head.




Step 2: Ensure all hair is covered.

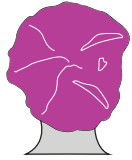


Step 3: Massage the head all over for 2-3 minutes to make sure the hair is completely saturated.

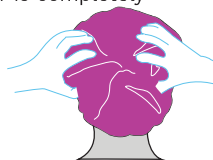


Step 4: For patients with longer hair, remove cap and work through the ends of the hair to ensure all hair is properly washed.



Step 5: Finally, towel dry hair if required



Step 6: Dispose of the cap and packaging responsibly. Do not flush or put into a macerator.



Australia
44 Hampden Road
Artarmon, NSW, 2064
enquiries@reynardhealth.com.au
1300 667 199

New Zealand
PO Box 8470
Havelock North
Sales@reynardhealth.com
0508 494 737

UK
13 White Horse Business Park,
Stanford in the vale SN7 8NY
uk@reynardhealth.com
+44 (0)845 263 7335

International
PO Box 8470
Havelock North
Sales@reynardhealth.com
+64 (0)6 650 0709