

Foldable Electric Wheelchair Installation Procedures



1. When you receive the goods, you will see this wooden box.



2. Please lay flat the wooden box carefully and lightly to protect the wheelchair.



3.1 Use screwdriver to open the wooden box.
(Note: this wooden box should be kept for a period of 14 days, in case you need warranty services.)



3.2 Open the wooden box carefully as per the picture, do not damage the wooden box.



4.1 After opening the wooden box, check the parts as below: ① User guide ② Tool kit ③ Safety belt ④ Anti-tip wheels & extendable standing legs ⑤ Plastic covers



4.2 Tool kit contents
You may need this tool kit to install some parts onto your wheelchair.



4.3 Overview of the wheelchair in opened wooden box.



5. Take out the wheelchair carefully.



6. Insert the controller joystick into the armrest tube.



7. Install the plastic covers with included screws.



8. Install the metal standing legs with included screws.



9. Install the extendable standing legs and fix them with the knobs.



10. Insert the anti-tip wheels into the tubes.



11. Plug the battery connector (c) to the battery pack we sent in separate package.



12. If you will use your own battery pack, you need the battery cable we provided to match the battery connector (c).



Note: We will ship 24V 10AH or 24V 15AH battery based on buyer's choice.