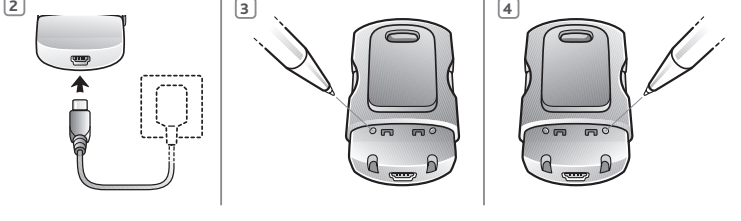
The pairing procedure allows you to control which Bluetooth devices can communicate with each other. Pairing only needs to be done once for each device to be used with your phone clip.

You can pair up to 8 Bluetooth devices with the Phone Clip, but only 2 such devices can be connected/active at a time.

The pairing procedure is described in the following section. Once a mobile phone has been paired with the Phone Clip, the connection procedure configures the device to output its audio signal through the Bluetooth connection. This procedure is also described in the following section.

### Pairing with your mobile phone

The pairing procedure is controlled by your mobile phone. Different mobile phones have their own menu structures. Refer to your phone's manual if you did not complete the pairing procedure with the generic steps described here.

- Turn your Phone Clip on and remove the silver cap. Place your mobile phone next to the Phone Clip, and make sure your phone is on.
- Find the connectivity settings on your phone. Look for the "Bluetooth" feature in the main menu or one of the sub-menus, such as "Connectivity".
- On your mobile phone, ensure that Bluetooth is turned on.
- With the tip of a pen or similar object, press the small blue Bluetooth pairing button on the back of your Phone Clip once. See **figure 4**. The Phone Clip will now be in Bluetooth pairing mode for two minutes.
- Search for Bluetooth devices on the mobile phone.
- Your mobile phone should present a list of Bluetooth devices discovered. Select "Hearing Aid Phone" from this list.

Your mobile phone may then ask you to enter a passcode. If so, enter "0000" (four zeros). Your Phone Clip should now be paired to your mobile phone. Some mobile phones may ask which Bluetooth service you wish to enable. Select "Headset".

#### Connecting

Once your mobile phone and Phone Clip are paired, the mobile phone may need to be connected before it will actually send the appropriate audio signals.

This function should also appear in your mobile phone's "Bluetooth" menu, usually under a list of "Paired devices." In that list, highlight your Phone Clip and select "Connect."

When your Phone Clip and mobile phone are connected, you may see a headset symbol on the phone's display. See **figure 5**.

The connection between the Phone Clip and the mobile phone will be maintained for as long as both devices are turned on and remain within range of each other. If either device is switched off or gets out of range, you may have to perform the connection procedure again.

Some phones allow for "Automatic reconnection." It is available, when you select that you activate this feature if it is available. Refer to your mobile phone's manual for instructions.

#### Phone operation

The below table provides an overview of how to use the Phone Clip to control phone calls.

**Note:** Except for the answer and end call operations one or more of the below described operations may be phone dependent, i.e. they may either differ slightly or be absent altogether.

<b>Question / How to do it / Also good to know</b>	
<b>Answering and ending a call</b>	Push the call answer/hang-up button once. If another call is already active it will be terminated and the new incoming call, indicated by clicking tones, will become active.
<b>Rejecting a call</b>	If another call is currently on hold it will become the active call. Double-click the call answer/hang-up button.
<b>Transferring a call initiated on the mobile phone to Phone Clip</b>	If another call is already active it will remain active, if no new incoming call, indicated by clicking tones, will be rejected. Push the call answer/hang-up button once.
<b>Put an active call on hold</b>	Press and hold the call answer/hang-up button for 2 seconds.
<b>Press and hold the call answer/hang-up button for 2 seconds.</b>	If another call is currently on hold or an incoming call is pending, it will now become the active call.
<b>Last number redial</b>	Double-click the call answer/hang-up button while no call is active or pending.
<b>Voice dialling</b>	If two mobile phones are connected, this operation applies to the most recently paired mobile phone.
<b>Increase or decrease the volume of the speaker at the other end of the phone call</b>	Push the + sign on the side of the Phone Clip to increase the volume.
<b>Mute or unmute your sound processor(s) microphones during a phone call</b>	Push the microphone mute/unmute button on the front of the Phone Clip**.

## Welcome

Congratulations on your purchase of a Cochlear™ Wireless Phone Clip.

#### Intended use

The Cochlear Wireless Phone Clip is intended to transfer signals between a Bluetooth® enabled mobile phone and a compatible Cochlear sound processor.

#### Indications:

This accessory has no therapeutic indications in itself! It will support the therapeutic indications of the Cochlear wireless sound processor used with this accessory.

#### Clinical Benefit:

The Cochlear Wireless Phone Clip is intended to further improve audibility during phone conversation.

#### Key to device See figure 1

- ON/OFF switch
- Sound processor program toggle\*\*
- Micro-USB charging socket
- Microphone
- Call answer/hang-up button
- Sound processor microphone mute button\*\*
- Volume control
- Mouting clip
- Bluetooth pairing button (under the cover)
- Sound processor pairing button (under the cover)

#### Introduction

The Cochlear™ Wireless Phone Clip will allow you to connect your compatible Cochlear sound processor to Bluetooth™-enabled mobile phones. This enables clear and trouble-free phone connectivity, whether you are at home, in the office or on the move.

When used with a compatible Cochlear sound processor, the Phone Clip also doubles as a basic remote control making it possible for you to adjust the volume of your sound processor and toggle through programs when you are not talking on the phone\*\*.

You can also choose to mute your sound processor microphone to shut out ambient sound while you are talking on the phone or when using the Phone Clip as a remote control\*\*.

Please read this manual carefully to fully benefit from the Phone Clip. Ask your hearing care professional if you have any questions. Your vision and experiences with our products and services are important to us. If you have any comments you would like to share, please contact us.

#### Warranty

The warranty does not cover defects or damage arising from, associated with, or related to the use of this product with any non-Cochlear processing unit and/or any non-Cochlear implant. See "Cochlear Global Limited Warranty card" for more details.

#### Contacting customer service

We strive to provide the best possible products and services to our clients. Your vision and experiences with our products and services are important to us. If you have any comments you would like to share, please contact us.

#### Key to symbols

The following symbols will be used throughout these documents.

	Caution
	CE-mark
	Serial Number
	Medical device
	Unique device identifier
	Date of manufacture
	"Consult instructions for use" or "Operating instructions"
	This device includes a RF transmitter that operates in the range of 2.4 GHz – 2.48 GHz

#### Remote Control operation\*\*

If you have a Cochlear Baha Sound Processor, and are not making a call, the Phone clip can be used as a remote control for your sound processor(s).

#### Volume control

**Increasing the volume of your sound processor(s)\*\*** Push the "+" sign on the volume control to increase the volume.

**Decreasing the volume of your sound processor(s)\*\***

- Push the "-" sign on the volume control to decrease the volume.

- Depending on the model, you can also utilize the volume control that is built into your mobile phone.

Please note that if you have two sound processors, changing the volume will apply to both of them\*\*.

**Mute or unmute the microphone(s) on your sound processor(s)\*\***

- To mute your sound processor microphones, push the microphone mute/unmute button on the front of the phone clip once.

- To unmute your sound processor microphones push the microphone mute/unmute button on the front of the phone clip once.

#### Toggle through sound processor programs

Push the program button to toggle through available programs in your sound processor(s).

#### Operating time

Battery operation time and the operating time of your Phone Clip depend greatly on how the device is used.

Your Phone Clip battery can be recharged hundreds of times. If the operating time becomes noticeably shorter, please contact your hearing care professional.

#### Wearing your Phone Clip

Clip your Phone Clip on your clothing so that the microphone faces your hands. This ensures that the microphones in the Phone Clip are placed optimally to pick up your voice during phone calls.

The Phone Clip has a swivel clip to ensure that you can always place it so that the microphone faces upwards towards your mouth.

For optimal voice pickup, keep the device within a range of 10-30 cm from your mouth.

#### Light indicators (LED)

The LED at the top of your Phone Clip serves as a multipurpose user interface, providing information on the status of your device.

<b>LED blinking pattern /Meaning</b>	
Normal operation, green light	
Battery low, yellow light	
Charging, red light	
Fully charged – still connected to charger, red and green light	
Ready for pairing with hearing instrument(s), yellow light	
Ready for pairing with Bluetooth device, blue light	
Bluetooth call active, blue light	

#### Bluetooth Operating Range

Your phone clip supports Bluetooth operation up to a range of approximately 10 metres. Line-of-sight between the devices is not required. However, the following factors may affect the maximum range:

- Ambient interference
- The mobile phone's range may be less than the 10 metres supported by the Phone Clip
- If your mobile phone moves out of range, the connection will be broken. If the device is moved within range again, the Phone clip will attempt to re-establish the connection. Whether this is successful will depend on the length of time since the link was broken and on the mobile phone's properties.

#### Important Information

#### Maintenance and Care

- Clean the Phone Clip using a damp cloth. Never use household cleaning products (washing powder, soap, etc.) or alcohol to clean the device.
- When the Phone Clip is not in use, turn it off and store it safely.
- Protect the Phone Clip from excessive moisture (bathing, swimming) and heat (radiator, car dashboard)
- Protect it from excessive shock and vibration. Protect the microphone and charger sockets from debris and dirt.

**General warnings**

- Keep this device out of reach of children.
- The Phone Clip uses low-powered digitally coded transmissions in order to communicate with other wireless devices. Although unlikely, nearby electronic devices may be affected. In such cases, move the Cochlear Phone Clip away from the affected electronic device.

- When using the Phone Clip and it is affected by electromagnetic interference, move away from the source of the interference.
- Avoid exposing your product to rain, moisture or other liquids to protect against damage to the product or personal injury.

- Keep the product and any cords and cables away from operating machinery.
- Do not use the product overhats, has been dropped or damaged, has a damaged cord or plug, or has been dropped in a liquid, discontinue use and contact your hearing care professional.
- Dispose of the product according to local standards and regulations.
- Do not use in areas where there is a risk of explosion.
- Contains small parts that may present a choking hazard.

- Do not use the product in areas where it could ignite flammable gases.
- Do not use this device in locations where the use electronic devices are prohibited.
- X-ray radiation (e. g., CT/MRI scans) may adversely affect the functioning of this device. Switch it off before undergoing the procedure and keep it outside the room.
- High-powered electronic equipment, larger electronic installations and metallic structures may impair and significantly reduce the operating range.
- If the sound processor does not respond to the device because of an unusual type of interference, move away from the source of this interference.
- Your sound processors and this device were given unique communication codes during fitting. This ensures that your device will not affect sound processors worn by others.
- Do not alter or modify this device.
- Opening the Phone Clip might damage it. If problems occur that cannot be resolved, consult your hearing care professional.
- The Phone Clip may only be repaired by an authorized service centre.
- Do not use the Phone Clip in areas where RF transmission is prohibited, e.g. in airplanes, etc.
- Only connect the Phone Clip to connections for which it is explicitly intended.

#### General precautions

- Do not use this device in locations where the use electronic devices are prohibited.
- X-ray radiation (e. g., CT/MRI scans) may adversely affect the functioning of this device. Switch it off before undergoing the procedure and keep it outside the room.
- High-powered electronic equipment, larger electronic installations and metallic structures may impair and significantly reduce the operating range.
- If the sound processor does not respond to the device because of an unusual type of interference, move away from the source of this interference.
- Your sound processors and this device were given unique communication codes during fitting. This ensures that your device will not affect sound processors worn by others.
- Do not alter or modify this device.
- Opening the Phone Clip might damage it. If problems occur that cannot be resolved, consult your hearing care professional.
- The Phone Clip may only be repaired by an authorized service centre.
- Do not use the Phone Clip in areas where RF transmission is prohibited, e.g. in airplanes, etc.
- Only connect the Phone Clip to connections for which it is explicitly intended.

#### Toggle through sound processor programs

Push the program button to toggle through available programs in your sound processor(s).

#### Operating time

Battery operation time and the operating time of your Phone Clip depend greatly on how the device is used.

Your Phone Clip battery can be recharged hundreds of times. If the operating time becomes noticeably shorter, please contact your hearing care professional.

#### Serious incidents

Serious incidents are rare, any serious incident in relation to your device should be reported to your Cochlear representative and to the medical device authority in your country, if available.

#### Built-in battery care

Please observe the following:

- Your product is powered by a rechargeable battery. A new battery performs optimally after two or three complete charge and discharge cycles.
- The battery can be charged and discharged hundreds of times, but will eventually wear out.
- Unplug the charger from the mains socket and the product when not in use.
- If left unused, a fully charged battery will lose its charge over time.
- Leaving the product in hot places will reduce the battery's capacity and service life.
- A product with a cold battery will not work temporarily, even if fully charged. Battery performance is particularly limited in temperatures well below freezing.

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.
- Changes or modifications can void the user's authority to operate the equipment.
- List of countries:** The products are in compliance with the following regulatory requirements:
  - In EU, the device conforms to the General Safety and Performance Requirements according to Annex 1 of Medical Device Regulation 2017/745 (MDR) and essential requirements and other relevant provisions of Directive 2014/53/EU (RED). The declaration of conformity may be consulted at [www.cochlear.com](http://www.cochlear.com)
  - Products with wireless functionality are intended for sale in countries within the European Economic Area as well as Switzerland. Specification of restrictions: You are not allowed to operate the equipment within 20 km of the center of My Alesund, Norway.
  - In US: FCC CFR 47 Part 15, subpart C, section 15.249.
  - Other identified applicable international regulatory requirements in countries outside the EU and US. Please refer to local country requirements for these areas.
  - Products in compliance with EN 300 328.
  - In Canada the Phone Clip is certified under the following certification number: IC:8039C-BT82. This device complies with Industry Canada license-exempt RSS standard(s).
  - This Class B digital apparatus complies with Canadian ICES-003.
- Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept interference, including interference that may cause undesired operation of the device.

- Equipment includes** RF transmitter.
- \*\* Note: Feature only available to Baha Sound Processor users.
- When you disconnect the power cord or any accessory, grasp and pull the plug, not the cord.
- Never use a charger that is damaged.
- Do not attempt to disassemble the charger because it may expose you to a dangerous electric shock.
- Incorrect reassembly can cause electric shock or fire when the product is subsequently used.
- Avoid charging your product in extremely high or low temperatures and do not use the charger outdoors or in damp areas.
- For availability of approved accessories, please check with your hearing care professional.
- Charge the product according to the instructions provided.

#### Charger warning

- When you disconnect the power cord or any accessory, grasp and pull the plug, not the cord.
- Never use a charger that is damaged.
- Do not attempt to disassemble the charger because it may expose you to a dangerous electric shock.
- Incorrect reassembly can cause electric shock or fire when the product is subsequently used.
- Avoid charging your product in extremely high or low temperatures and do not use the charger outdoors or in damp areas.
- Do not charge the Cochlear Phone Clip while it is being worn.

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<b>Operating range</b>	Up to 7 meters or more
<b>Power supply</b>	Mains socket
<b>Wireless</b>	2,4 GHz
<b>Bluetooth range</b>	Up to 10 m at clear line of sight
<b>Sound Processor interface range</b>	Up to 3 m at clear line of sight
<b>Operating temperature</b>	0 to 45° C
<b>Storage temperature</b>	-20 to 45° C

#### Temperature test, transport and storage information

The product is subject to various tests in temperature and damp heating cycling between -25° C and +70° C according to internal and industry standards.

#### Wireless accessory type designations for models included in this User Manual are:

Model: BTB-2
FCC ID: Q23BTB-2
IC: 8039C-BT82
MSIP-CRI-1CK-BTB-2
202-SMC095
14-0054-2002
CCAHSLP175078
CMIT ID:2015DJ4571
Model: BTB-2
FCC ID: Q23BTB-2
IC: 8039C-BT82
MSIP-CRI-1CK-BTB-2
202-SMC095
14-0054-2002
CCAHSLP175078
CMIT ID:2015DJ4571

**Statement** This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device must not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**Note:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy which, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.
- Changes or modifications can void the user's authority to operate the equipment.
- List of countries:** The products are in compliance with the following regulatory requirements:
  - In EU, the device conforms to the General Safety and Performance Requirements according to Annex 1 of Medical Device Regulation 2017/745 (MDR) and essential requirements and other relevant provisions of Directive 2014/53/EU (RED). The declaration of conformity may be consulted at [www.cochlear.com](http://www.cochlear.com)
  - Products with wireless functionality are intended for sale in countries within the European Economic Area as well as Switzerland. Specification of restrictions: You are not allowed to operate the equipment within 20 km of the center of My Alesund, Norway.
  - In US: FCC CFR 47 Part 15, subpart C, section 15.249.
  - Other identified applicable international regulatory requirements in countries outside the EU and US. Please refer to local country requirements for these areas.
  - Products in compliance with EN 300 328.
  - In Canada the Phone Clip is certified under the following certification number: IC:8039C-BT82. This device complies with Industry Canada license-exempt RSS standard(s).
  - This Class B digital apparatus complies with Canadian ICES-003.
- Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept interference, including interference that may cause undesired operation of the device.

- Equipment includes** RF transmitter.
- \*\* Note: Feature only available to Baha Sound Processor users.
- When you disconnect the power cord or any accessory, grasp and pull the plug, not the cord.
- Never use a charger that is damaged.
- Do not attempt to disassemble the charger because it may expose you to a dangerous electric shock.
- Incorrect reassembly can cause electric shock or fire when the product is subsequently used.
- Avoid charging your product in extremely high or low temperatures and do not use the charger outdoors or in damp areas.
- For availability of approved accessories, please check with your hearing care professional.
- Charge the product according to the instructions provided.

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.
- Changes or modifications can void the user's authority to operate the equipment.
- List of countries:** The products are in compliance with the following regulatory requirements:
  - In EU, the device conforms to the General Safety and Performance Requirements according to Annex 1 of Medical Device Regulation 2017/745 (MDR) and essential requirements and other relevant provisions of Directive 2014/53/EU (RED). The declaration of conformity may be consulted at [www.cochlear.com](http://www.cochlear.com)
  - Products with wireless functionality are intended for sale in countries within the European Economic Area as well as Switzerland. Specification of restrictions: You are not allowed to operate the equipment within 20 km of the center of My Alesund, Norway.
  - In US: FCC CFR 47 Part 15, subpart C, section 15.249.
  - Other identified applicable international regulatory requirements in countries outside the EU and US. Please refer to local country requirements for these areas.
  - Products in compliance with EN 300 328.
  - In Canada the Phone Clip is certified under the following certification number: IC:8039C-BT82. This device complies with Industry Canada license-exempt RSS standard(s).
  - This Class B digital apparatus complies with Canadian ICES-003.
- Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept interference, including interference that may cause undesired operation of the device.

- Equipment includes** RF transmitter.
- \*\* Note: Feature only available to Baha Sound Processor users.
- When you disconnect the power cord or any accessory, grasp and pull the plug, not the cord.
- Never use a charger that is damaged.
- Do not attempt to disassemble the charger because it may expose you to a dangerous electric shock.
- Incorrect reassembly can cause electric shock or fire when the product is subsequently used.
- Avoid charging your product in extremely high or low temperatures and do not use the charger outdoors or in damp areas.
- For availability of approved accessories, please check with your hearing care professional.
- Charge the product according to the instructions provided.

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