# DURONIC

Creative Consumer Electronics

## BPM200 - Blood Pressure Monitor

PLEASE READ ALL THE INSTRUCTIONS BEFORE USE

[duronicon.com](http://duronicon.com)

[duronicon](https://www.youtube.com/channel/UCduronicon)

[@duroniconmedia](https://www.instagram.com/duroniconmedia)

[Duronicon](https://www.facebook.com/duronicon)

[@duroniconmedia](https://www.twitter.com/duroniconmedia)

**Contents**

<b>English Manual</b>	<b>5</b>
<b>Manuel d'utilisation en français</b>	<b>32</b>
<b>Gebrauchsanleitung auf Deutsch</b>	<b>55</b>
<b>Manual de Instrucciones en Español</b>	<b>78</b>
<b>Manuale D'istruzioni In Italiano</b>	<b>101</b>
<b>Instrukcja obsługi. Polski</b>	<b>124</b>
<b>Gebruikershandleiding Nederlands</b>	<b>147</b>

**English Manual**

Thank you for selecting SHINEMART arm type blood pressure monitor.

The monitor features blood pressure measurement, pulse rate measurement and auto-save the result. The design provides you with two years of reliable service. Readings taken by the blood pressure monitor are equivalent to those obtained by a trained observer using the cuff and stethoscope auscultation method. This manual contains important safety and care information, and provides step by step instruction for using the product.

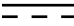
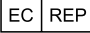
Read the manual thoroughly before using the product.

## Features:

- 59x47mm Digital LCD display
- Maximum 99 x 2 groups records
- Inflating measuring technology

## Icon Key

The symbols shown below may appear in the user manual, labeling or other components. They are the standard requirement.

	THE OPERATION GUIDE MUST BE READ		Type BF Applied Parts
	This product complies with the MDD93/42/EEC requirements.		DISPOSAL: Do not dispose this product as unsorted municipal waste. Collection of such waste separately for special treatment is necessary.
	Manufacturer		Direct Current
	Specifies Serial Number		Wellkang Ltd Suite B, 29 Harley Street, London, W1G 9QR, UK
	Manufacture Date		Degree of protection against the ingress of water.

## Safety Information

- This device is intended for adult use only. Do not use this device on children.
- This device should not be used on any female who is, or is suspected of being, pregnant. Besides providing inaccurate readings, the effects of this device on the foetus are unknown.
- This device is intended for non-invasive measuring and monitoring of arterial blood pressure. It is not intended for use on any extremities other than the arm and should not be used for any other functions other than obtaining a blood pressure measurement.
- Contact your doctor if you are concerned about your blood pressure. This device is for self-monitoring and should not be used for self-diagnosis.
- Do not adjust, begin or end medical treatment or medication without asking a doctor for treatment advice.
- If you are taking medication, consult your doctor to determine the most appropriate time to measure your blood pressure.
- When this device is used to measure patients who have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, the results from this device may vary.
- If you are concerned about your blood pressure result, please consult your doctor.
- If the arm cuff pressure exceeds 40 kPa (300 mmHg), the unit will automatically deflate the cuff. Should the cuff not deflate when pressure exceeds 40 kPa (300 mmHg), detach the cuff from the arm and press the 'START/STOP' button to stop inflation.
- The device is not AP/ APG equipment and is not suitable for use in the presence of a flammable anaesthetic mixture of air with oxygen or nitrous oxide.

- To avoid measurement errors, please avoid using the device in areas with a strong electromagnetic field radiated interference signal or electrical fast transient/ burst signals.
- The user must check that the equipment functions safely and see that it is in proper working condition before being used.
- This device is not suitable for continuous monitoring during medical emergencies or operations. Otherwise, the patient's arm and fingers can become numb, swollen and even purple due to a lack of blood.
- Please use the device as instructed in this user manual. If you do not do so, the performance and lifetime usage of the device will be impacted and reduced.
- During use, the user/patient will be in contact with the cuff. The materials of the cuff have been tested and found to comply with the requirements of ISO 10993-5:2009 and ISO 10993-10:2010, therefore should not cause any irritation.
- Use only Duronic authorised parts and accessories. Parts and accessories not approved for use with the device may result in operational failure.
- The device should not need to be calibrated within the first two years of use.
- If you have any problems with this device, such as setting up, maintaining or using the device, please contact the Duronic Customer Care Team for assistance.
- Do not open or repair the device by yourself. If you experience any problems with your device, contact Duronic for assistance.
- When cleaning the device, please use a soft cloth to clean the whole unit. Do not submerge it in water or any other liquid, and do not use any abrasive or volatile cleaners.

***When using Batteries***

- The operator or user of this device should not touch the output of batteries and the patient simultaneously.
- If using batteries in the device, please ensure that the correct size batteries are used, as described in this manual. Do not use any other type of battery as this may damage the device.
- If any battery fluid should leak and get into your eyes, rinse your eyes with plenty of clean water and contact a doctor immediately.

***When using a Power Adaptor***

- If using a power adaptor with this device, please ensure that you do not handle the power cable or plug with wet hands.
- Do not use a power adaptor if the cable or plug is damaged.
- Do not yank the power cable to remove it from the plug socket. Always grasp the plug and pull it out to disconnect it.
- When plugging the device into a mains socket, please ensure the power supply is 220-240V ~ 50/60Hz.

---

**Contact Information**

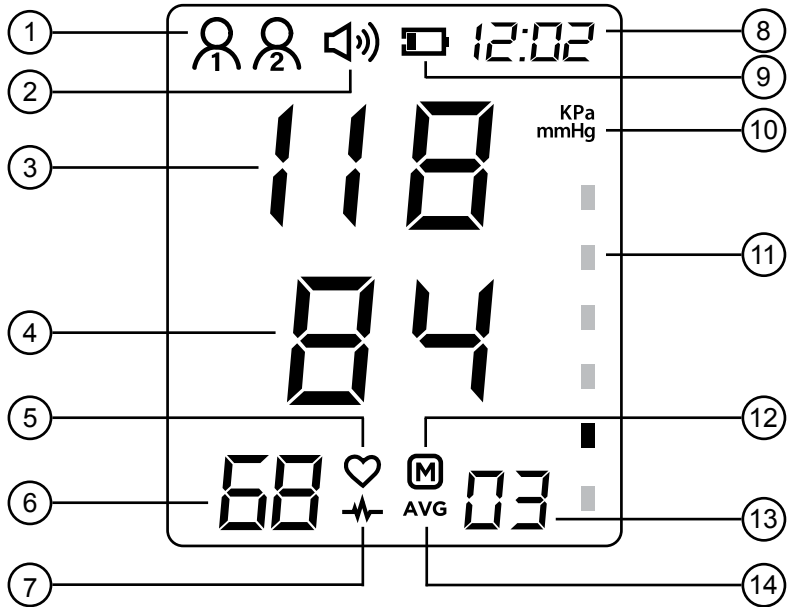
**Distributed by:** Shine-mart Ltd

**Manufactured by:** Shenzhen Jumper Medical Equipment Co. Ltd

**Authorised European Representative:** Wellkang Ltd




LCD Display

- |                       |                      |                                      |
|-----------------------|----------------------|--------------------------------------|
| 1. User Icon          | 6. Pulse             | 11. Blood Pressure / Level Indicator |
| 2. Sound on           | 7. Arrhythmia        | 12. Memory Icon                      |
| 3. Systolic Pressure  | 8. Date/Time Display | 13. Memory Number                    |
| 4. Diastolic Pressure | 9. Low Battery       | 14. Average Reading Value            |
| 5. Heartbeat          | 10. Unit             |                                      |

Symbol	Description
<b>SYS</b>	Systolic blood pressure
<b>DIA</b>	Diastolic blood pressure
<b>Pul/min</b>	Pulse per minute
<b>8:59</b>	Time (hour:minute)
<b>M 03</b>	Memory log value
<b>mmHg</b>	mmHg (unit of measurement)
<b>LO + [Battery Icon]</b>	Low battery
<b>[Irregular Pulse Icon]</b>	Arrhythmia / irregular heartbeat

## Components Included

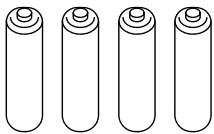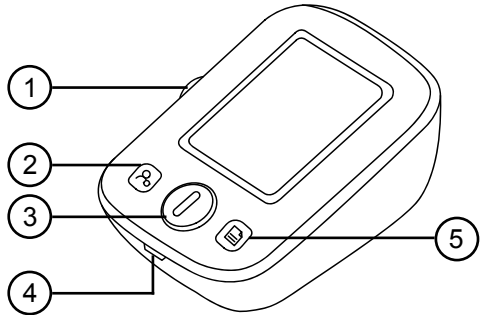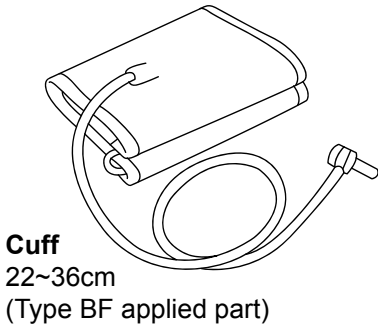- ### Blood Pressure Monitor
1. Air Jack
  2. User/Set Button
  3. Start/Stop Button
  4. USB External Power Supply
  5. Memory Button

## Choice of Power Supply

1. Battery powered mode: 4\*AAA batteries
2. DC adaptor powered mode: 5V through a USB cable (not included)

**Caution:** It is important to ensure that you use the correct batteries or power adapter in your machine to avoid any damage incurring.

## The Installment and Replacement of Battery

1. Slide off the battery cover.
2. Install the batteries by matching the correct polarity, as shown.
3. Replace the cover.

Replace the batteries when the following occurs:

- The  icon shows
- The display dims
- The display does not light up

**Caution:**

- *Remove batteries if the device is not likely to be used for some time.*
- *Old batteries are harmful to the environment, do not dispose them with other daily rubbish. Remove the old batteries from the device and follow your local recycling guidelines. Do not dispose of batteries in fire; batteries may explode or leak.*

## Arrhythmia / Irregular Heartbeat Detector

This blood pressure monitor is equipped with Irregular Heartbeat (IHB) Detection. During each measurement this monitor records the heartbeat intervals and works out the standard deviation. If the calculated value is larger than or equal to 15, this equipment will light up the 'IHB' symbol on the screen when displaying the measured result.

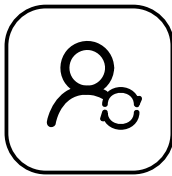
**Caution:** *The appearance of the 'IHB' icon indicates that a pulse irregularity consistent with an irregular heartbeat was detected during measurement. Usually this is NOT a cause for concern. However, if the symbol appears often, we recommend you seek medical advice. Please note that the device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.*

## Setting the Date, Time and Unit

It is important to set the clock before using your blood pressure monitor, so that a time stamp can be assigned to each record that is stored in the memory.

(Year: 2000-2050, Time: 24 H)

1. When the monitor is off, press and hold down the 'USER/SET' button for 5 seconds. The 'User 1' icon will flash.
2. Press the 'USER/SET' button to change the **year**. Scroll through the year using the 'MEMORY' button.
3. Press the 'USER/SET' button to change the **date** (month/day). Scroll through the year using the 'MEMORY' button.
4. Press the 'USER/SET' button to change the **time** (hour/minute). Scroll through the time using the 'MEMORY' button.
5. Press the 'USER/SET' button to change the **unit of measurement** (mmHg/ KPa). Scroll through the time using the 'MEMORY' button.

**USER/SET  
BUTTON**


**MEMORY  
BUTTON**

## Selecting or Changing Users

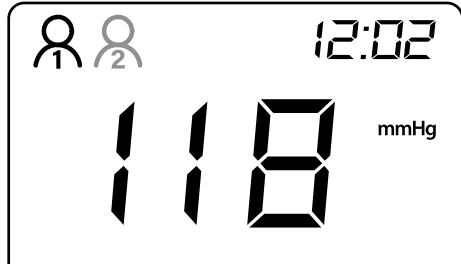When the monitor is off, press and hold down the 'USER/SET' button for 5 seconds. The 'User 1' icon will flash, then you can press the 'MEMORY' button to change the setting to 'User 2'.

**USER/SET  
BUTTON**



## Clearing the Memory

To clear the memory and delete all of the saved records, press and hold the 'MEMORY' button for 3 seconds. When it shows 'M' on the screen, continue to hold the 'MEMORY' button and press the 'USER/SET' button at the same time.

The screen will show 'ALL EE'. This means the records have all been deleted.

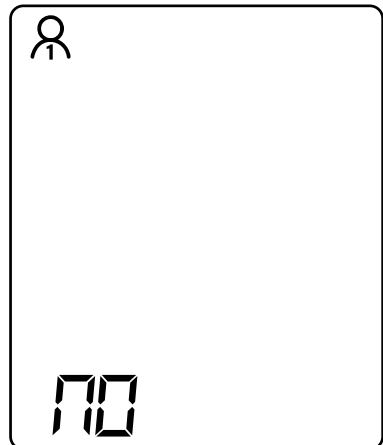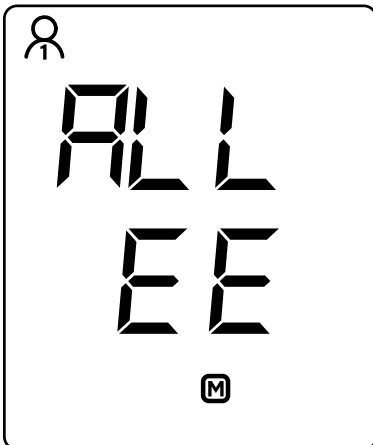When there are no records stored, the screen will show 'NO'.


## Accessing the Memory

The reading measured will save automatically. To access and scroll through the saved readings, press the 'MEMORY' button. You can switch between 'User 1' and 'User 2' by pressing the 'USER/SET' button.

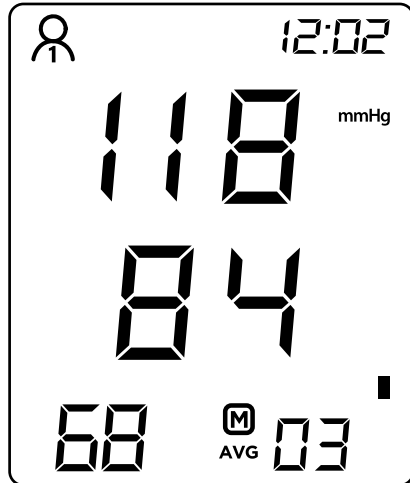**MEMORY  
BUTTON**

The blood pressure monitor can automatically store 99 measurements for two users, and automatically replace the 99 measurement data when the memory is full.


After the blood pressure monitor has been used several times, press the 'MEMORY' button in the shutdown state to display the average blood pressure of the last three measurements.

Press the 'MEMORY' button again to display the first group of blood pressure measurement values and look up the memory group number next to it.


## How to Use
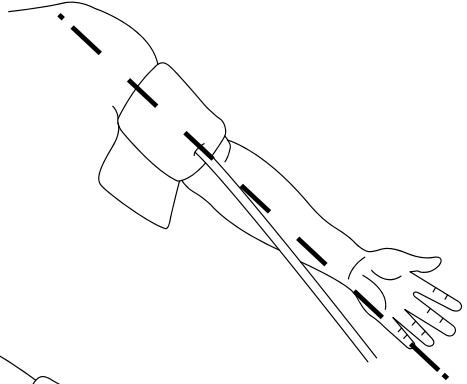
### Fastening the Arm Cuff


1. Tie the cuff on your upper arm, then position the tube off-center toward the inner side of arm in line with the little finger.

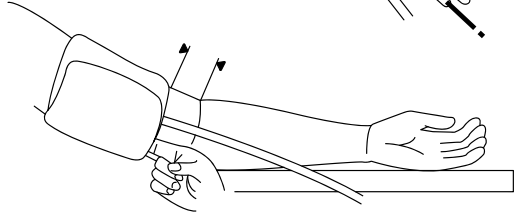2. The cuff should be snug but not too tight. You should be able to insert one finger between the cuff and your arm.

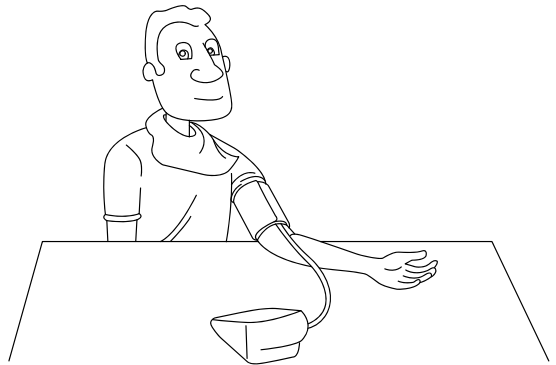3. Sit comfortably with your left arm resting on a flat surface.


### Tips

- *Rest for 5 minutes before measuring.*
- *Wait at least 3 minutes between measurements. This allows your blood circulation to recover.*
- *For a meaningful comparison, try to measure under similar conditions. For example, take daily measurements at approximately the same time, on the same arm, or as directed by a medical professional.*


## How to Take a Measurement

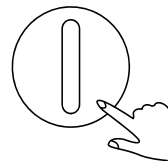
1. Ensure that the cuff is fitted properly on your arm. Press the 'START/STOP' button to turn on the monitor. The device will finish the whole measurement automatically.

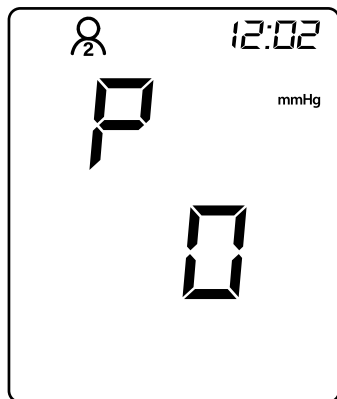2. The cuff will inflate. When fully inflated the cuff can feel slightly tight or uncomfortable. This is normal. If it is intolerable or you have any concerns, press the 'START/STOP' button to stop the machine and seek professional medical advice.


*Display when cuff inflates*

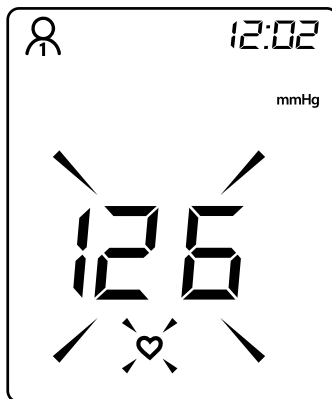*Display when cuff deflates.*

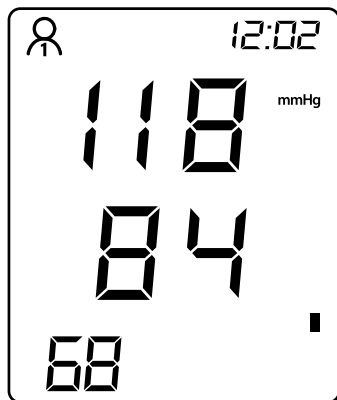When the pulse is detected, the screen will display the "heart" symbol and flash to your pulse rhythm.


3. Your reading is finished. The screen will continue to display the measurement results, unless you press 'START/STOP' button to turn off the machine. Otherwise, the machine will shutdown automatically after 30 seconds.

The reading measured will save automatically. To access and scroll through the saved readings, press the 'MEMORY' button.

## Tips for Obtaining an Accurate Measurement

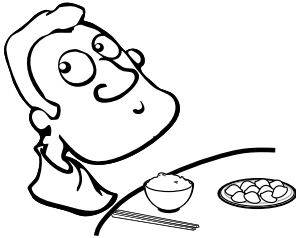Please follow the below tips to ensure you achieve an accurate measurement:

Wait at least 1 hour after eating or drinking.

Do not measure immediately after consuming caffeinated drinks such as tea and coffee, or after smoking.

Wait at least 20 minutes after taking a bath.

Do not take a measurement when talking or moving your fingers.


Do not take a measurement in a very cold environment.


Do not take a measurement when you need to use the toilet.

## Maintenance

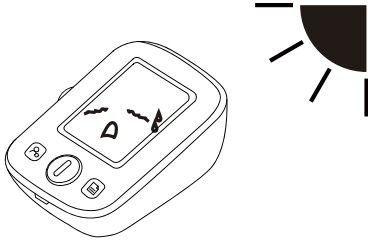
In order to get the best performance, please follow the below instructions.

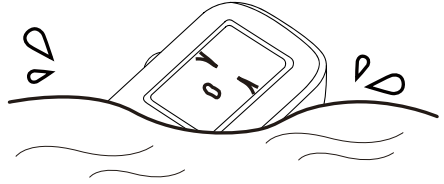
Store in a dry place and avoid direct sunlight.

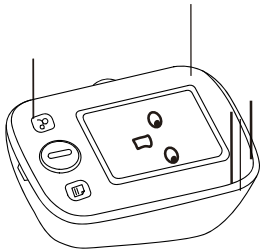
Avoid the monitor touching water. If the machine gets wet, dry immediately afterwards.


Avoid intense shaking and collision.

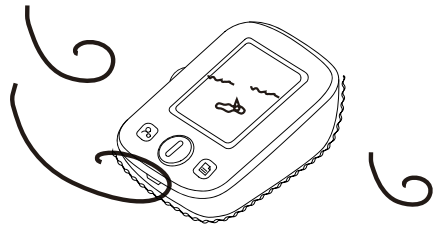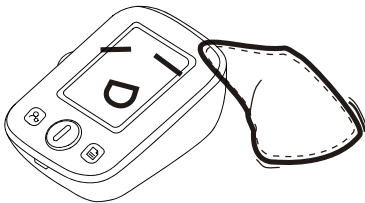Avoid dusty environments and unstable temperatures.

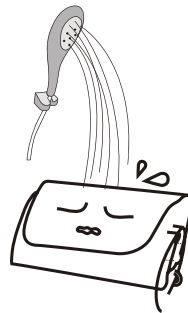Use a wet (well wrung) cloth to remove any dirt.

Do not clean the cuff with water and never immerse the cuff in water or any other liquid.

## Why does my blood pressure vary throughout the day?

Individual blood pressure varies naturally multiple times during the day. It is also affected by the way you tie your cuff around your arm and the position of your arm during measurement. For this reason, we advise you take measurement at the same time of day and in the same position.

**Please note:** *If the user takes medication the pressure will vary more.*

*If you wish to measure a second time, wait at least 3 minutes before taking another measurement.*

## Why do I get a different blood pressure at home compared to the hospital?

Blood pressure is different even throughout the day due to weather, emotions, exercise etc.

Also, there is the “white coat” effect, which means blood pressure usually increases in clinical settings due to anxiety or nerves.

What you need to pay attention to when you measure your blood pressure at home:

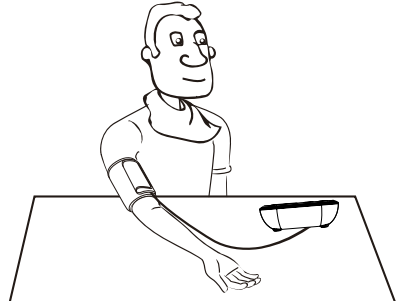
- If the cuff is tied properly.
- If the cuff is too tight or too loose.
- If the cuff is tied on the upper arm.
- If you feel anxious.

Advice: Take 2-3 deep breaths before beginning a measurement as this will help get a more accurate result. If you are particularly anxious or upset, relax yourself for 4-5 minutes until you calm down.


## Which arm should I use to take measurements?

It is fine to use the cuff on either arm, but some people will experience a difference in measurement results.

For this reason, we recommend using the same arm every time.

## Understanding the Result

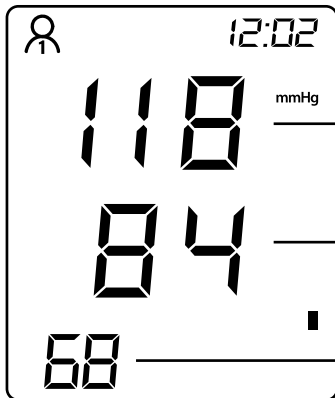**Systolic pressure (SYS):** The highest level of your blood pressure – when your heart beats, it contracts to pump blood through your arteries.

**Diastolic pressure (DIA):** The lowest level of your blood pressure – when your heart relaxes between beats.

**Pulse:** Number of heartbeats per minute. Most adults have a resting heart rate between 60 and 100bpm. Athletes are likely to have a lower resting heart rate.


## Standard Blood Pressure Classification

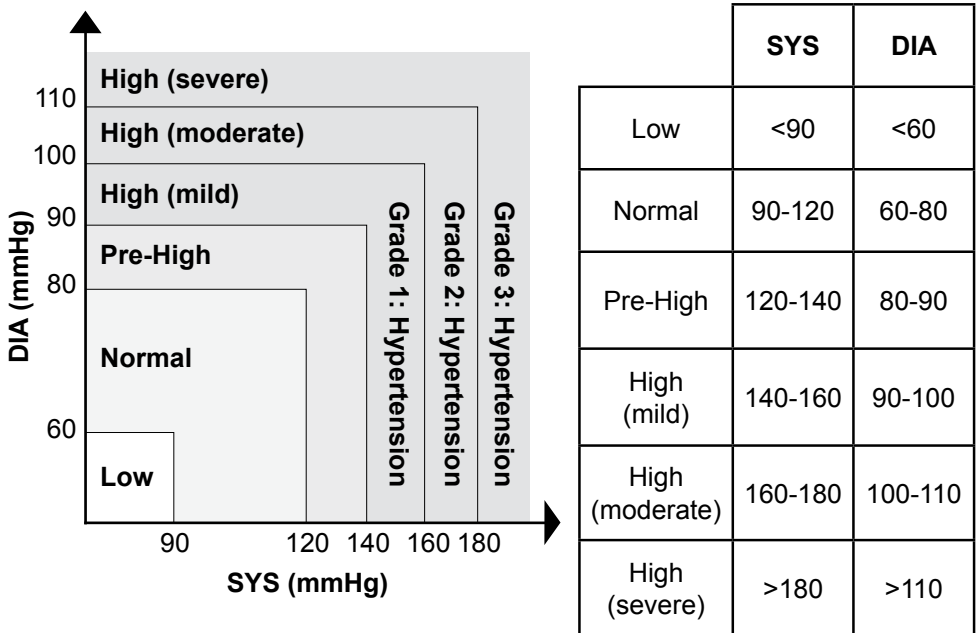Below illustrates the standard blood pressure classification for adults made by World Health Organisation (WHO) and International Society of Hypertension (ISH) in 1999.

**Caution:**

- Only a doctor can tell you your normal blood pressure range and the point at which you are at risk. Consult your doctor to obtain these values.
- If the measurements taken with these products fall outside the range, consult your GP/doctor for advice.

## Troubleshooting

This section includes a list of error messages and frequently asked questions for problems you may encounter with your blood pressure monitor. If the monitor is not operating as you think it should, check here before arranging for servicing.

<b>LCD Display Information</b>	<b>Reason / Meaning</b>
Err is displayed in the SYS area, 1 is displayed in the DIA area	The leakage is too fast or the pulse signal is too weak. Please check the cuff, re-fasten it around your arm and try again.
Err is displayed in the high SYS area, 2 is displayed in the DIA area.	The blood pressure signal cannot be detected due to too much noise. Please remove the noise sources and measure again.
Err is displayed in the SYS area, 3 is displayed in the DIA area.	The test result is abnormal. Please measure again.
Err is displayed in the SYS area, P is displayed in the DIA area.	The inflation fails. Please check the cuff, re-fasten it around your arm and try again.
HI is displayed in the DIA area.	The inflation pressure is greater than $307 \pm 8$ mmHg ( $41 \text{ kPa} \pm 1 \text{ kPa}$ ). Please measure again.

## Troubleshooting

Problem	Check/Test	Solution
<b>No Display when pressing 'START/STOP' Button</b>	Check whether the batteries are low	Replace batteries or plug Micro USB cable into device
	Are the batteries inserted correctly?	Install in accordance with the positive and negative poles of the batteries
<b>No pressure in the cuff</b>	Is it plugged in correctly?	Insert the cuff into the air jack tightly
	Is the cuff or tube broken or leaking?	Buy a new cuff
<b>Error display, unable to measure</b>	Did your arm move when taking the reading?	Remain still and try again
	Did you talk when taking the reading?	Keep quiet and try again
<b>The cuff leaks</b>	Is the cuff wrapped too loose?	Please tighten the cuff
	The airbag in the cuff is broken	Replace with a new cuff

**Caution:** *If blood pressure still cannot be measured normally with the above method, please contact Durovic. Do not disassemble it by yourself.*

**Product Specifications**

<b>Product Name</b>	Electronic Blood Pressure Monitor	
<b>Model</b>	BPM200	
<b>Display Mode</b>	Digital Display	
<b>Measuring Method</b>	Oscillographic	
<b>Measuring Part</b>	Upper Arm	
<b>Measuring Range</b>	Blood Pressure Value	30mmHg - 255mmHg (5.3kPa-37.3kPa)
	Pulse Rate	40-199 pulse rates / minute
<b>Static measurement accuracy</b>	Pressure Value	±3mmHg (±0. 4kPa)
	Pulse Rate	±5% of the read value
<b>LCD Display</b>	Pressure	Unit: mmHg / kPa.
	Pulse	Pulse rate per minute, showing three digits
<b>Memory Capacity</b>	99 groups of measurement data for each of two users.	
<b>Power Supply</b>	4 AAA dry batteries or DC 5V USB external power supply	
<b>Power off mode</b>	Manual shutdown / 30s delayed automatic shutdown	

<b>Body weight (excluding batteries)</b>	About 195g±5g		
<b>Dimensions</b>	120mm(length) 90mm(width) 43mm(height)		
<b>Screen Size</b>	59mm(length) 47mm(width)		
<b>Cuff</b>	Upper arm cuff (measuring arm circumference 22-36cm or 22-42cm )		
<b>Accessories</b>	Cuff, User manual		
<b>Battery Life</b>	High performance dry batteries can be used about 300 times under		
<b>Service Life</b>	5 years		
<b>Date of environment</b>	See the label		
<b>Working environment</b>	Temperature Condition	5°C - 40°C	If the product is stored or used outside the specified temperature and humidity range, it will not work properly.
	Humidity Condition	15% - 90%RH	
	Atmospheric Condition	70kPa - 106kPa	
<b>Storage and transportation environment</b>	<p>Strong impact, direct collision, exposure to the sunlight or rain should be avoided during transportation.</p> <p>The packaged blood pressure monitor should be stored in an environment with temperature ranging from -20°C to 55°C, relative humidity ranging from 10% to 93%, and atmospheric pressure ranging from 70 kPa to 106 kPa, with no corrosive gas and positive ventilation.</p>		

**EMC Guidance**

Table 1 Guidance and MANUFACTURER’s declaration – ELECTROMAGNETIC EMISSIONS- for all ME EQUIPMENT and ME SYSTEMS

<b>Guidance and manufacturer’s declaration – electromagnetic emissions</b>		
<p>The BPM200 Electronic Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the Blood Pressure Monitor should assure that it is used in such an environment.</p>		
<b>Emissions test</b>	<b>Compliance</b>	<b>Electromagnetic environment - guidance</b>
<p>RF emissions CISPR 11</p>	<p>Group 1</p>	<p>The BPM200 Electronic Blood Pressure Monitor uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.</p>
<p>RF emissions CISPR 11</p>	<p>Class B</p>	<p>The BMP200 Electronic Blood Pressure Monitor is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.</p>
<p>Harmonic emissions IEC 61000-3-2</p>	<p>Not applicable</p>	
<p>Voltage fluctuations/ flicker emissions IEC 61000-3-3</p>	<p>Not applicable</p>	

Table 2 Guidance and MANUFACTURER's declaration – ELECTROMAGNETIC IMMUNITY – for all ME EQUIPMENT and ME SYSTEMS

<b>Guidance and manufacturer's declaration – electromagnetic immunity</b>			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
<b>IMMUNITY test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment - guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV line(s) to line(s)	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 s	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

Table 4 Guidance and MANUFACTURER’s declaration – ELECTROMAGNETIC IMMUNITY – for ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacturer’s declaration – electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3 Electrical fast transient/burst IEC 61000-4-4	3 Vrms 150 kHz to 80 MHz  3 V/m 80 MHz to 2.5 GHz	3 Vrms  3 V/m	<p>Portable and mobile RFvcommunications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> $d=1.167\sqrt{P}$ $d=1.167\sqrt{P} \text{ 80MHz to 800MHz}$ $d=2.333\sqrt{P} \text{ 800MHz to 2,5GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup></p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the device.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

Table 6 Recommended separation distances between portable and mobile RF communications equipment and the ME EQUIPMENT or ME SYSTEM – for ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

<b>Recommended separation distances between portable and mobile RF communications equipment and the device.</b>			
The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.			
<b>Rated maximum output power of transmitter (W)</b>	<b>Separation distance according to frequency Rated maximum output of transmitter (m)</b>		
	150 kHz to 80 MHz $d=1.167\sqrt{P}$	80 MHz to 800 MHz $d=1.167\sqrt{P}$	800 MHz to 2.5 GHz $d=2.333\sqrt{P}$
0.01	0.117	0.117	0.233
0.1	0.369	0.369	0.739
1	1.167	1.167	2.333
10	3.690	3.690	7.378
100	11.67	11.67	23.33
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. NOTE 1 At 80MHz and 800MHz, the separation distance for the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

# Warranty

1 Year Guarantee from Shine-Mart Ltd, Trademark owners of Duronic Products.  
 NOTE: THESE STATEMENTS DO NOT AFFECT YOUR STATUTORY RIGHTS AS A CONSUMER

This product has been manufactured under the strictest or quality control procedures, and using the highest quality of materials, to ensure excellent performance and reliability. It will give very good and long lasting service, provided it is properly used and maintained. The product is guaranteed for 1 years from the date of original purchase. If any defect arises due to a faulty materials or poor workmanship, the faulty product must be returned to the original place of purchase. Refund or replacement is at the discretion of that company. Duronic Products are offered with a 1 year guarantee under the following conditions:

1. The product must be returned to the retailer with original proof of purchase.
2. The product must be installed and used in accordance with the instructions contained in this operating manual.
3. It must be used only for domestic purposes.
4. It does not cover wear and tear, malicious damage, misuse, neglect, inexpert repairs or consumable parts.
5. Shine-Mart Ltd has no responsibilities for accidental or consequential loss or damage.
6. Shine-Mart Ltd are not liable to carry out any type of servicing work, under the guarantee
7. Valid in the UK only

This product is fitted with CE:1363 plug and fuse  
 Imported by Shine-Mart Ltd, RM3 8SB

Information on waste Disposal for Consumers of  
 Electrical & Electronic Equipment.

This mark on a product and/or accompanying documents indicates that when it is to be disposed of, it must be treated as Waste Electrical & Electronic Equipment, (WEEE).

Any WEEE marked products must not be mixed with general household waste, but kept separate for the treatment, recovery and recycling of the materials used.

For proper treatment, recovery and recycling; please take all WEEE marked waste to your Local Authority Civic waste site, where it will be accepted free of charge.

If customers dispose of Waste Electrical & Electronic Equipment correctly, they will be helping to save valuable resources and preventing any potential negative effects upon human health and the environment, of any hazardous materials that the waste may contain.

## Follow Us

[duronic.com](http://duronic.com)

[duronic](https://www.youtube.com/duronic)

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	THE OPERATION GUIDE MUST BE READ		Type BF Applied Parts
	This product complies with the MDD93/42/EEC requirements.		DISPOSAL: Do not dispose this product as unsorted municipal waste. Collection of such waste separately for special treatment is necessary.
	Manufacturer		Direct Current
	Specifies Serial Number		Wellkang Ltd Suite B, 29 Harley Street, London, W1G 9QR,UK
	Manufacture Date		Degree of protection against the ingress of water.

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5. Shine-Mart Ltd heeft geen verantwoordelijkheid voor accidentele schade of verlies.
6. Shine-Mart Ltd is niet aansprakelijk voor onderhoud onder de garantie.
Geïmporteerd door Shine-Mart Ltd, RM3 8SB