




AUDIMAX II+

SOFTWARE FOR SA 201-204

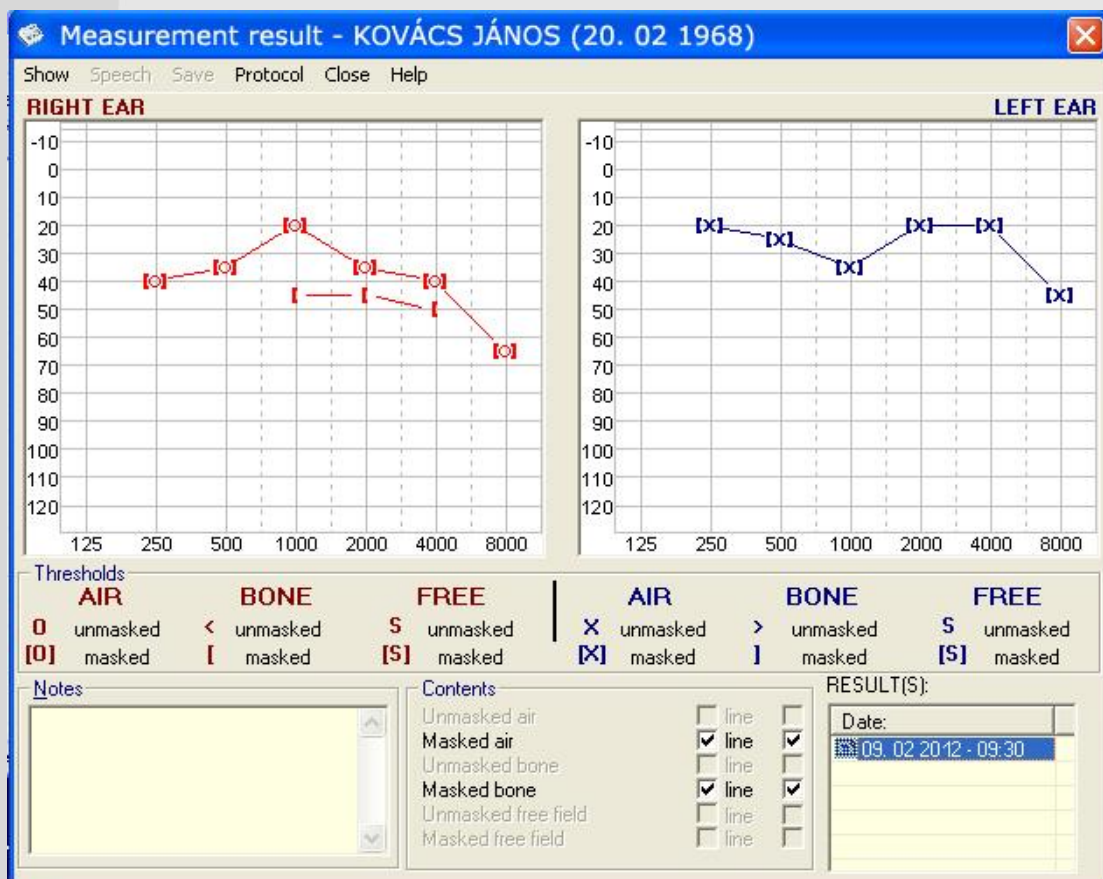

AudiMax II+ is a Windows based application for SA 201-204 pure tone audiometers.

The audiometers can store audiograms together with corresponding patient ID in their internal non volatile memory. AudiMax II+ allows the stored test results to be downloaded to a PC through the standard RS 232 serial port. Downloaded audiograms can be viewed on the screen and saved to patients in the AudiMax II+ database. Final report printing facility and possibility of comparing different test results of the same patient are also available. AudiMax II+ can be run either as an individual application or under Kristály 1.1 occupational health care and Ursula 2.1 general practice softwares. Three different languages (English, Spanish and Hungarian) are available, support for other languages coming shortly (Q2 2012) Easy operation is supported by online help menu.

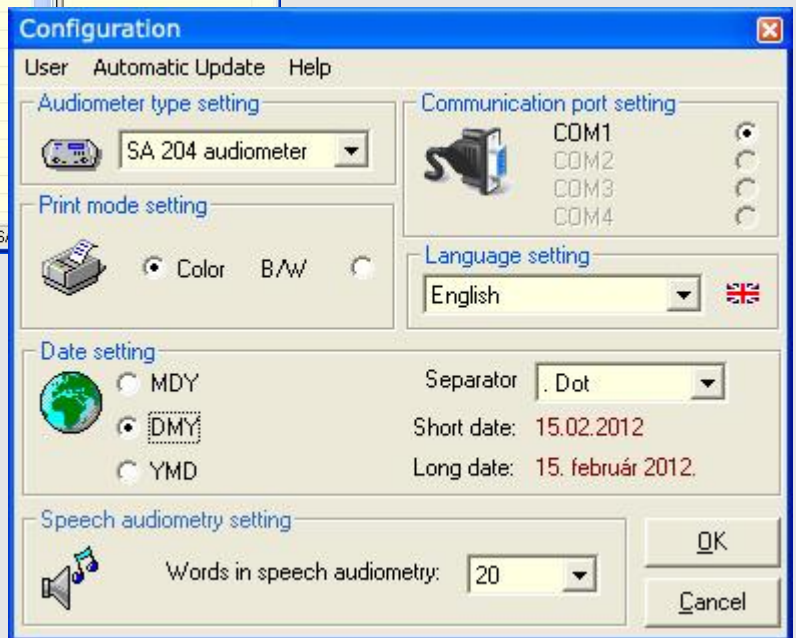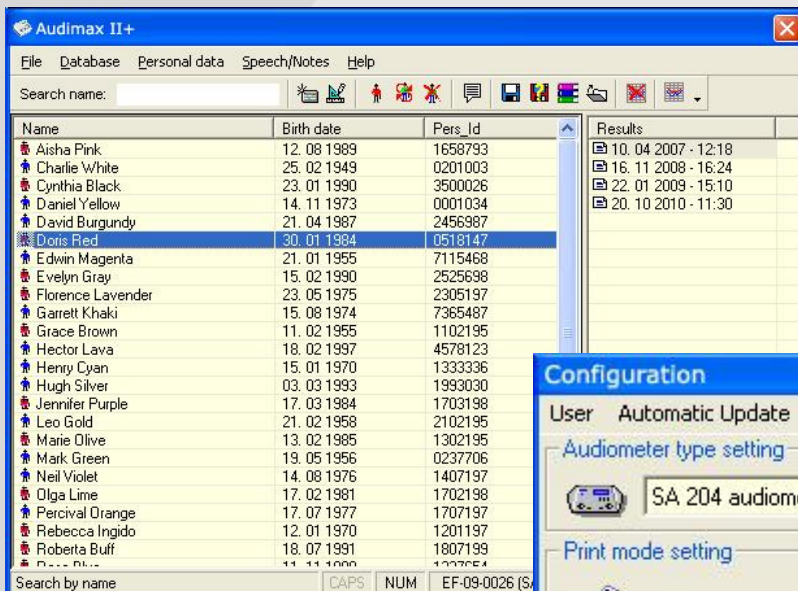

Ordering information

EN 101 181 - Audimax II+ for SA 201E/SA 202
 EN 101 211 - Audimax II+ for SA 203/SA 204

Accessories supplied

EN 200 235 – RS232 serial cable
 EN 200 236 – USB/RS232 adapter

Hardware and software requirements

1 GHz 32-bit (x86) or 64-bit (x64) processor
 512 MB of system memory
 50 MB available hard disc space
 32 bit or 64 bit Windows operating system
 Printer (b/w or colour) for protocol printing (optional)
 Internet access for automatic software update (optional)

Audiometers – works with following audiometers

SA 202-II/GSI66/SA 201E-IV/SA 202-IV
 SA 203-II/GSI67/SA 204-II/GSI 68/SA 203-IV/SA 204-IV



The SA 20X-IV generation of the Entomed audiometers is digital and RoHS compatible and thus, well adapted to the modern user and environmental concerns. All products in the SA 20X-series are user friendly, robust, reliable and yet affordable. Entomed SA20X-Series covers user needs from basic screening capabilities to advanced diagnostic functionalities required by audiologists dispensing hearing aids.

SA 201™ - Screening Audiometer
 SA 202™ - Screening Audiometer
 SA 203™ - Diagnostic Audiometer
 SA 204™ - Diagnostic Audiometer

The SA 20X-IV series is replacing the previous platform SA 20X-II also sold under the brand names GSI 66, GSI 67 and GSI 68.

For further inquiries do please visit Entomed website or contact your local dealer.

Entomed AB
 Malmö, Sweden

TecDoc312 Rev 1

Entomed
 Jägersrovägen 204
 SE-213 77 Malmö, Sweden
 Tel: +46 40 21 00 90
 Fax: +46 40 21 00 99
 E-mail: mailbox@entomed.se
 Web: www.entomed.se

To order or for more information, please contact:
docStock

MEDICAL EQUIPMENT SPECIALISTS™

DocStock Pty Ltd

ABN 48 115 268 203

P: (02) 8850 7177

E: info@docstock.com.au

www.DocStock.com.au