

Healthcare

VITALOGRAPH Alpha

Changing the way you look at desktop spirometry.



A Global Leader in Respiratory Diagnostics

 **vitalograph**[®]

vitalograph.com

VITALOGRAPH Alpha

The next generation touchscreen diagnostic desktop spirometer.

Whether you are testing on the go, remotely or in a clinic, the Alpha is the ultimate desktop spirometry solution.

The new Alpha encompasses compliance with the ATS/ERS 2019 spirometry guidelines, multiple testing options, quick loading integrated thermal printer and direct links to Vitalograph Device Studio to generate PDF reports. Vitalograph Connect provides versatile connectivity to your electronic medical records, taking spirometry testing to the next level.



Features and Benefits

- Meets **2019 ATS/ERS standardization of spirometry** ensuring that testing meets the most up-to-date, internationally recognised guidelines.
- The updated test quality grading system and **live testing prompts** give you the power to get the best quality results possible.
- Supports **Full Feature Spirometry Testing and Bronchodilator Responsiveness Testing** allowing you to take your spirometry clinic wherever you go.
- Quickly evaluate results using **GLI predicted equations** with LLN, %Pred and Z-scores.
- Highly accurate, robust and stable flow measuring technology giving peace of mind for accurate results.
- **Low running costs:** no need for costly disposable sensors, turbines, spirettes or flow tubes.
- **Automatic interpretation** based on ATS/ERS & GOLD standards.
- **Free 5-year warranty** with Alpha product registration.



Keep up to date

Vitalograph Device Studio not only generates PDF reports but also receives the latest firmware updates.

Ensure accuracy

The Alpha features quick and easy calibration verification routines as recommended by 2019 ATS/ERS spirometry guidelines.


Versatile Reporting

Get instant spirometry reports with the fast integrated printer or via Vitalograph Device Studio.

Save time in consultations

It's easy to link EMR systems with the Alpha - Vitalograph Connect middleware software (included as standard).

- Eliminate errors and improve workflow with subject details automatically transferred from your clinical system to the Alpha.
- Read codes and PDF reports are automatically filed to the subject's medical record providing enhanced data safety and saving precious time.

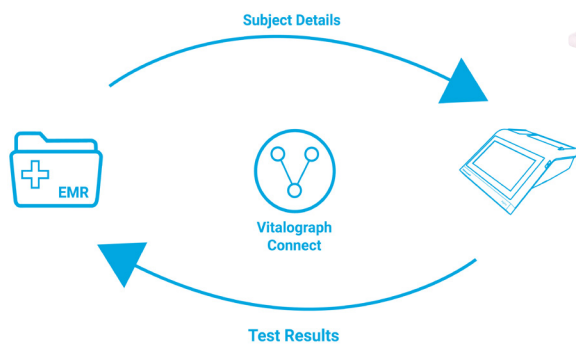

Save time and money on cleaning



Using the Vitalograph Bacterial Viral Filters (BVF™) provides a significant level of protection for the subject, the device and the user against cross contamination during spirometry manoeuvres. When a new BVF is used for every patient, Vitalograph recommends that only the exterior of the device requires cleaning.

Technical Specifications

Product: Vitalograph Model 6000 Alpha

Model: 6000

Dimensions: 204mm (length) x 253mm (width)

Weight: 1.5kg

Communications: USB (Device Studio), Ethernet/WiFi (Connect)

Flow Detection Principal:

Fleisch type pneumotachograph

Essential Performance Test Limits:

Flow Accuracy $\pm 10\%$ or ± 20 L/min, whichever is greater

Volume detection: Flow integration sampling @100Hz

Maximum test duration: 90 seconds

Maximum displayed volume: 10L

Volume accuracy: Better than $\pm 2.5\%$

Power supply: Input 100-240V, 50/60Hz Output 12V, 1.5A, Battery: 7.2V, 2.2Ahr

Accuracy when operated in operating temperature range conditions: Flow $\pm 10\%$, Max. flow rate ± 14 L/s, Min. flow rate ± 0.02 L/s

Performance standards: The Vitalograph Alpha meets or exceeds: ATS/ERS 2019, ISO 23747:2015 & ISO 26782:2009

Printer Type: Thermal

QA/GMP standards: EN ISO 13485, FDA 21 CFR 820, CMDR SOR/98-282 & JPAL, MDSAP Program

Ordering information

Product Model Number	Customer Order Number	Product Description
6000	69002	Vitalograph Model 6000 Alpha

Optional extras / Consumables

Product Model Number	Customer Order Number	Product Description
2040	36020	3-L Precision Syringe
2820	28554	Eco BVF with Bite Lip (75)
	28553	Eco BVF with Bite Lip + Disposable Nose Clip (75)
	28501	Eco BVF (100)
	69140	Thermal Printer Paper (10 rolls)

Vitalograph, Vitalograph® and BVF™ are trademarks or registered trademarks of Vitalograph Ltd. All other trademarks and trade names are those of their respective owners

CE
2797
062976_2

Contact us

Our trained customer service team are available for telephone support on sales, technical queries or information requests.

USA & Canada

Vitalograph Inc.

Toll Free: 1-800 255 6626
Tel: (913) 730 3200
Fax: (913) 730 3232
contact@vitalograph.com
13310 West 99th Street
Lexxa, Kansas, 66215, USA

Ireland

Vitalograph (Ireland) Ltd

Tel: +353 65 686 4100
Fax: +353 65 682 9289
sales@vitalograph.ie
Gort Road Business Park
Ennis, Co Clare,
V95 HFT4, Ireland

United Kingdom

Vitalograph Ltd, UK

Tel: 01280 827110
Fax: 01280 823302
sales@vitalograph.co.uk
Maids Moreton,
Buckingham, MK18 1SW,
England

Germany

Vitalograph GmbH

Tel: +49 40 547391-0
Fax: +49 40 547391-40
info@vitalograph.de
Rellinger Straße 64a
D-20257 Hamburg
Germany

All other countries

**Vitalograph Ltd,
International**

Tel: +44 1280 827120
Fax: +44 1280 823302
sales@vitalograph.co.uk
Maids Moreton,
Buckingham, MK18 1SW,
England

Authorised Dealer Stamp

