



Operation and technique

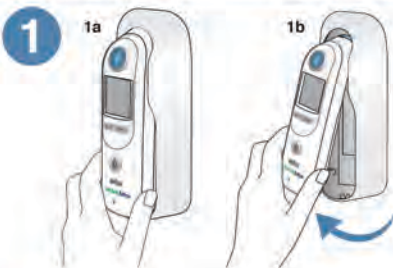


Remove thermometer from cradle by gripping thermometer at base and pivoting up.

Thermometer will turn on automatically. Probe cover icon  will flash on display indicating a new probe cover is needed.



Attach a new probe cover by pushing probe tip straight into box, then pulling thermometer out.




Wait for the ready indication. Ring around MEASURE  button turns green, thermometer will beep once and three lines on display indicate thermometer is ready.



Place probe snugly in ear canal and direct toward opposite temple. Keep thermometer probe steady in the ear canal. Correct probe placement is essential for accurate measurements.




Press and release MEASURE  button. The thermometer will beep once, running dashes will appear on the display then the green ExacTemp light will flash, indicating consistent probe position.



Temperature measurement. A long beep and steady green ExacTemp light will signal the end of the measuring process. The temperature will show on the display.



Remove used probe cover by pressing Probe Cover Ejector button .

To achieve accurate measurements, use a new, clean probe cover for each measurement.

To take another measurement, place a new, clean probe cover on the thermometer.

Additional functions



Memory

Press the MEMORY button to reveal the last completed temperature.




Temperature conversion

Press and release C/F button to view the measurement in the alternate temperature scale temporarily. Press button again to return to default temperature scale.


Default temperature scale can be set with the C/F switch located inside the battery compartment.




Manual timer

Press and hold  button for one second to activate timer. A beep is heard at the initiation of the timer.

First segment will flash until first 15 seconds are complete and then beep. (Repeats for 3 remaining segments.) At 60 seconds a long beep will sound. The thermometer will exit timer mode 5 seconds after last segment.


You can exit timer anytime by pressing  button again.

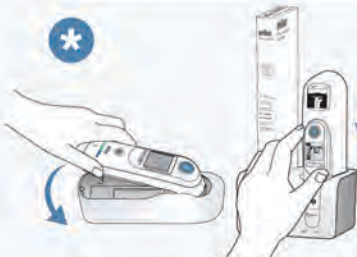


Security and charging

To UNLOCK, thermometer must be returned to charging station or compatible vital signs device.



UNLOCK  icon displays when Security mode is active and counts remaining time available (if under 60 minutes) before thermometer is locked.

When time runs out LOCK icon  remains on display.

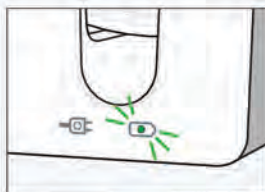

Charge light indicator

Illuminates indicating rechargeable battery is charging. Indicator changes from amber to green when battery is fully charged.