DET30™ PLUS

ENGLISH

FOR GENERAL USE

This mask is not a substitute for a medical / surgical mask or a respirator. It is not suitable for use in a clinical setting where infection risk through inhalation is high. Always observe comprehensive hygiene measures.

ATTENTION

1. Wash before first use (see wash instructions) 2. Washable 30 times 3. Each pack contains two masks to facilitate use on alternate days 4. For personal use only; do not share one mask 5. Do not use if: (i) the mask is damaged, soiled or has odor; (ii) you have any allergic reactions / breathing difficulties; (iii) you are under poor ventilation conditions 6. Not suitable for children under 3 years of age

USE INSTRUCTIONS

1. Clean your hands before and after putting on or removing the mask; avoid touching the surfaces of the mask 2. Ensure the side with the QR code label faces outwards 3. Adjust the stoppers to secure the loops 4. Mold the wire over nose bridge 5. Extend the mask to fully cover nose, mouth and chin 6. Wash after each day of use; allow to dry completely

WASH INSTRUCTIONS

1. Gentle hand wash (wash separately) 2. Soak in lukewarm water (up to 40°C / 104°F) with gentle detergent or soap for 2 minutes; rinse with clean water 3. Drip dry, or tumble dry on delicate cycle 4. Do not use antiseptic detergents or bleach 5. Do not dry clean 6. Do not rub, brush or wring

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