


# XTREME DEFENSE

## FACE SHIELD

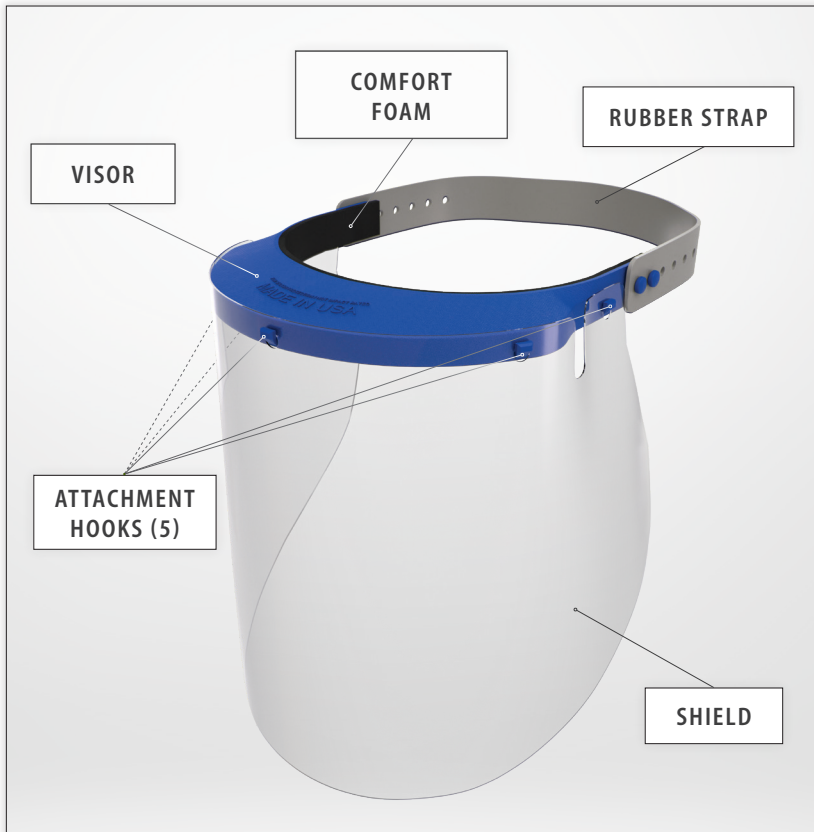

The 180-degree face shield provides best in class protection. Its design features an enclosed top that helps prevent anything entering from above. The anti-fog shield includes two small ports to allow for airflow, which finds the perfect balance between quality, comfort and protection for its users. The face shield includes a closed cell comfort foam to rest on the users' head allowing for maximum comfort and stability. The strap is made with allergy friendly non-latex rubber which is FDA approved. The face shield is adjustable and can fit around heads, hats, bump caps and helmets. The visor, strap and shield are reusable and can be easily cleaned or replaced. All components of the face shield are proudly manufactured and assembled in the USA.

### ASSEMBLY INSTRUCTIONS:

1. Remove the adhesive backing from the **COMFORT FOAM** strip and apply it to the inside edge of the **VISOR**, where it will be in contact with your forehead.

*Note: Kids shields will come with two foam strips.  
Select the foam that will give the best fit.*

2. Remove the protective film from the **both sides** of the shield, making note of the side that has the "B" sticker, this side is scratch resistant and should be facing out.
3. Line up the center hole at the top of the **SHIELD** with the center **ATTACHMENT HOOK** on the **VISOR**, making sure the scratch resistant side is facing outside then push until it snaps into place.
4. Repeat this action with the remaining 4 **ATTACHMENT HOOKS** and holes.
5. Push the holes of the **RUBBER STRAP** over the two large pegs that are on the back right and left side of the **VISOR**. Adjust the strap holes to achieve a comfortable fit.

### PRODUCT FEATURES:

- 180-degree shield protection
- Enclosed top
- Anti-fog shield
- Widely adjustable
- FDA approved non-latex rubber strap
- Comfort foam forehead bumper
- Reusable

**QUALITY. COMFORT. PROTECTION.**



# XTREME DEFENSE

## FACE SHIELD

### **RECOMMENDED CLEANING:**

The best method for cleaning the face shield is soap and water. Gently rub soap solution on all parts of the shield and then rinse thoroughly with water. Shake off any excess water and pat dry with a paper towel.

The face shield can also be cleaned with disinfectants (alcohol, peracetic acid, bleach solution) if needed.

Instead of cleaning the shields, replacement shields can be bought in single packs or bulk to snap into the user's visor. If the cleaning process damages the foam forehead bumper guard, replacements can also be purchased.

### **BODY CONTACTING MATERIALS:**

**COMFORT FOAM:** Closed Cell Foam

**STRAP:** Non-latex rubber, 60 Duro Food Grade Nitrile Rubber

**SHIELD:** Polycarbonate

**VISOR:** Polypropylene copolymer SB912

### **DISCLAIMER:**

The face shield is appropriate for use in healthcare settings, industry or personal wear. Using a face shield will not ensure your safety from catching or spreading airborne/contagious viruses, illnesses or diseases. The face shield is not impact rated and should be used for sanitary use only. The face shield has not been FDA cleared or approved. The face shield has been authorized by the FDA under an emergency use authorization (EUA) for use by healthcare providers as personal protective equipment. This product is only authorized for the duration of the declaration that circumstances justifying the authorization of emergency use under Section 564(b)(1) of the Act, 21 USC 360bbb-3(b)(1) unless the authorization is terminated or revoked.