

SHARING IS CARING



HOW TO WEAR A MASK

FOLLOW THESE EASY STEPS WHEN APPLYING YOUR MASK



1

CLEAN YOUR HANDS WITH SOAP AND WATER OR HAND SANITIZER



2

HOLD THE MASK BY THE EAR LOOPS AND PLACE A LOOP AROUND EACH EAR



3

PULL THE BOTTOM OF THE MASK OVER YOUR MOUTH AND CHIN



4

AVOID TOUCHING THE FRONT OF THE MASK WHEN WEARING

HOW TO REMOVE A MASK

FOLLOW THESE EASY STEPS WHEN REMOVING YOUR MASK



1

CLEAN YOUR HANDS WITH SOAP AND WATER OR HAND SANITIZER



2

AVOID TOUCHING THE FRONT OF THE MASK. ONLY TOUCH THE EAR LOOPS



3

HOLD BOTH OF THE EAR LOOPS AND GENTLY LIFT AND REMOVE THE MASK



4

CLEAN YOUR HANDS WITH SOAP AND WATER OR HAND SANITIZER

DO YOU NEED MORE AMERICAN MADE FACE MASKS?



DON'T WORRY WE'VE GOT YOU COVERED!

SCAN HERE FOR MORE MASKS



WE'RE ALL IN THIS TOGETHER!