

## **Operating Instructions**

- To attach dermatoscope head to the handle, gently push the head downward on the handle top and turn the head clockwise.
- To detach dermatoscope head from the handle, gently push the head downward and then turn the head counter clockwise.
- Installing Batteries: The handle takes 2 Medium (C Size) batteries. Please ensure that + sign on the batteries is inserted in the handle towards the top of handle for both batteries.
- Turning unit On and Off: The handle is equipped with a rheostat switch. To turn on the unit, gently rotate rheostat switch clockwise and to turn off gently rotate rheostat switch counter clockwise.

**VISIT US AT: [WWW.CROSSINSTRUMENTS.COM](http://WWW.CROSSINSTRUMENTS.COM)**