




User Manual

ELECTRIC BABY NASAL ASPIRATOR

Model: BC033

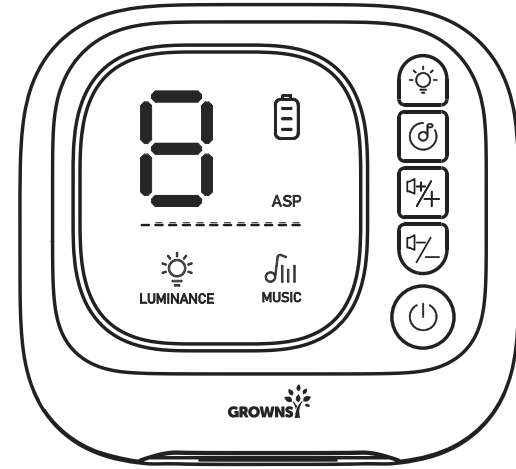

Read this booklet thoroughly before using and save it for future reference

HEARTFELT THANK YOU

Thank you so much for purchasing from us and placing your trust in our brand.

GROWNSY - Parent's best helper. GROWNSY creates modern and thoughtful baby products for parenthood. Starting from smart feeding essentials, GROWNSY provides a broad portfolio for all scenarios related to parents and babies. We are committed to building a one-stop maternal and baby life brand. With the core concept of "making ideal products that express love and care", Grownsy insists on solving most difficult problems during manufacturing and offering the most convenient items to our user. Grownsy's products are modern in design, creative in structure, and smart in use.

For any questions or issues, please email us: service@grownsy.com
Or SMS GROWNSY to (833)524-0498

We will go above and beyond to resolve any customer issue within 24 hours.


Randell
Founder of GROWNSY




Product Features

The electric nasal aspirator is an excellent solution for keeping your baby's nasal passages clear and comfortable.

- With 9 adjustable suction power levels
- 5 music tracks, and the ability to be turned into a white noise machine, it offers a customized and calming experience.
- The gradient lighting helps distract your baby while sucking snot.
- Designed with a separate main unit and suction chamber, and featuring easy cleaning function, this aspirator is thoughtfully crafted with both functionality and convenience in mind.

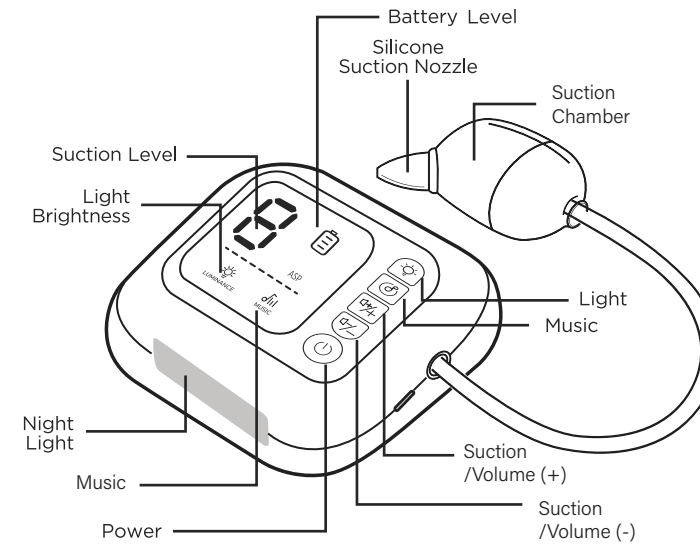

Component Description

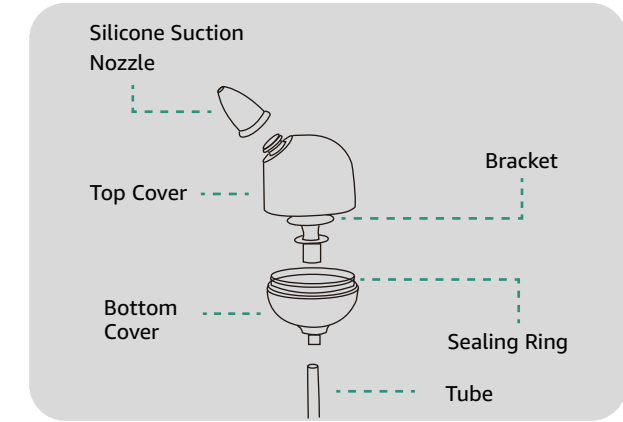

Assemble the Suction Chamber

- 1 Align suction bracket with center of base and insert tightly
- 2 Fit the Top lid onto the base
- 3 Choose appropriate nozzle and fit on top of the suction chamber cap

Operating Instruction

POWER BUTTON

Press and hold the **POWER BUTTON** for 2 seconds to turn the device on/off, and short press the button to pause the device. Once the device is turned on, the default suction gear will be set at Level 3, and the music and night light will remain turned off by default.

NIGHT LIGHT:

After turning on the device, long press the night light button to activate the night light, which displays gradient colors. A short press will fix the light to a single color, while another short press will switch it back to the gradient colors. Long press the button again to turn off the light.

MUSIC FUNCTION:

After turning on the device, long press the music button to start playing music. Short press to switch between songs. Long press the music button again to turn off the music.

Note: MUSIC and LIGHT buttons can be individually turned on or off. However, when the product is powered on, if you long-press the power button to shut down, all functions will be turned off.

PLUS/MINUS:


Short press to adjust the suction power levels, and long press to adjust the music volume.

BATTERY LEVEL:


The battery level indicator icon will flash when the power is low. When the device is charging, the icon will continue to flash at a slower pace, and once it is fully charged, the indicator will remain turned on.

4


Suction Chamber Cleanse

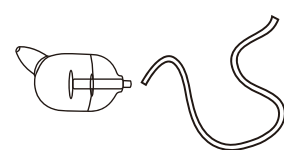


1 Remove the suction chamber from the tube

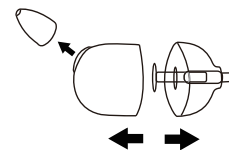

2 Detach the silicone nozzle and open the lid on the suction bin

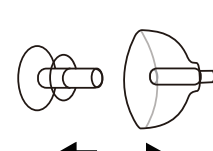

3 Remove the bracket in the suction bin

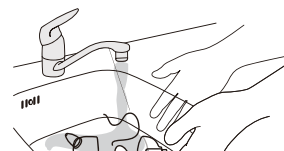

4 Wash the entire component with clean water

(If disinfection is needed, place the component in boiling water for about 3 minutes.)

5

Precautions

After use, please remove the suction chamber to clean. Please use a 5V adapter to charge the device. A adapter over 5V CANNOT charge the device.

Do NOT use the nasal aspirator while charging.

Only use the Baby Nasal Aspirator to relieve nasal cavity congestion. Do not use the device on the ears or throat.

Keep Out of Reach of Children.

PRODUCT SPECIFICATIONS

Item Name	Electric Baby Nasal Aspirator
Model	BC033
Input Voltage	5V--1A
Rated power	4W
Noise Limit	< 65dBA
Charging Interface	Type-C
Product Dimensions	130x90x50mm

FOR BEST RESULTS

Gently massage your baby's nose with a few drops of saline solution or a warm wet towel to soften the mucus build-up. Wait for 30 seconds to 1 minute for best results.

6

WARRANTY & CUSTOMER SERVICE

GROWNSY strives for user friendly products for newborn and household items. We aim not only to create a product, but also an item that is filled with care love from us to our customers. If you have any issues at all, please contact us directly (service@grownsy.com) and we will help you out. Or SMS GROWNSY to (833)524-0498



REGISTER YOUR FREE WARRANTY

At GROWNSY we only use the highest quality products standard - but please don't forget to register for your FREE one year warranty just in case...

www.grownsy.com/warranty



7