



# Liquid Pyroneg



# L99z

## Specialist ultrasonic and spray wash detergent

### Description

Liquid Pyroneg is specially formulated for the superior cleaning of medical instruments in ultrasonic and washer disinfectors and for manual cleaning.

### Features

Liquid Pyroneg is a mildly alkaline detergent containing a blend of wetting agents and sequestrants and is formulated specifically for cleaning of medical instruments in theatre and sterile service departments. The product is ideal for the pre-soaking of surgical instruments to remove gross soiling in theatre prior to transfer to sterile services and provides superior results in ultrasonic washers and washer disinfectors.

### Benefits

- Suitable for use in all water conditions.
- Ensures rapid and complete removal of soils.
- Concentrated for greater economy in use.
- Ease of handling in convenient small pack.
- Accurate manual dosing with portion controlled dosing cap.
- Complies with recommendations of AS/NZS 4187:2014 as a suitable instrument wash detergent.
- Registered as a Class I Medical Device, AUST L 367451.

### Application areas

- Healthcare
- Education
- Government





Diversey™



suma®

# Liquid Pyroneg



# L99z

## Specialist ultrasonic and spray wash detergent

### Directions for use

#### Manual or Ultrasonic Washing:

1. Rinse items with cold water.
2. Prepare wash solution of Liquid Pyroneg using standard dosing cap supplied at 1-2 doses (20-40ml)/10L warm water (45-50°C).
3. Soak or brush items.
4. Rinse with fresh water.
5. Items such as laboratory glassware and syringes should then be rinsed with distilled or deionised water.
6. Allow to dry.

#### Washer/Disinfector

Liquid Pyroneg is normally applied using Diversey dosing equipment or integral dosing pumps where fitted at a concentration of 2-4ml/L. Food surfaces must be thoroughly rinsed with potable water before production starts.

#### Technical data

Appearance	Clear purple liquid	pH (neat)	11.5
		Specific gravity	1.04

*The above data is typical of normal production and should not be taken as a specification.*

Product	Pack size	SKU Code	Availability
Suma® Liquid Pyroneg	6x2L	HH12291	AU & NZ

#### Safe handling and storage information

Store in original container. Avoid extremes of temperature. Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet.

#### Product compatibility

Under recommended conditions of use, Suma® Liquid Pyroneg is suitable for use on most materials commonly encountered in the CSSD. Do not however use on alkali sensitive items such as copper, brass, aluminium.

#### Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available online at [www.diversey.com](http://www.diversey.com) or by calling Toll free: Australia, 1800 647 779 or New Zealand, 0800 803 615.

For additional information, please contact us at:  
Diversey Australia Pty Ltd Toll free: 1800 647 779, Diversey New Zealand Ltd Toll free: 0800 803 615 [www.diversey.com](http://www.diversey.com)

© 2018 Diversey, Inc. All Rights Reserved.

Sep-21