

# Vectograph Viewers Guide

Part Number: 585600, 587800, 585700, 587900

The Vectograph viewer involves a few easy steps to complete setup.

1. Insert long white card and position to the center as shown in figure one.
2. If you are using the #587800 VAC Double, remove the two vectographs from the clear plastic holder and slide into the groves both top and bottom with the rear vectograph in the rear groove and the vectograph slide with the solid blue bar in the front groove. All other viewers will require a clear plastic holder.
3. Position the patient and adjust the chair height to ensure the proper angle and 16 inch (30 cm) test distance as shown in figure 3.

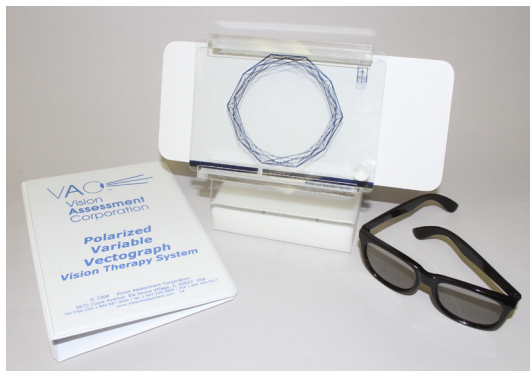


Figure 1

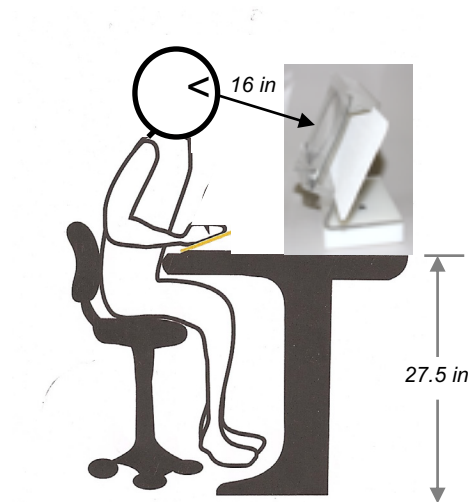


Figure 2