

True Daylight Illuminator

Part Number: 612600



Tray lays flat for arrangement color tests
(Farnsworth D15 Color Test not included)



Tray folds up to a proper 45 degree angle for the use of color
testing/screening books
(HRR Pseudoisochromatic Test, 4th Edition not included)



Storage area on back of illuminator
Dimensions: 10.75" x 7.6875"

The LED Daylight Illuminator provides proper lighting for color vision testing. The key components are the daylight LEDs and the arrangement plate. The LEDs provide color temperature between 6000-7000K and the plate insures the correct distance and visual position for color arrangement and book tests, such as the Farnsworth D15 or HRR color book.

Step 1.

Unwrap and lift the illuminator out of the carton, place on a flat surface.

Step 2.

Examine the contents of the illuminator, power supply, and connector plug.

Step 3.

Insert power supply connector plug in the receiver on the backside of the illuminator head before plugging the power supply into the wall outlet. Slide switch to power on.

Step 4.

When patient is seated, position the illuminator to the correct testing distance and angle from the patient's eyes.

