

056-06-3002 Rev E

Instructions for Use

- Capt-all®Amalgam Capture Devices are designed for single use (one per patient).
- 2. Do not Reuse or Autoclave the Capt-all® device.
- The Capt-all® device is designed for HVE (high volume evacuation) when removing or placing amalgams.*
- Place the short end of the Capt-all® Amalgam
 Capture Device on the HVE valve normally used for HVE tips. (see Figure A)
- 5. When using the capture Capt-all® Amalgam Capture Device to remove or replace amalgam, limit the use of the air water syringe to less than 10 seconds per minute, total water usage including all other devices maximum 100ml/m.
- 6. When the **Capt-all® Amalgam Capture Device** is reaching the maximum fill volume, there will be a noticeable drop in the vacuum rate. This signals the time to change the capture tip.
- 7. After use of the Capt-all® Amalgam Capture
 Device, record the miscellaneous Insurance
 Code D9999 (Amalgam Removal Compliance)
 in the patient's chart for billing and tracking.

- 8. Place the Capt-all® Amalgam Capture Device in the provided lined recycle container. Replace the provided small cap and firmly tighten it on the recycle container. Keep the provided small cap on the recycle container when not in use or in storage.
- When the recycle container is full, visit dovedentalproducts.com/capt-all/recycle or 1-855-800-3683 (DOVE) to receive your prepaid shipping label.



Shipping Label

- 10. After receiving your prepaid shipping label, tighten the small cap securely onto the recycle container. Next place the plug hole cap over the container opening and tape the prepaid shipping label over the plug hole cap. (see **Figure B)**. Call for shipper pickup.
- Our recycler will send you a receipt that you met EPA guidelines for recycling your amalgam waste.
- 12. To reorder **Capt-all®** call customer service at **1-855-800-3683**.

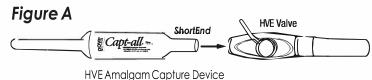


Figure B

*Do not use the cuspidor, saliva ejector or other supplemental suction devices when removing amalgams with Capt-all®. Capt-all® is compliant with 40 C.F.R. § 441.30(a)(2) when used in accordance with Capt-all®'s instructions for use.











Patent No.: 8,393,898 Others Pending Made in the U.S.A.

Reorder Numbers:

3001 RF-25: 25ct Capt-all® without recycle container 3001-25: 25ct Capt-all® with 75ct recycle container 3001-75: 75ct Capt-all® with 75ct recycle container 3001 W: 75ct recycle container replacement

Plug Hole Cap

Small Cap

Small Cap

Capt-all

Capt-all

Small Cap

Capt-all

Recycle

An essential part of your Total Health Environment**

Container

Recycle your Capt-all® container annually or when the container is full - whichever comes first