

Directions for Use

1. **Capt-all[®] Amalgam Tips** are designed for single use (one per patient).
2. **Do not Reuse or Autoclave the Capt-all[®] tip.**
3. The **Capt-all[®]** tip is designed for HVE (high volume evacuation) when removing or placing amalgams.*
4. Place the short end of the **Capt-all[®] Amalgam Capture Tips** on the HVE valve normally used for HVE tips. (see **Figure A**)
5. When the capture tip is reaching the maximum fill volume, there will be a noticeable drop in the vacuum rate, which signals the time to change the capture tip.
6. **Time to replace the capture tip.** The vacuum rate will completely stop when the capture tip is full.
7. After use of the **Capt-all[®] Amalgam Capture Tips**, record the miscellaneous Insurance Code D9999 (Amalgam Removal Compliance) in the patient's chart for billing and tracking.
8. Place the **Capt-all[®] Amalgam Capture Tips** in the provided lined recycle container. Replace the provided small cap and firmly tighten it on the recycle container. **Keep the provided small cap on the recycle container when not in use or in storage.**
9. When the recycle container is full, visit **DoveDentalProducts.com/Capt-all/recycle** or **1-855-800-6383 (DOVE)** to receive your prepaid shipping label.
10. After receiving your prepaid shipping label, tighten the small cap securely onto the recycle container. Next place the plug hole cap over the container opening and tape the prepaid shipping label over the plug hole cap. (see **Figure B**). Call for shipper pickup.
11. Our recycler will send you a receipt that you met EPA guidelines for recycling your amalgam waste.
12. To reorder **Capt-all[®]** call customer service at **1-855-800-6383**.

Figure A


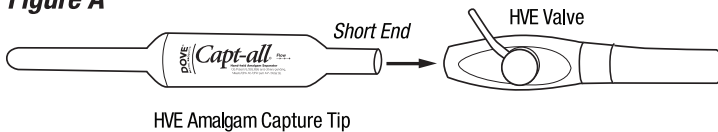


Figure B



* Do not use the cuspidor, saliva ejector or other supplemental suction devices when removing amalgams with **Capt-all[®]**.

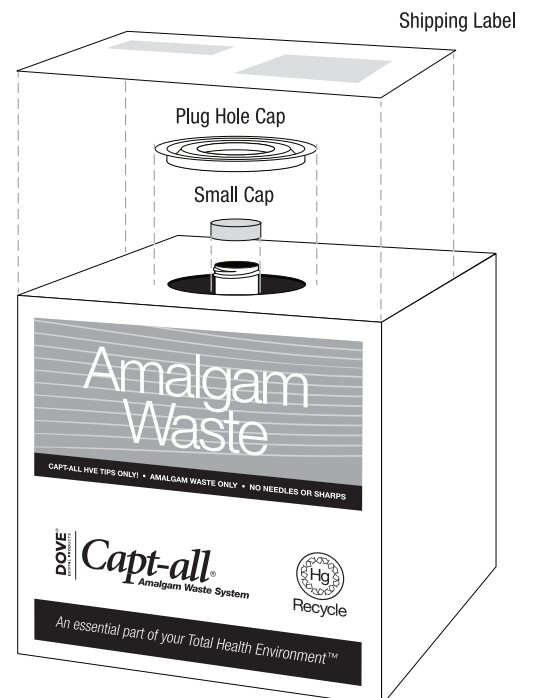

Patent No.: 8,393,898 Others Pending

Made in the U.S.A.

Reorder Numbers:

- 3001RF-25:** 25ct Capt-all without recycle container
3001-25 Recy: 25ct Capt-all with 75ct recycle container
3001-75 Recy: 75ct Capt-all with 75ct recycle container
3001W: 75ct recycle container replacement