

# Hydrofera Blue **READY** Foam

# Hydrofera Blue **READY-TRANSFER** Foam

## WOUND DRESSING APPLICATION GUIDE

Hydrofera Blue **READY** foam.  
**No secondary dressing required.**

Hydrofera Blue **READY-TRANSFER** foam.  
 Secondary dressing required.

### Application

#### Step 1.

- Cleanse the wound in accordance with established procedures
- Carefully dry the periwound skin
- Select and apply the appropriate dressing size to ensure the dressing will cover the entire wound
- No hydration is required



#### Step 2.

- Secure the dressing using a bandage or tape



#### Step 3.

- Dressing can be left in place up to 7 days
- Can be used in conjunction with total contact casting



### Application

#### Step 1.

- Cleanse the wound in accordance with established procedures
- Carefully dry the periwound skin
- Select and apply the appropriate dressing size to ensure the dressing will cover the entire wound
- No hydration is required



#### Step 2.

- Apply an absorptive secondary dressing (e.g. gauze, bordered foam)



#### Step 3.

- Dressing can be left in place up to 7 days
- Can be used in conjunction with compression wraps and total contact casting



### Hydrofera Blue READY

### Hydrofera Blue READY Transfer

Product Code	Description
HBRS2520	Hydrofera Blue READY 6cm x 6cm (2.5" x 2.5")
HBRS4520	Hydrofera Blue READY 10cm x 12cm (4" x 5")
HBRS8820	Hydrofera Blue READY 20cm x 20cm (8" x 8")

Product Code	Description
HBRT2525	Hydrofera Blue READY Transfer 6cm x 6cm (2.5" x 2.5")
HBRT4050	Hydrofera Blue READY Transfer 10cm x 12cm (4" x 5")
HBRT8080	Hydrofera Blue READY Transfer 20cm x 20cm (8" X 8")

### Secondary Dressing Selection Guide

