

**CU-GRS-9R**  
**CU-GRR-9R** (As Shown)

**CU-GRS-9L**  
**CU-GRR-9L** (Assembly "B" Extending to the right)



# INSTALLATION INSTRUCTIONS

*Notes: Where possible, locate Vertical Assembly "A" centered on a wood stud in the wall or install heavy plywood backing in Wall.*

## Step 1: Attach the 2 Grab Bar Assemblies.

While holding the Assembly "A" section of the grab bar, position the connector Nipple on the end of Assembly "B" into the hole in the side of Assembly "A". Screw the Assembly "B" in all the way. Check that the Assembly "A&B" bracket posts are aligned. If needed, turn the bracket on Assembly "B" to make slight adjustment.

## Step 2: Locate and fasten to the wall.

1. Use a stud finder to accurately locate the center of any usable studs.
2. Remove the Cap on the end of the 3 posts to allow opening for mounting screws.
3. Place Grab Bar on wall at desired mounting location and mark screw hole location with pencil. Mark is made by inserting pencil through each post.
4. Drill 5/32" hole into the wall stud or wood backing on each mark.
5. Slip Backplates onto Gra Bar Posts.
6. Fasten the Grab Bar to mounting surface using #12 Pan Head Screw supplied and tighten.
7. Screw caps back into posts.

