



QUICK START GUIDE

Want to get up and running quickly? 4 Quick and Easy Steps

- 1** Power up the Feevr device by first powering up the mobile phone (power button located on top edge).
- 2** Once mobile unit is fully turned on (Samsung logo +/- carrier logo), insert the USB cable that has already been connected to the powerbank, into the bottom of the Feevr device (via the Flir camera) with the appropriate USB-C connector.
- 3** Watch for appropriate color sequence on the power button located on the Flir camera (Solid red to briefly green to blinking green), Should take approximately 12-15 seconds
- 4** If Feevr device prompts user to answer "Do you want to automatically launch the Feevr app once the Flir camera connection is detected?" Answer "Yes!"

If Feevr app automatically launches, you're all set!

If Feevr app does not automatically launch, follow initial prompts (please allow Flir camera access "Always" to the Feevr app, as well as access to pictures) and press the Feevr App icon (Red square with lowercase white "f").

You're ready to go! Allow the thermal imaging technology to adjust to the scene and fully power up (approximately 3 minutes) before calibrating.

In the meantime, please visit our website feevr.tech and/or our direct link to our Standard Operating Procedures here:

[Standard Operating Procedures \(New!\) - Featuring v1.2 Update and Added Bonus Features](#)