



VOLK

How to Use your ClearPod™




flange → patient

The Volk ClearPod™ is intended to reduce lens fogging while performing eye examinations on patients wearing facemasks.

1. Slide your Volk lens down into the ClearPod (as shown above), with the bottom of the “V” in Volk logo facing toward the flange of the ClearPod.

2. Check that the lens is flush with the ledge on the ClearPod (a). Ensure that the ClearPod wings fit snugly around your lens; they should rest between the ridges of your lens ring (b).

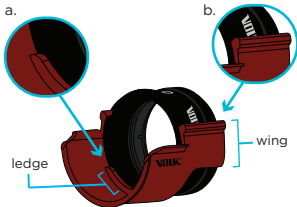

3. With the flange facing toward the patient, proceed with eye examination.



Note: Though the ClearPod design is meant to accommodate the lens-specific working distance, some adjustment may be necessary to obtain the proper working distance as every patient and doctor are unique. This may result in the ClearPod lightly touching the patient's skin.

Caring For Your ClearPod

Your ClearPod should be cleaned and disinfected only using the methods outlined in this booklet.

Cleaning

Cleaning should be performed regularly to remove bodily fluids, dust, and soils from the surface of the ClearPod.

The ClearPod can be cleaned with mild soap or detergent before being rinsed with clean water. The device should be dried using a clean, cotton cloth.

Thoroughly wipe the ClearPod using at least 70% Isopropyl Alcohol (IPA) wipe (DO NOT use this method to clean lens)

Always perform cleaning before disinfecting.

Disinfection

Soak the device in a 0.525% Sodium Hypochlorite (Bleach) solution for 25 minutes. Thoroughly rinse the device in clean water and dry using a clean, cotton cloth.



Caution

- Do not autoclave or use heat to clean the ClearPod
- Do not use the ClearPod if you observe any signs of damage or dirt
- Do not use on injured or broken skin
- Always store your ClearPod in a cool, dry place

Warranty Volk Optical warrants the ClearPod against defects in materials or workmanship for six months or 1000 uses, whichever comes first from the date of receipt by the end user. This warranty does not cover misuse, lack of care, mishandling, accidents, abuse or other abnormal use or degradation due to general wear and tear. For more information on Volk's warranty terms, please refer to the Volk Terms and Conditions of Sale on volk.com.



Volk Optical Inc.
7893 Enterprise Drive
Mentor, Ohio 44060 USA

EC REP

Rudolf Riester GmbH Bruckstraße 31
72417 Jungingen, Germany
Email: info@riester.de
Phone: +49 74 77 / 92 70-0



Toll Free USA: +1 (800) 345-8655

Phone: +1 (440) 942-6161

Fax: +1 (440) 942-2257

Email: volk@volk.com

Website: www.volk.com