

# HeroShield Instructions

<b>Product:</b>	HeroShield	<b>Manufacturer:</b>
<b>Model No.(s):</b>	FS-001-A, FS-001-B - Colours may vary	Denroy Plastics LTD, 9-11 Balloo Drive, Balloo Industrial Estate, Bangor, Co. Down, BT19 7QY
<b>Classification:</b>	PPE Class I of REGULATION (EU) 2016/425	<a href="http://www.denroy.com">www.denroy.com</a>

## Assembly



**1)** Peel away any film attached to the shield component. Film may be on both sides of the shield material and may be either blue or clear



**2)** Attach the shield to the visor band (colour may vary) working from one outside corner to the last corner and securing all 4 holes in place on the off-stands of the visor brim



**3)** Feed the ball chain through the key-hole from above and adjust to fit comfortably - do not overtighten (for your comfort)

## Terms of Supply

The Recipient acknowledges and agrees that the Goods are supplied on an "as-is" basis, and while the Goods have been manufactured using reasonable care and skill, to the maximum extent permitted by applicable law, the Suppliers hereby exclude from these Conditions all warranties in respect of the Goods, express or implied, including but not limited to sections 13 to 15 of the Sale of Goods Act 1979. Full terms of supply can be found at [www.denpro.co.uk](http://www.denpro.co.uk)

## Use and Care Instructions

The HeroShield is intended to be worn by Health Care Professionals to protect the face from liquid splashes and bodily fluids when treating infectious patients. It is not designed to prevent acute respiratory infections (ARIs) or to protect from impacts. It is highly recommended that eye protection and a FFP3 Respirator are used with the Hero Face Shield when appropriate to protect against respiratory infections or diseases which may be present. The user should follow all safety protocols as recommended by their specific organisation during all **Aerosol Generating Procedures (AGP)**.

The visor can be used as short-term wear of up to 8 hours. A reused headband should be used by the same person only. Do not share or reuse headbands between different people under any circumstances.

## Maintenance and Servicing:

### low risk use

HeroShield can be disinfected with alcohol wipes or isopropyl alcohol. The headband component can be disinfected with boiling water if other methods are unavailable.

### For Health care / high risk use

The band and visor may be sterilised for reuse by appropriate hospital (low temperature) procedures. Alternatively, HeroShield can be incinerated/ disposed with medical waste.

Store at room temperature

<b>Part Numbers</b>	FS-001	
<b>Materials</b>	Main Headband	Polypropylene
	PET shield	Minimum 0.3mm PET sheet
<b>Size adjustment range</b>	46cm to 65 cm head circumference, visor is offset from face by 50mm to allow space for glasses wearers and prevent fogging	
<b>Other material safety information</b>	PVC free Phthalate free BPA free	
<b>Conformity</b>	PPE Class I of REGULATION (EU) 2016/425 FDA Coronavirus Disease 2019 (COVID-19) Emergency Use Authorizations for Medical Devices	

# EU DECLARATION OF CONFORMITY

**Product:** Hero Face Shield  
**Model No.(s):** FS-001-A, FS-001-B  
**Date of issue:** 24 March 2020  
**Manufacturer:** Denroy Plastics LTD  
**Manufacturer address:** 9-11 Balloo Drive,  
Balloo Industrial Estate,  
Bangor,  
Co. Down,  
BT19 7QY

The HeroShield is intended to be worn by HealthCare Professionals to protect the face from liquid splashes and bodily fluids when treating infectious patients.

**This declaration of conformity is issued under the sole responsibility of the manufacturer, in accordance with the following Directive(s):**

REGULATION (EU) 2016/425 - Personal Protective Equipment (Classification: PPE Class I)

**Harmonised Standards Applied:**

N/A

HeroShield Limited hereby declare that the equipment named has been assessed and found to comply with the relevant sections of the above referenced specifications.

The unit complies with relevant Essential Requirements of the Directive.



Fiona Bennington  
Head of Research & Development

**Place of issue:**

HeroShield Limited  
17 Jubilee Road,  
Newtownards,  
BT23 4YH  
Northern Ireland  
Tel: 028 9181 5169