
How to use your Remi whitening gel



Thank you for choosing Remi. Here are a few tips:



Spread a thin layer of gel across the top and bottom trays. Bite down fully, and gently wipe away any excess gel from your gums. For best results, wear trays overnight for 14 consecutive nights.



Put your whitening trays in at least 30 minutes before bed. By the time you fall asleep you will barely notice them.



To clean, rinse your whitening trays after use, shake/pat dry.

To maintain your pearly whites, repeat every 3 months. For additional gel or custom whitening trays, visit us at shopremi.com. Use promo code JOINREMI20 for 20% off your purchase.

Have other questions, comments, or requests?
Email us at smiles@shopremi.com.

For more tips and lots of smiles, follow us on our channels.



@remitribe



facebook.com/remitribe



@remi_tribe



remi