

HYDROPEPTIDE ENHANCED SIGNATURE HYDRAFACIAL

TREATMENT TIME:
30 minutes

HYDROPEPTIDE PRODUCTS:

CLEANSING GEL
POWER SERUM
EYE AUTHORITY

CHOOSE ONE:
FACE LIFT
POWER LIFT
POWER LUXE

CHOOSE ONE:
SOLAR DEFENSE TINTED
SOLAR DEFENSE NON-TINTED

PERFECTING GLOSS
LIP SERVICE

*REFERENCE YOUR
HYDRAFACIAL
PROTOCOL FOR
DIRECTIONS ON
MOVEMENTS WITH
MACHINE

PRE-CLEANSE : 3 minutes

½ tsp. CLEANSING GEL

Perform cleansing facial massage with damp hands.
Remove with warm towel or facial sponges.

CLEANSE + PEEL: 7 minutes

HydraFacial® Machine
ACTIV-4

GLYSAL™ PEEL

Refer to original HydraFacial® protocol.
Cleanse with Active-4 cleanser with your HydraFacial machine. Use blue tip.
Peel with GlySal™ with your HydraFacial machine. Use teal tip.

EXTRACT + HYDRATE: 7 minutes

HydraFacial® Machine
BETA-HD™ CLEAR

Refer to original HydraFacial® protocol.
Extract and hydrate with the Beta-HD™ Clear with your HydraFacial machine. Use same teal tip used for Peel step.

FUSE + PROTECT: 7 minutes

HydraFacial® Machine

Refer to original HydraFacial® protocol.

Fuse with the HydroPeptide Power Serum for HydraFacial booster with your HydraFacial machine. Use clear tip.

ANTIOX+

Protect with Antiox+ with your HydraFacial® machine. Use clear tip.

TREAT: 1 minute

⅓ tsp. POWER SERUM

Apply to entire facial area.

EYE REPAIR: 1 minute

½ tsp. EYE AUTHORITY

Apply to lower and outer orbital areas.

MOISTURIZE: 1 minute

¼ tsp.
FACE LIFT or
POWER LIFT or
POWER LUXE

Apply to entire facial area.

PROTECT: 1 minute

¼ tsp. SOLAR DEFENSE TINTED
¼ tsp. SOLAR DEFENSE NON-TINTED

Choose SPF based on client's preference. Apply to client's entire face and massage until blended.

LIP TREATMENT: 1 minute

⅓ tsp. LIP SERVICE and/or

⅓ tsp. PERFECTING GLOSS

Apply a small portion of chosen lip product within lip line. If using both, apply Lip Service first.

