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## USER MANUAL

**ULTIMATE**  
*toning roll*

Fascial Releaser  
XI PROFESSIONAL

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**ULTIMATE**  
*toning roll*

Fascial Releaser  
XI PROFESSIONAL

Congratulations on purchasing the ULTIMATE TONING ROLL / the Fascial Releaser Pro from kybun Joya. The product is marketed under several names, but for the sake of simplicity the name Fascial Releaser Pro will not be used in this user manual.

The ULTIMATE TONING ROLL from kybun Joya is a professional and robust appliance, Made in Switzerland. The massage components provide the ideal amount of vibration to improve general well-being.

Please read this user manual carefully before use and keep it in a safe place for future reference.

## Content

- 11 Important safety information
- 12 Description of the appliance
- 12 Transport / Moving the appliance
- 13 Setting up the appliance
- 13 Mains connection / Turning on/off / Operation
- 14 How to use the appliance
- 15 Cleaning / Maintenance
- 15 Technical details
- 16 Warranty

## Safety information

When using an electrical appliance, certain basic precautions must always be observed. They include:

### Hazard

As a safety precaution and to reduce the risk of electric shock, always remove the plug from the socket after exercising. This also applies when cleaning the appliance.

- Never remove the appliance cover.
- Never use the appliance if a cable or plug is defective, if the appliance has been damaged or dropped into water.
- Do not insert any objects into the openings and do not allow anything to fall inside the appliance.
- Keep the power cord away from hot surfaces.
- Do not use or leave the appliance outdoors. Never allow the appliance to come into contact with water.
- Keep all parts of the appliance, e.g. power cord and on/off switch, away from liquids.
- Only use the appliance for the purposes described in this User Manual.

### Attention

Only connect the appliance to an earthed socket.

## Personal safety

Make sure the appliance is placed and used on a flat and stable surface.

Tie long hair back.



Always inspect the appliance for damage, especially the flexible massage unit cover before use. If cracks or holes are visible in the cover, do not use the appliance for safety reasons (risk of pinching). In this case, the cover must be replaced by a qualified member of staff or the appliance manufacturer.

## Description

The ULTIMATE TONING ROLL from kybun Joya is an appliance designed for professional application. It is robust and very easy to use.

The speed of the drive is infinitely variable, thereby allowing full adjustment of the massage unit impulse rate.

- 1 Massage width (> 50 cm)
- 2 Handles
- 3 Transport wheels
- 4 Speed regulator
- 5 Mains connection & on/off switch
- 6 Anti-slip pad
- 7 Type plate

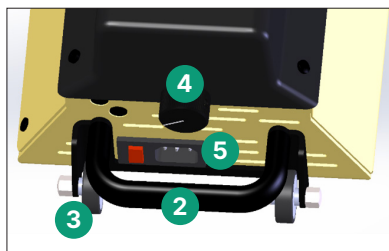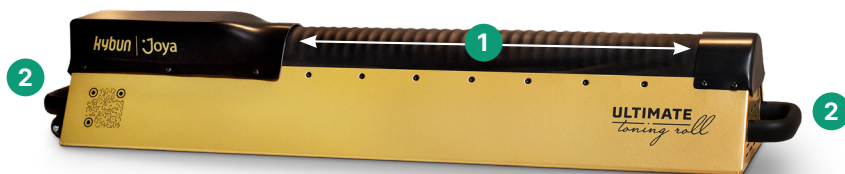## Transport / Moving the appliance

### Attention

The appliance is quite heavy (approx. 25 kg). The wheels on one end of the appliance makes moving it around much easier. Simply use the handle at the other end to lift the appliance and easily pull it along.

If the appliance nevertheless needs to be picked up, it should only be carried by a physically fit person using correct lifting techniques.

## Setting up the appliance

Place the appliance on a flat, non-slip surface (preferably vibration-free) that can support its weight. Do not expose the appliance to high levels of humidity and moisture.

## Mains connection

Connect the supplied power cord to the power socket on the appliance and insert the plug into a mains socket. (Pay attention to the specifications on the type plate)

### Attention

The electrical connection must be earthed!


## Turning on/off / Operation

Use the rocker switch next to the power socket **5** to switch the appliance on. If the control lamp lights up, the appliance is ready for use.

Use the rotary knob switch **4** to start the massage mechanism and to control the speed. Note: It is quite normal for the motor to start only after the rotary knob switch has made a half-turn clockwise. However, the starting speed can be reduced by turning the rotary knob switch anticlockwise.

At a certain anticlockwise threshold, the motor stops. If the rotary knob switch is turned clockwise again, the motor starts up in the opposite direction of rotation.

Once you have finished exercising, turn the rotary knob switch anticlockwise as far as it will go to stop the appliance and then turn off the rocker switch **5**. The control lamp must go out.

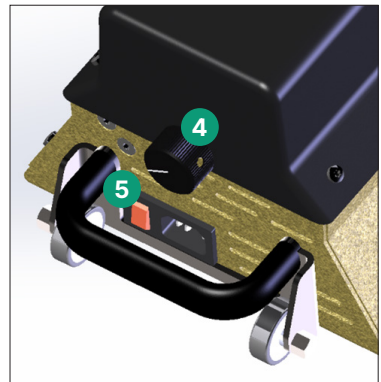## How to use the appliance

The ULTIMATE TONING ROLL from kybun Joya can be used for the following purposes:

- Increasing mobility and flexibility in addition to yoga and other muscle length training exercises
- Releasing fascial adhesion as part of a therapy plan
- Pain relief for people with foot, knee, hip, back or neck problems
- Rehabilitation when recovering from one of the aforesaid problems
- Faster recovery after intense physical activity
- Preventing injuries

Please read the following page for more information:

[www.kybun-joya.swiss](http://www.kybun-joya.swiss)

Or take a look at our exercise videos:


## Cleaning / Maintenance

The appliance is basically maintenance-free. Note, however, that the flexible cover over the massage unit must be in perfect condition to ensure safe operation. If the cover shows signs of increased wear or even cracks or holes, do not use the appliance for safety reasons until the cover has been replaced. In this case, contact the seller or kybun.

Clean the appliance with a soft, slightly damp cloth and a mild cleaning liquid.

## Technical details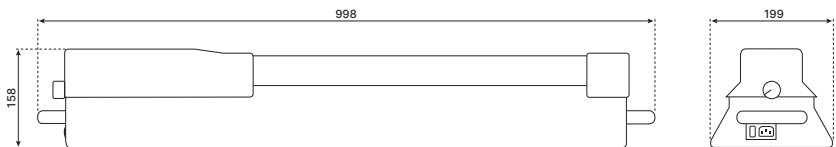

Power supply	100 ÷ 240 VAC 50/60 Hz
Power input	340 W
Peak motor output	600 W
Weight of appliance	25 kg
Max. load on massage unit	30 kg
Speed	5 ÷ 350 rpm
Massage impulse rate	30 ÷ 2100 /min.
Noise development	< 70 dB
Degree of protection	IP 20
Operating temperature	+ 5 °C ÷ 40 °C
Electrical protection class	Class I
Manufacturer	kybun AG Mühleweg 4 CH-9325 Roggwil

## Warranty

If this product fails due to a defect in material or workmanship within two years from the date of purchase, the manufacturer shall repair or replace the product or the necessary components free of charge.

### The warranty does not apply:

- to damage caused by an accident, misuse, improper handling or transport;
- to parts that have been subjected to unauthorised repair work;
- to parts that have not been used in accordance with the servicing instructions provided by kybun;
- to damage that exceeds the value of the product;
- to deterioration of the supplied product due to improper storage and/or protection conditions at the customer's premises;
- if proof of purchase cannot be provided.

Some countries do not allow damage limitation, meaning the aforesaid restrictions may not apply in your case. This warranty guarantees specific legal warranty claims, but a legal right may be enforced differently depending on the country you are in. This warranty service is available through the manufacturer's operating policies and procedures. Please contact the retailer from whom you purchased the product directly for warranty and return procedures. This warranty is initiated and carried out by the manufacturer.