



WWW.OSUNGUSA.COM

Instructions For Use Columbia Cheek Retractor, CRC

Purpose:

The purpose of the Columbia cheek retractor is to hold the cheeks or lips away from the teeth and gums during dental procedures.

1. Preparation

Before using this product, be familiar with the instructions and follow the instructions.

- 1) Significant deformation of the product's appearance, pinholes, attachment of foreign substances, etc. should be checked.
- 2) Follow CDC recommendation for sterilization: Minimum exposure periods for sterilization of wrapped healthcare supplies are 30 minutes at 121°C (250°F) in a gravity displacement sterilizer or 4 minutes at 132°C (270°F) in a pre-vacuum sterilizer.

3. Method of use

- 1) Follow the method of use per dental clinician procedures.
- 2) Columbia cheek retractor should be used with caution to avoid any discomfort or injury to the patient. It should be inserted gently and positioned correctly to achieve the desired retraction without causing excessive stretching or pain.

4. Storage and keeping conditions

- 1) After use, wash with a hand or washing machine using a cleaning agent and water to remove blood or foreign substances.
- 2) Store in a dry and cool place out of direct sunlight.
- 3) Check the appearance to check the wear condition or deterioration of the product.

5. Cautions in use or precaution in use

- 1) The sterilizer manufacturer's instructions should be followed in-order to avoid staining when the instrument is sterilized. Keep the cover closed until the steam is completely removed.
- 2) Do not use the product other than the intended purpose, store at room temperature.

PRODUCT WARRANTY:

Replacement warranty is limited due to the degradation of materials at high temperatures after repeated autoclaving.

11/2/23

11601 Shadow Creek Parkway, Suite 111-524, Pearland, Tx 77584

Tel: (312) 895-1773 Fax (847) 589-1298

www.osungUSA.com