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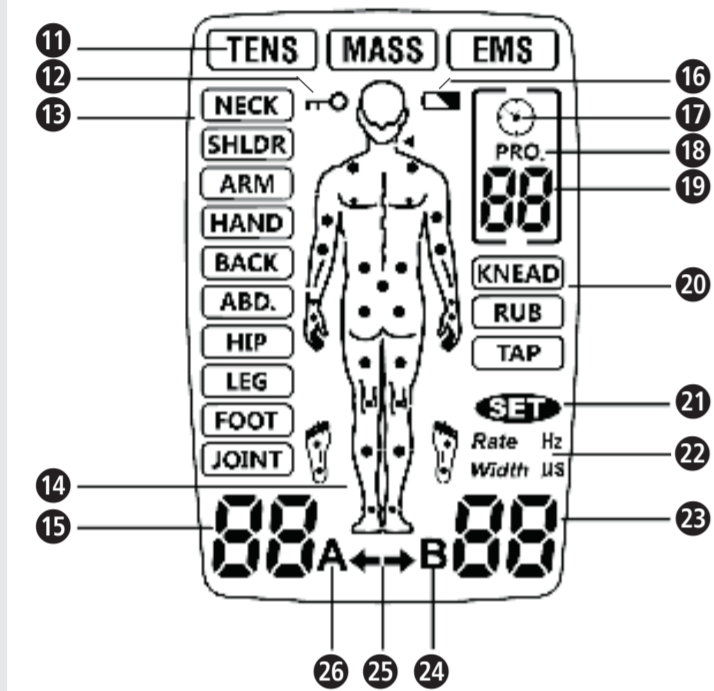
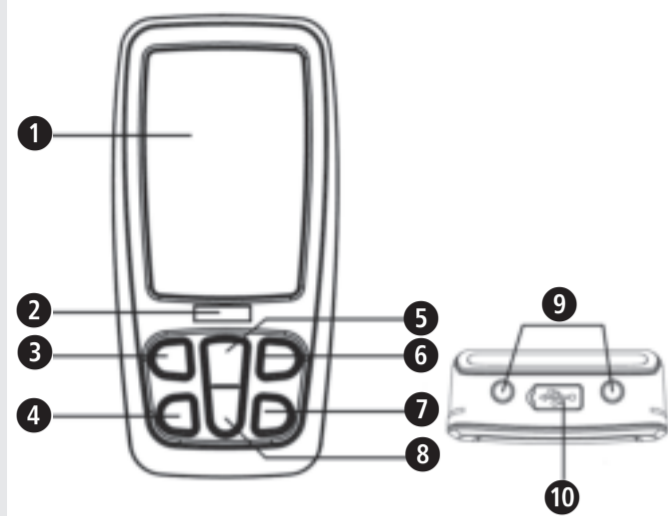
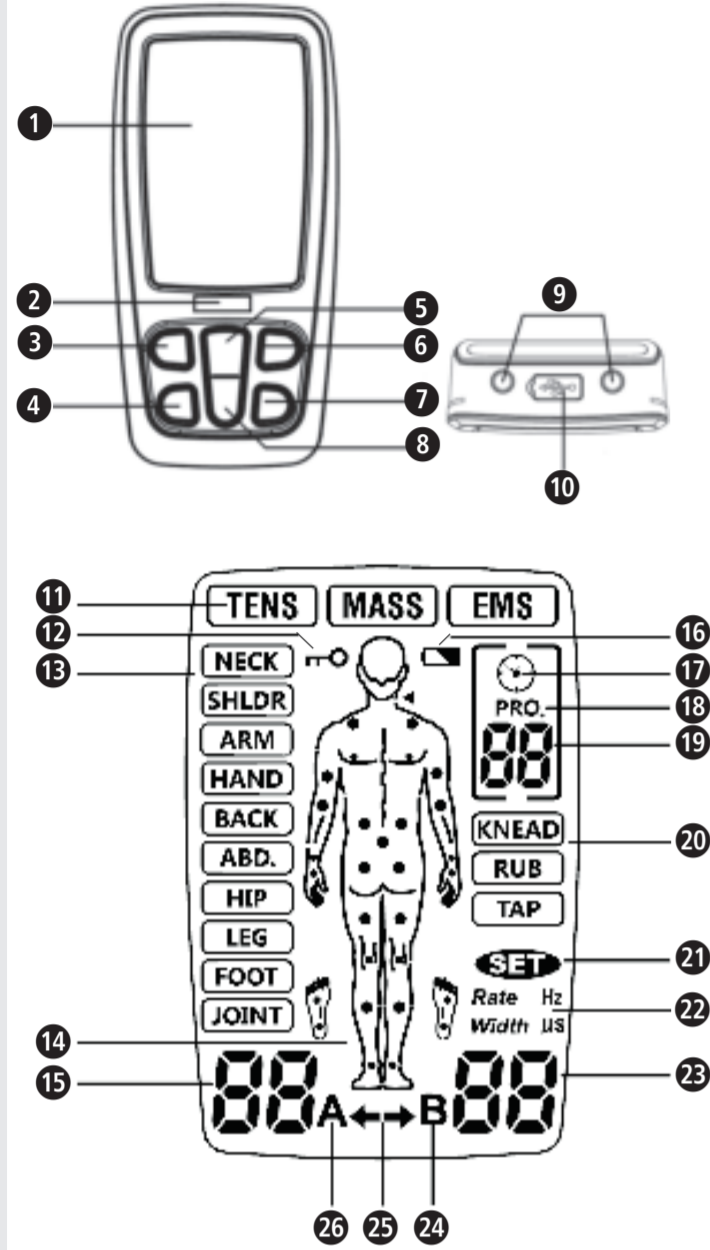
Shenzhen Roundwhale Technology Co., Ltd.
Add: 202, 2/F, Building 27, Dafa Industrial Park, Longxi community, longgang street, longgang district, Shenzhen, China.
Email: info@roovjoy.com

EC REP Shanghai International Holding Corp. GmbH (Europe)
Address: Eiffestr. 80, 20537 Hamburg, GERMANY

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Device and controls

Legend

IMPORTANT!
Failure to follow these instructions may result in serious injury or damage to the device.

WARNING
These warnings must be followed to prevent possible injury to the user.

ATTENTION
These instructions must be followed to prevent possible damage to the device.

NOTE
These instructions provide you with useful additional information regarding installation or operation.

Device classification: Type BF

LOT Batch number

IP22 Information about protection against foreign objects and water

EC REP Authorised EU representative

Manufacturer

Manufacture date

EN Safety instructions

Please read these instructions for use carefully before using the device, in particular the safety instructions. Keep the instructions in a safe place for future reference. If you give the device to someone else, be sure to include these instructions for use.

Intended use
The medisana 3-in-1 Electrotherapy device TT 205 is intended for the following areas of application:

- **TENS mode:** Temporary relief of muscle pain in the neck, shoulder, back, joints, hip, hand, abdomen, foot, upper (arm) and lower (leg) extremities due to exercise or normal house-work.
- **EMS mode:** The EMS stimulation program stimulates healthy muscles to improve and promote muscle performance.
- **Massage mode:** The massage program ensures muscle relaxation through vibration to loosen tense muscles.

Contraindications

- Do not use the device if you have a pacemaker, defibrillator, or other electronic or metallic implants in your body. These could result in electric shock, burns, electrical interference or death.
- Do not use the device if there are cancerous or other lesions in the treatment area.
- Do not treat any part of the body that has swelling, burns, inflammation, rashes (e.g. phlebitis, thrombophlebitis, varicose veins, etc.), wounds or sensitive areas.
- The electrodes must not be attached so that current can flow through the carotid sinus region (front throat) or transcranial (through the head).
- This device should not be used on excessively stressed skin areas.
- The device must not be used if there is an inguinal hernia.
- Do not treat scarred parts of the body after surgery - wait at least 10 months after surgery before using the device.
- Do not use the device in the case of severe circulation problems in the lower limbs.

- Therapies with the medisana TT 205 do not replace medical diagnoses or treatments. If you suffer from any pain or illness, consult your doctor before using the device.
- If your pain does not go away or does not become any milder or if it lasts longer than 5 days, stop using the device and contact your doctor.
- Do not use the device on the front and side of the neck as this may result in severe muscle spasms, airway obstruction, difficulty breathing, or adverse effects on the heart rhythm or blood pressure.
- Do not use the device on the chest as the application of electricity to the chest can lead to cardiac arrhythmia, which can be fatal.
- Do not use the device over or near cancerous lesions.
- Electronic monitoring devices (such as ECG monitors and ECG alarms) may not work properly when electrical stimulation is in use.
- Do not use the device while bathing or taking a shower.
- Do not use the device while sleeping.
- The device must not be used while operating a machine, while driving or during any activity where electrical stimulation poses a risk of injury.
- The stimulation may only take place on normal, undamaged, clean and healthy skin areas.
- The long-term effects of electrical stimulation are unknown. Electrical stimulation cannot replace medication.
- The device must not be used if the user is connected to high-frequency surgical devices. It can lead to burning of the skin under the electrodes or cause problems with the device.
- Do not use the device near short-wave or microwave therapy devices, as this can influence the output power of the device.
- Never use the device near the heart. The electrodes should never be placed on the chest. There is a risk of ventricular fibrillation or cardiac arrest.
- The device must also not be used near the eyes, on the head or face.
- Do not use near the genitals!
- Do not apply to skin areas where the sensation is disturbed.
- The electrodes must not come into contact with each other during use. This can lead to malfunctions and skin burns.
- Keep the device out of the reach of children.
- The treatment should feel pleasant. If you feel pain or find the application unpleasant, do not increase the intensity but rather stop the application and consult your doctor.
- The TENS mode is not effective for pain of central origin, including headache. It does not replace medical therapy or prescribed pain relievers.
- TENS is a symptomatic treatment and suppresses the sensation of pain that would otherwise serve as a protective mechanism.
- The effectiveness depends to a large extent on the selection of a doctor qualified in the treatment of pain patients.
- The effects of stimulating the brain are unknown, which is why it should not be used on the head. The electrodes should never be placed on opposite sides of the head.
- The safety of electrical stimulation during pregnancy has not been verified.
- Skin irritation or hypersensitivity reactions may occur due to electrical stimulation with an electrically conductive medium.
- Be absolutely sure to consult your doctor before using the device if you suffer from heart disease or epilepsy.
- Use the device with caution or only after consulting your doctor if you tend to bleed internally, e.g. after a fracture or if you have a menstrual period.
- If you have recently had surgery, consult your doctor before using the device as the stimulation may interfere with the healing process.
- This device may only be used by one patient.
- The device must not be used in the case of dementia or other mental disorders.
- Use the device only as described in these operating instructions. Any other use is considered improper and can lead to property damage or even personal injury. medisana assumes no liability for damage caused by improper use. The device is intended for private use.
- In rare cases, skin irritation can occur with long-term use in the area of the electrodes.
- Do not use the device together with other devices that transmit electrical impulses to your body.
- Do not use sharp or pointed objects such as a ballpoint pen to operate the buttons of the device.
- Check the electrode connections before each use.
- Use only electrodes recommended by the manufacturer for this device.

Side effects

- Skin irritation or burns can occur in the area of the electrode application.
- In very rare cases, first-time users of an EMS device have reported drowsiness or fainting. We recommend that you use the device while sitting until you get used to its use.
- If you feel uncomfortable, reduce the stimulation intensity to a comfortable level. Contact your doctor if the problems persist.

Scope of delivery

First, check whether the device is complete and is not damaged. If in doubt, do not operate the device and contact your supplier or service centre.

The contents include:

- 1 medisana 3-in-1-Electrotherapy device TT 205
- 4 Electrodes 50x50 mm
- 2 Connection cable
- 1 USB cable
- 1 Instructions for use

If you notice any transport damage while unpacking, please contact your supplier immediately.

WARNING
Keep packaging away from children!
Choking hazard!

Device and controls

- 1 Display field
- 2 Charge indicator
- 3 On/Off/M button
- 4 P button
- 5 + key
- 6 B button
- 7 CH button
- 8 button
- 9 Output port
- 10 USB port
- 11 Therapy mode
- 12 Device locked
- 13 Body region
- 14 Recommended treatment points
- 15 Intensity Channel A
- 16 Battery low
- 17 Timer symbol
- 18 Program symbol
- 19 Program no. / Therapy time
- 20 Massage mode
- 21 SET symbol
- 22 Frequency/Pulse width
- 23 Intensity Channel B
- 24 Symbol for Channel B
- 25 Channel indicator
- 26 Symbol for Channel A

Charge battery

Disconnect the connection cables from the output connections 9 (if necessary).

The device cannot be used while charging.

Connect the USB cable to the USB port 10 on the device and a suitable 5V USB charging adapter.

The charging indicator 2 lights up yellow while the device is being charged.

When charging is complete, the 2 charging indicator will light up green. Charging can take up to 2 hours.

Cable connection

Make sure that the device is switched off before you connect the cable. First connect the electrodes with the two connection cables and then the cables with the device. On the upper side of the device there are two output connections 9 for connecting the electrode lines. The device has two connections for two channels (A and B). Depending on your needs, you can decide whether you want to use only one channel with one pair of electrodes or connect both channels with two pairs of electrodes. With two pairs of electrodes you can stimulate two different body regions at the same time.

WARNING
NEVER insert the electrode cables into an electrical outlet or any other power source!

Applying the electrodes

The reusable carrier film can be removed immediately before use - keep it for later use. Then place the electrodes according to your pain area (follow the instructions of your doctor).

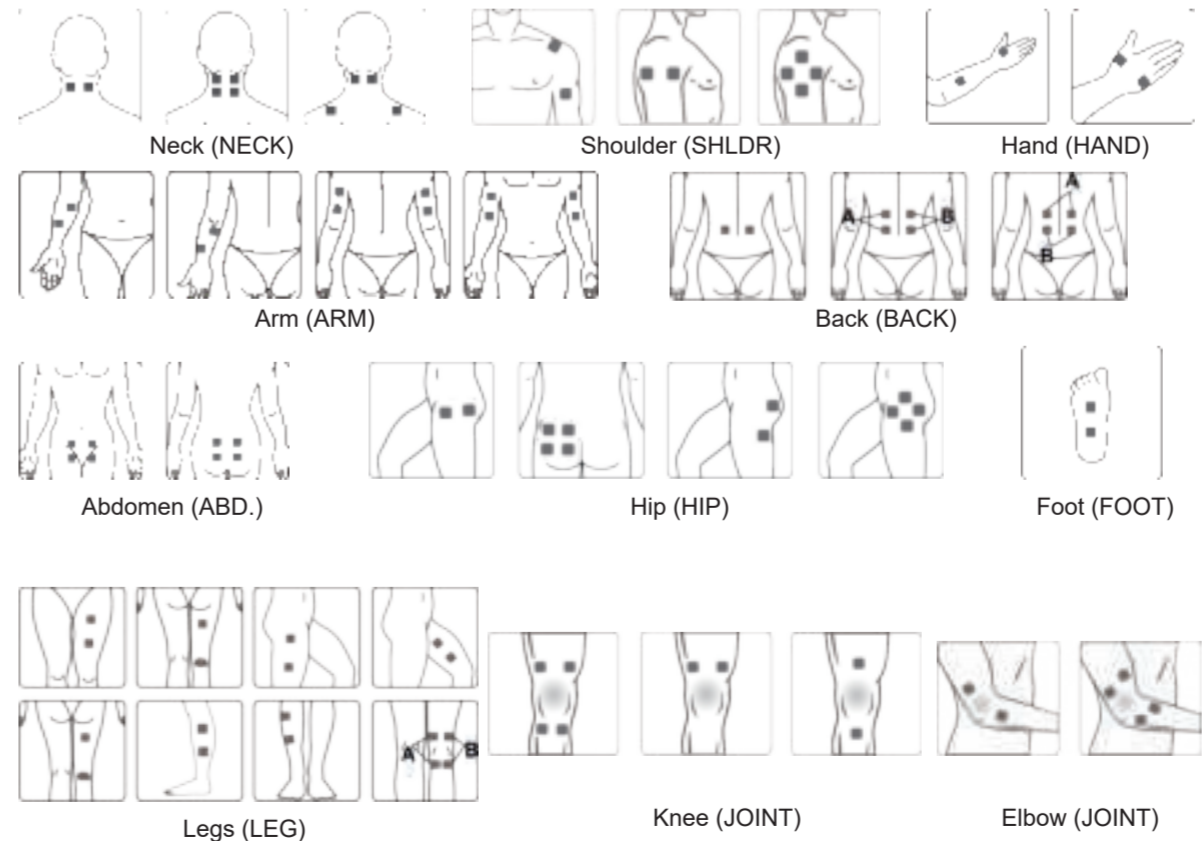
WARNING

- The device must be switched off when applying the electrodes!
- Remove any skin cream or ointment residue before applying the electrodes. The skin must be fat-free and dry.
- Do not stick the electrodes on injured or inflamed skin areas (wounds, pimples, rashes, redness, etc.).
- Always apply the electrodes according to your doctor's instructions and only then switch on the device.
- Make sure that the electrodes cannot fall off during the therapy. This can lead to an unintended electric shock!
- Switch off the device immediately if an electrode detaches from the skin during treatment.


In the following you will find placement suggestions for the electrodes depending on the therapy mode. However, please note that each patient reacts differently to stimulation current therapies - therefore the positioning of the electrodes and the program selection for your requirements may differ from the therapies presented here. You should make a note of the optimum parameters for you (electrode position, program and current) for subsequent treatments.

The recommended treatment points 14 for each program are shown in the illustration. The following illustrations show an example of how to attach the electrodes to these treatment points.

Examples of electrode positioning in the EMS therapy mode:

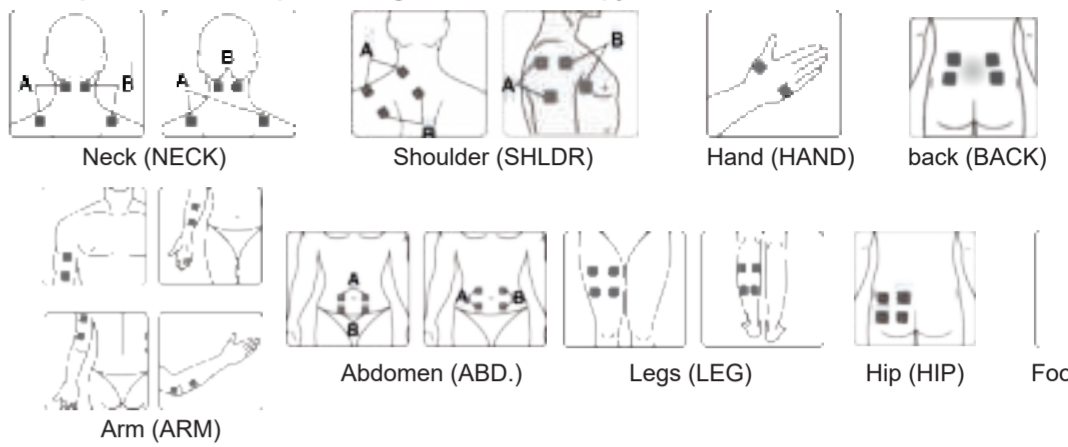Examples of electrode positioning in the EMS therapy mode:


Switching on

To switch on, briefly press the On/Off/M button 3. The indicator lights up and the device is in standby mode. Now select the therapy mode using the On/Off/ M button 3:


TENS : for pain relief (TENS = Transcutaneous Electrical Nerve Stimulation)

MASS : Massage to relax tense muscles

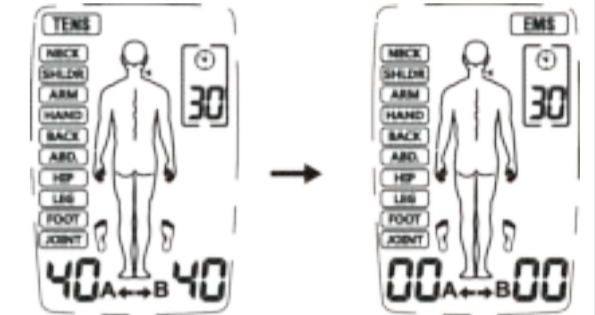EMS : For muscle building (EMS = Electrical Muscle Stimulation)

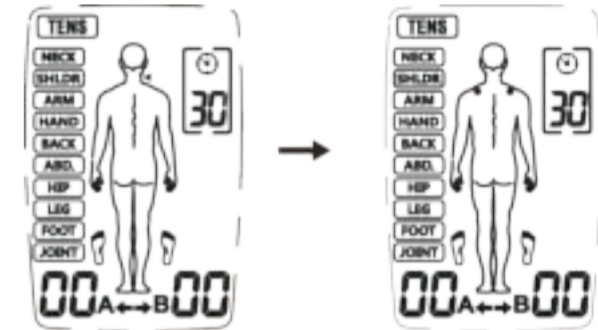Then use the B button 6 to select the area of the body to be stimulated:


Select program: Press the P button 4 repeatedly to choose between the three programs 01, 02 and U1 (user-defined).


Setting the program parameters
Press and hold the P button to make settings for the selected program:

In programs 01 and 02 you can make the following settings with the [+]/[-] button: Treatment time.

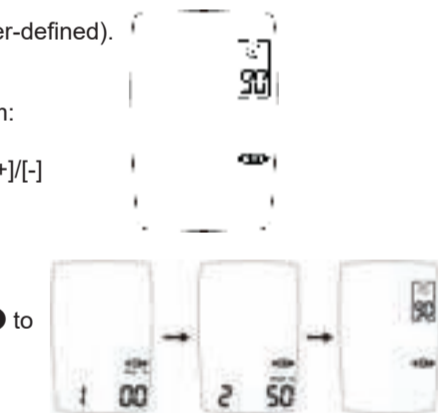
In program U1, press the P button 4 to set the frequency, pulse width and treatment time one after the other. To do this, press the [+]/[-] buttons. Then press the On/Off/M button 3 to return to the main screen. Your settings will be saved.

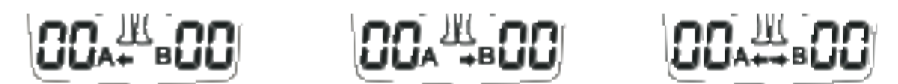Now use the CH button 7 to select the channel (Channel A, Channel B or both channels):

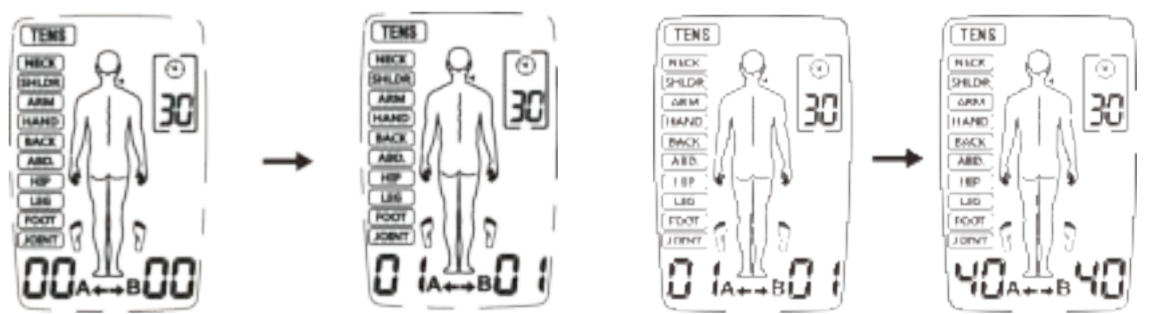
Now press the + button 5 to increase the intensity of the selected channel. The corresponding intensity level is shown on the display. Each time you press the + button 5, you can increase the intensity further up to the maximum level 40:



With the - button 5 the intensity is reduced accordingly. If you reduce the intensity to 0, the device switches to standby mode.

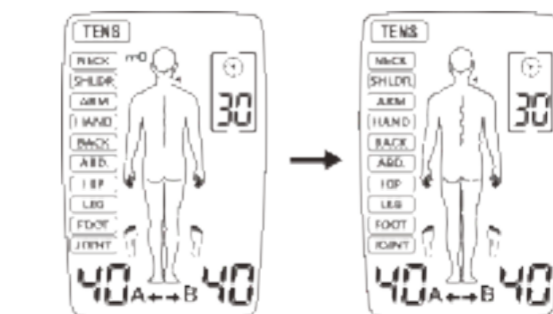During use, you can lock the device by pressing and holding the B button 6. The symbol 12 then appears on the display. You cannot accidentally adjust the intensity while locking. To release the lock, press and hold the B button 6 until the symbol 12 disappears.

EN Instructions for use 3-in-1-Electrotherapy device TT 205

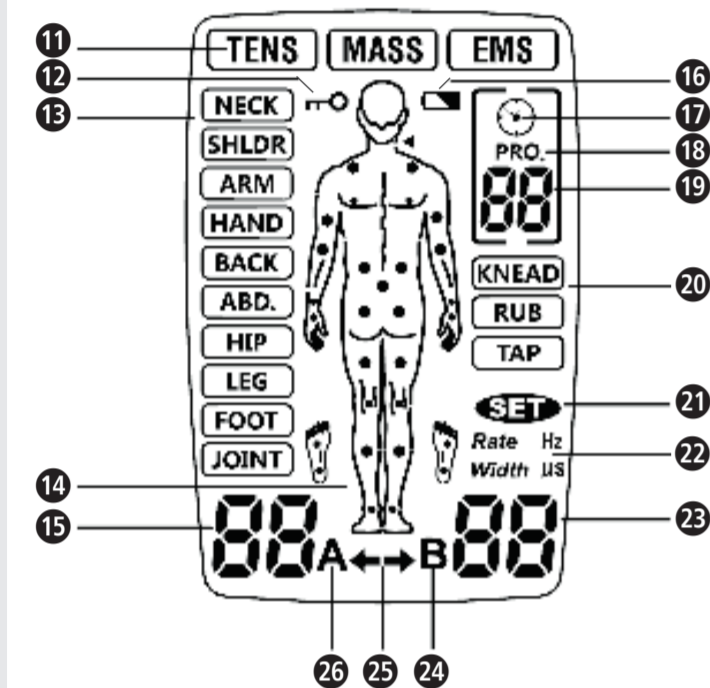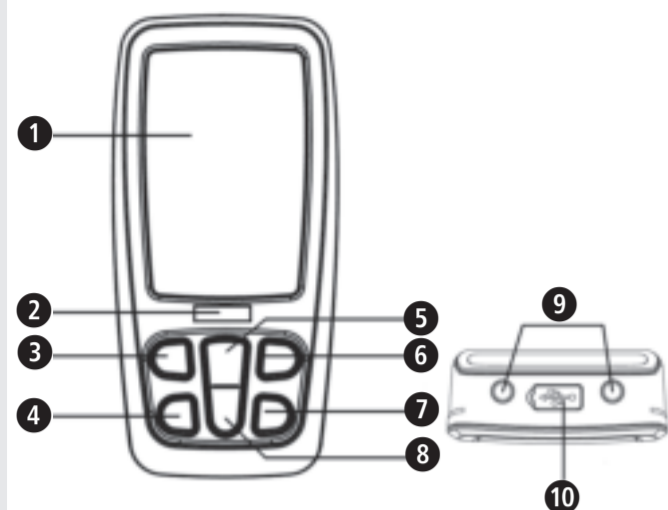Device and controls

Legend

IMPORTANT!

Failure to follow these instructions may result in serious injury or damage to the device.

WARNING

These warnings must be followed to prevent possible injury to the user.

ATTENTION

These instructions must be followed to prevent possible damage to the device.

NOTE

These instructions provide you with useful additional information regarding installation or operation.

Device classification: Type BF

Batch number

IP22

Information about protection against foreign objects and water

 Authorised EU representative

Manufacturer

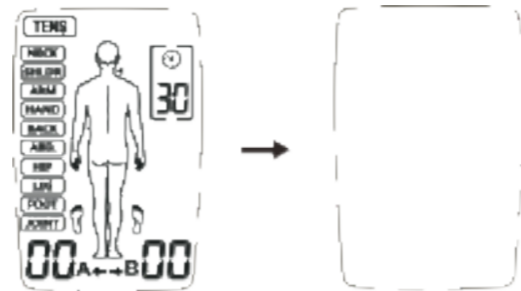
Manufacture date

2460

Press the On/Off/M button  to exit the application. Pressing the On/Off/M button  again for a long time switches the device off.

The device has an automatic load detection at all set intensities above level 5. If the electrodes do not stick to the skin properly or there is another connection problem, the device automatically switches to level 0 and the "A" or "B" symbol flashes. The device then switches back to standby mode.

If the battery voltage is too low, the "Battery low" symbol  flashes. Stop the application and replace the batteries.

Use of the electrodes

- The electrodes supplied may only be used with the medisana 3-in-1 Electrotherapy device TT 205. Make sure that the device is switched off before applying and removing the electrodes.
- If you want to reposition the electrodes during use, switch off the device first.
- Use of the electrodes can cause skin irritation. If you should notice this (e.g. red skin, blistering or itching) stop the use. Do not use the electrodes regularly on the same skin areas.
- Never use the electrodes together with other people.
- The electrodes must be in full contact with the skin to avoid possible "hot spots", which can cause burns.
- Do not use the same electrodes more than 10 times because the connection to the skin surface deteriorates with each use.
- The adhesive force of the electrodes depends on the skin properties, the storage condition and the number of applications. If your electrode pads no longer stick completely to the surface of the skin, replace them with new ones. After use, stick the electrode pads back onto the protective film and keep the electrodes in the storage bag so that they do not dry out. As a result, the adhesive force is retained over a longer period of time.
- Clean and dry the skin areas where the electrodes are to be applied.
- Never remove the electrodes from the skin while the device is still in operation.
- Only use original electrodes that are recommended by the manufacturer. Other electrodes could cause injuries.
- Each person reacts differently to electrical nerve stimulation. There may be individual deviations in the correct placement of the electrodes. Contact your doctor to find out which placement is best for you.
- Do not use electrodes smaller than the ones provided. Such use could make the current density too high and cause injury.
- Do not use electrodes supplied, e.g. by cutting.
- Make sure that the area from which the pain originates is completely covered with the electrode pads. In the case of painful muscle groups, all affected muscles must be reached by the electrodes.

NOTES on the TENS application

- If you find the output intensity too strong, you can reduce it with the "-" button .
- As long as you feel comfortable during the application, continue it to the end. Pain improvement usually occurs after about 5 to 10 minutes;
- We recommend 1 to 2 applications per day for about 1 week;
- If there is no significant improvement in pain after this time, contact your doctor.

NOTES on the EMS application

- Pay attention to the correct placement of the electrodes as described in these instructions;
- We recommend 1 to 2 applications per day for about 1 week;
- If you feel uncomfortable during the application, pause the application and/or reduce the intensity.

Maintenance and cleaning

In order to ensure trouble-free and long-term functional reliability of the device, the following points should be observed:

- Remove the electrodes from the device and clean the device with a soft, slightly damp cloth. In the case of heavy soiling, you can also use a mild detergent.
- Protect the device from moisture. Do not hold the device under running water and do not immerse it in water or other liquids.
- Do not place the device on hot surfaces and do not expose it to direct sunlight.
- Clean the surface of the electrode pads with a slightly damp cloth. Make sure that the device has been switched off beforehand!
- For hygienic reasons, each user should use their own electrode pads.
- Do not use aggressive substances or chemical cleaners to clean the device.
- Make sure that no liquid gets into the device. Do not use the device again until it is completely dry.
- Never clean the device during operation. Before cleaning, make sure that the device is switched off and the batteries have been removed!

Maintenance

- The manufacturer cannot be held responsible for damage caused by unauthorised repairs. If your device has a technical problem, contact customer service.
- Do not try to carry out repairs yourself.
- Opening the device will render the warranty void.
- Your device was tested before being sold and does not require calibration or regular maintenance. If you have any questions, please contact customer service.

Troubleshooting

Problem	Possible causes	Countermeasures
No display after changing the battery	Batteries not inserted correctly or foreign objects in the battery compartment. Battery type may be wrong.	1. Check and clean the battery compartment. 2. Insert new batteries of the correct type. Pay attention to the correct polarity!
No or too weak stimulation	- Electrodes not placed correctly on the skin - Connection to the device is not ideal - Used batteries - Too dry skin	1. Check that the electrodes are correctly attached to the skin and that the cable connection to the device is optimum. 2. Replace the battery. 3. Wipe your skin and the electrodes with a damp cotton cloth.
The skin tickles or burns during use	- The application is too long. - Electrodes are not placed correctly on the skin or the contact is not optimum - The skin is hypersensitive	1. Use the device once a day and shorten the time of use. 2. Check that the electrodes are correctly attached to the skin. 3. Wipe your skin and the electrodes with a damp cotton cloth. 4. Consult your doctor if you have hypersensitive skin.
The device stops during the application	- Electrodes are not placed correctly on the skin or the contact is not optimum - Used batteries	1. Check that the electrodes are correctly attached to the skin. 2. Replace the battery.

Program tables

Mode	Body part	Program	Frequency Hz	Impulse width µs	Duration min.	Waveform		
TENS	Neck	01	80-120	120-100	Standard: 30 Adjustable: 5-90	Modulation		
		02	4	150-200		Modulation		
		U1	Standard: 35 Adjustable: 2-100	Standard: 200 Adjustable: 100-300		Consistent		
		Shoulder	01	80-100		100	Modulation	
			02	2-60		260-160	Modulation	
			U1	Standard: 100 Adjustable: 2-100		Standard: 150 Adjustable: 100-300	Short Pulse groups	
	Arm	01	2	250	Consistent			
		02	100	150	Short Pulse groups			
		U1	Standard: 100 Adjustable: 2-100	Standard: 200 Adjustable: 100-300	Consistent			
	Hand	01	100	100	Consistent			
		02	2-10	200	Modulation			
		U1	Standard: 60 Adjustable: 2-100	Standard: 260 Adjustable: 100-300	Modulation			
	Back	01	60/50/45/10/50/35	200	Modulation			
		02	6/8/10	250	Modulation			
		U1	Standard: 55 Adjustable: 2-100	Standard: 200 Adjustable: 100-300	Modulation			
	Abdomen	01	80-120	120-100	Modulation			
		02	120	55	Consistent			
		U1	Standard: 80 Adjustable: 2-100	Standard: 100 Adjustable: 100-300	Consistent			
	Hips	01	100	150	Short Pulse groups			
		02	40/6/50	200	Modulation			
		U1	Standard: 80 Adjustable: 2-100	Standard: 180 Adjustable: 100-300	Consistent			
	Leg	01	40/6/50	250	Modulation			
		02	80	150	Modulation			
		U1	Standard: 6-10 Adjustable: 2-100	Standard: 200 Adjustable: 100-300	Modulation			
	Foot	01	80-120	100-120	Modulation			
		02	2-10	200	Modulation			
		U1	Standard: 2-60 Adjustable: 2-100	Standard: 260-160 Adjustable: 100-300	Modulation			
	Joints	01	100	150	Short Pulse groups			
		02	120	100-120	Modulation			
		U1	Standard: 80 Adjustable: 2-100	Standard: 180 Adjustable: 100-300	Consistent			
EMS	Neck	01	30	200		Synchronous		
		02	40	200		Synchronous		
		U1	Standard: 50 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous		
	Shoulder	01	45	200		Synchronous		
		02	55	200		Synchronous		
		U1	Standard: 80 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous		
	Arm	01	50	150		Synchronous		
		02	60	150		Synchronous		
		U1	Standard: 80 Adjustable: 20-100	Standard: 150 Adjustable: 100-300		Synchronous		
	Hand	01	4	200		Synchronous		
		02	5	300		Synchronous		
		U1	Standard: 20 Adjustable: 20-100	Standard: 150 Adjustable: 100-300		Synchronous		
	Back	01	60	200		Synchronous		
		02	70	200		Synchronous		
		U1	Standard: 80 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous		
	Abdomen	01	20	200		Synchronous		
		02	50	200		Synchronous		
		U1	Standard: 60 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous		
	Hips	01	30	150		Synchronous		
		02	60	150		Synchronous		
		U1	Standard: 40 Adjustable: 20-100	Standard: 150 Adjustable: 100-300		Synchronous		
	Leg	01	20	200		Synchronous		
		02	80	200		Synchronous		
		U1	Standard: 25 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous		
	Foot	01	4	200		Synchronous		
		02	5	300		Synchronous		
		U1	Standard: 20 Adjustable: 20-100	Standard: 200 Adjustable: 100-300		Synchronous		
	Massage	Knead	01	28-44	120-250	30	Modulation	
			01	25-79	120-250		30	Modulation
			01	49-97	100-240		30	Modulation

Technical information

Name and model: **medisana 3-in-1-Electrotherapy device TT 205**
 Model no: R-C4A
 Power supply: Li-Ion battery, 3.7 V
 Channels: 2
 Waveform: Biphasic rectangular pulse
 Output current: Max. 120mA (at 500 Ohm load)
 Intensity: adjustable in 40 steps
 Output precision: ± 20% for all parameters
 Application modes: TENS, EMS and MASSAGE
 Number of programs: TENS: 30; EMS: 27; MASSAGE: 3
 Impulse width: 55 - 260 µs, depending on the program
 Frequency: 2 - 120 Hz, depending on the program
 Application time: 5 - 90 minutes, adjustable depending on the program
 Automatic switch off: after 1 minute
 Operating conditions: +5°C to +40°C at 15% to 93% rel. humidity;
 atmospheric pressure 700 hPa to 1060 hPa
 -10°C to +55°C at 10% to 95% rel. humidity;
 atmospheric pressure 700 hPa to 1060 hPa

Storage conditions:

Dimensions approx.:
 Size of electrode pads approx.: 50 x 50 mm
 Weight: approx.: 82 g
 Item number: 88348
 EAN number: 4015588 88348 4

In the interest of constant product improvement, we reserve the right to make technical and design changes.

The current version of these user instructions can be found at www.medisana.com

Accessories

Cable set - Item number 88359
 8 self-adhesive electrodes - Item number 88349

Storage

Switch off the device and remove the cables. Place the electrodes on the carrier film and pack them together with the device and the cables in the original packaging. Observe the storage conditions in the technical data!

Disposal

This device must not be disposed of together with household waste. Every consumer is legally obligated to bring all used batteries, rechargeable or otherwise, whether they contain harmful substances or not, to a collection site in their town or city so that they can be transferred to the environmentally friendly channels for disposal.
 Contact your local authority or dealer regarding disposal.

Warranty and repair conditions

In case of warranty please contact your specialist shop or the service centre directly. Should you have to return the device, please state the defect and enclose a copy of the purchase receipt or invoice. The following warranty conditions apply:

- medisana products are guaranteed for 3 years from the date of purchase. In case of a warranty claim, the purchase receipt or invoice must be provided as proof of the purchase date.
- Defects due to material or manufacturing defects will be remedied free of charge within the warranty period.
- A warranty service does not extend the warranty period for either the device or for replaced components.
- The following are excluded from the warranty:
 - All damage caused by improper handling, e.g. by not following the instructions for use.
 - Damage due to repair or intervention by the purchaser or unauthorised third parties.
 - Transport damage that has occurred on the way from the manufacturer to the consumer or when sending it to the service centre.
 - Spare parts that are subject to normal wear and tear.
- Liability for direct or indirect consequential damage caused by the device is also excluded when the damage on the device is recognised as a warranty claim.

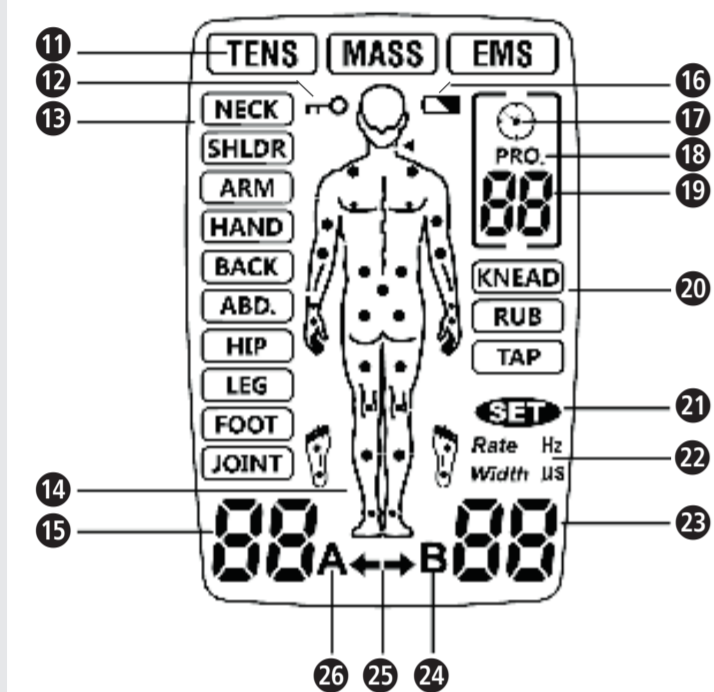
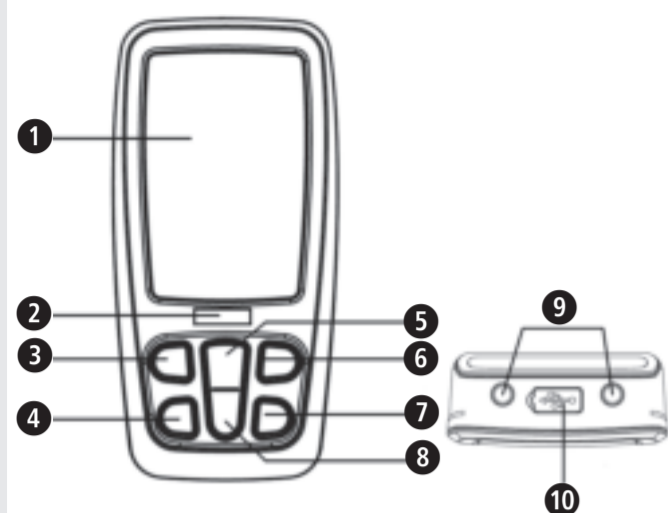
 Shenzhen Roundwhale Technology Co., Ltd.
 Add: 202, 2/F, Building 27, Dafa Industrial Park, longxi community, longgang street, longgang district, Shenzhen, China.
 Email: info@roovjoy.com

 Shanghai International Holding Corp. GmbH (Europe)
 Address: Eiffestr. 80, 20537 Hamburg, GERMANY

Imported & distributed by:
medisana GmbH, Jagenbergstr. 19, 41468 NEUSS, GERMANY

The service address can be found on the separate attachment sheet.

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Shanghai International Holding Corp. GmbH (Európa)
medisana GmbH, Jagenbergstr. 19, 41468 NEUSS, NEMECKO