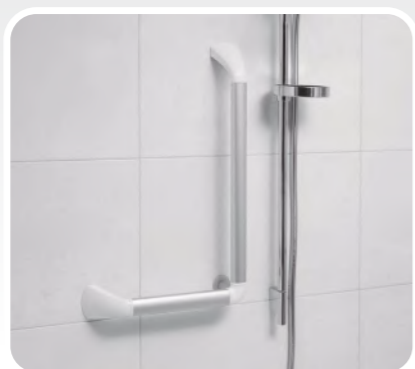
# WANDBEUGEL 45 & 90 GRADEN

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Wall Grab Rail,  
 45 & 90 degrees

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## GB

### Required tools :

- Electric drill/screwdriver
- Masonry drill, 8 and 10 mm
- Allen key, 6 mm
- Cap no. 10 (for driving electric drill or ratchet)
- Pozidrive screwdriver
- Optional hacksaw / lubricating grease

### Parts list, 45 degrees:

- 2 grip sections, aluminium 231 mm
- 2 mounting legs, grey
- 2 covers, white
- 2 sealing rings
- 1 coupling piece, grey 45 degrees
- 1 cover, white
- 1 aluminium foot, round
- 6 SS chipboard screws, 5x40
- 6 plugs, diam. 8 mm (for masonry)
- 1 SS Allen bolt, M8 x 60 mm
- 1 serrated lock washer, diam. 8 mm
- 1 Fischer MS Plug M8 (for masonry)
- 3 Tapitte SS hexagonal (no. 10)
- 2 Tapitte SS Pozidrive

### Parts list, 90 degrees left & right:

- 1 grip section, aluminium 231 mm
- 1 grip section, aluminium 431 mm
- 2 mounting legs, grey
- 2 covers, white
- 2 sealing rings
- 1 coupling piece, grey 90 degrees
- 1 cover, white
- 1 aluminium foot, round
- 6 SS chipboard screws, 5x40
- 6 plugs, diam. 8 mm (for masonry)
- 1 SS Allen bolt, M8 x 60 mm
- 1 serrated lock washer, diam. 8 mm
- 1 Fischer MS Plug M8 (for masonry)
- 3 Tapitte SS hexagonal (no. 10)
- 2 Tapitte SS Pozidrive

**Note: the wall grab rail will not attain its full strength until mounted on the wall; therefore caution is advised during assembly.**

1. Check the contents of the packaging to see if everything is complete.
2. Place the white sealing rings over the mounting legs and attach them to the aluminium grip sections using the supplied Tapitte screws (hexagonal version). Tighten them firmly! (fig. 1)
3. Place the preassembled grip sections in the 45 or 90 degree coupling piece. (fig. 3)
4. Now check the planned mounting location to see if any of the aluminium grip sections should be shortened. Attention: for the 90 degree model, check if you need a left-hand (fig. 4) or right-hand (fig. 5) version.
5. If the grip sections do not have to be shortened, continue to point 7.
6. Shortening the grip sections: Determine the desired length and shorten with a good hacksaw (fig. 6).

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## Disclaimer:

### Cleaning advice

Clean daily with a soft towel and, if necessary, with a non-aggressive, non-scouring detergent (e.g. green soap). Do not clean using bleach and/or descaling agents. After cleaning, rinse well and dry. Ensure no traces of detergent remain.

### Guarantee

Please keep your proof of purchase with the date of purchase and product description or name stated on it: this is your guarantee certificate. The five-year guarantee commences after the date of purchase by the end-user. The guarantee only applies in the event of production errors in the materials used in the SecuCare product. A successful claim under the guarantee will result to no entitlement other than delivery of a replacement product by SecuProducts BV free of charge. SecuProducts BV accepts no liability whatsoever for any injury or consequential damage that arises during the mounting and/or use of the product.

### The guarantee will lapse through:

- excessive pressure placed on the product or the incorrect, inappropriate or unnatural use of the product;
- damage caused deliberately or through neglect;
- application on an uneven surface;
- damage caused by lack of maintenance, incorrect maintenance or the use of unsuitable cleaning agents (see cleaning advice);
- damage caused during mounting;
- incorrect mounting (including mounting that is not in accordance with the mounting instructions or use of mounting material other than that supplied);
- influence of chemical/electro-chemical substances or electricity;
- damage caused by 'force majeure' or natural disasters (including flooding, fire and freezing).

### Important

You must follow this assembly manual carefully. And you must check regularly that the product is still tight. It is up to you to make sure that the product is firmly attached to any surface.

See important safety instructions on: [go.secu.nl/veiligheid](http://go.secu.nl/veiligheid)

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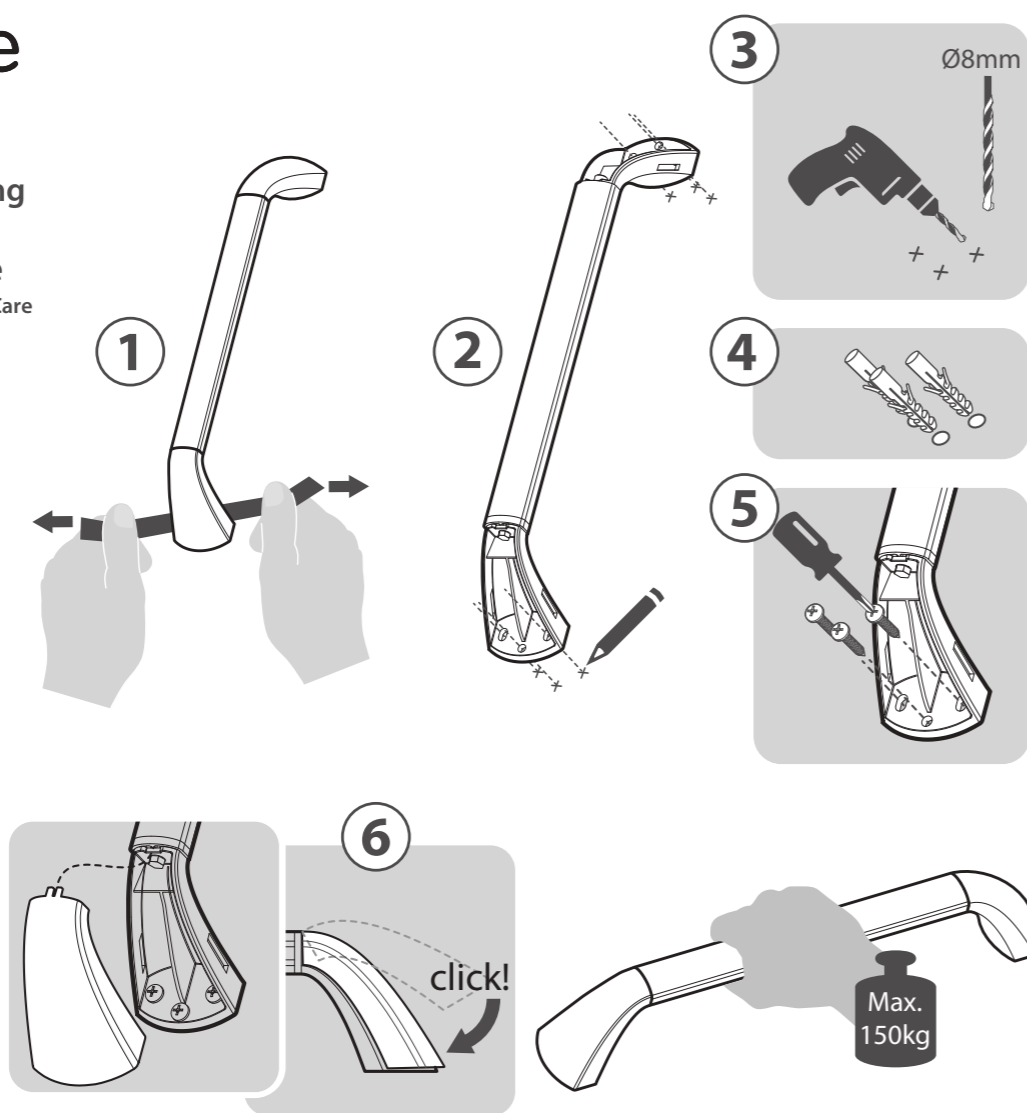
Assembly manual  
for SecuCare Wall Grab Rail

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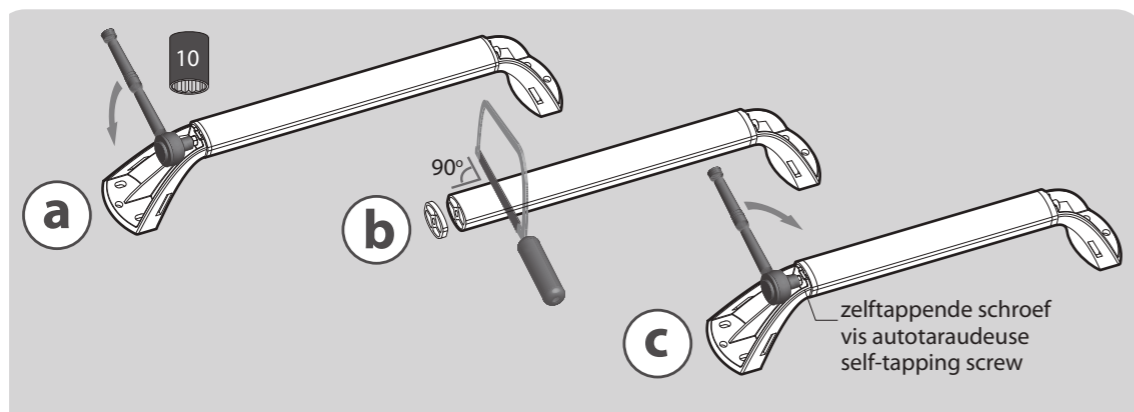
### Standard mounting hardware included:

6 Stainless steel chipboard screws 5x40mm and  
6 plugs ø8 (for stone)

### Surface Additional mounting hardware required:

Stone	Not Applicable
Wood	Not Applicable
Hollow wall	(plaster board) 6 Toggler plugs
Gypsum blocks	6 plugs tbv gypsum blocks
Aerated concrete	6 aerated concrete anchors

check the website for more information: [www.seccare.nl](http://www.seccare.nl)

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### Assembly instructions for application of adhesive stainless steel mounting plates for the SecuCare Aluminium wall bracket – length 40 cm. Art. Nr.: 8010.005.03 (Not suitable for longer lengths!)

#### Required for fitting:

- SecuCare Wall Bracket type 40 cm
- Adhesive stainless steel SecuCare mounting plates (supplied in the packaging)
- Bison PolyMax Express, not supplied (no other make or type)
- Open-end wrench no. 10
- Anti-lime scale
- All-purpose degreaser (e.g. washing-up liquid)
- Petroleum ether

**Important:** Also read the general disclaimer

#### General comment:

The wall bracket must be assembled carefully for maximum safety. The tile may be slightly rough (may also be smooth). Check that the wall tiles are properly attached to the wall surface.

1. Select the desired place to mount the bracket.
2. If necessary, shorten the SecuCare wall bracket (see assembly instructions for wall bracket).
3. First clean the surface with anti-lime scale, removing all lime scale residue.
4. Degrease using all-purpose degreaser (e.g. warm soapy water with washing-up liquid).
5. Dry and clean with petroleum ether.
6. Check that the mounting place is absolutely clean and free of grease.
7. Also degrease the bottom surface of the stainless steel mounting plates (with petroleum ether).
8. Remove the protective covers from the wall bracket.
9. Determine the correct position for the adhesive stainless steel mounting for the base of the wall bracket; on the inner surface of the wall bracket right at the base, on the other side protruding by approx. 2 mm – soon to be covered by the edge of the protective cover (see photo 1).
10. Place a small blob of PolyMax glue in the centre of adhesive stainless steel mounting (see photo 2).
11. Be sure to use enough PolyMax glue on the adhesive stainless steel mounting (see photo 3).
12. Place the adhesive stainless steel mounting on the bottom surface of the wall bracket bases and position them properly (see point 9).
13. Mount the wall bracket with the adhesive stainless steel bases in the desired position and press well.
14. Check the positions of the adhesive stainless steel bases in relation to the base of the wall bracket (step 9).
15. Now carefully remove the wall bracket from the adhesive stainless steel bases. NB: ensure that the adhesive bases remain in position.
16. Remove any remaining glue residue from the wall bracket base (using petroleum ether) (see photo 4).
17. Allow the adhesive stainless steel mounting to dry properly, removing any remaining PolyMax glue after around 4 hours.
18. After allowing to dry properly (2 x 24 hours), you can now mount the wall bracket onto the stainless steel adhesive plates using the nuts supplied (see photo 5). Attention, tighten the nuts hand-tight (max. 5 Nm).
19. Check before first-time use (and regularly thereafter) that the mounting is secure (see photo 6).
20. The maximum load is 90 kg.

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