## Montage handleiding

SecuCare Wandbeugel

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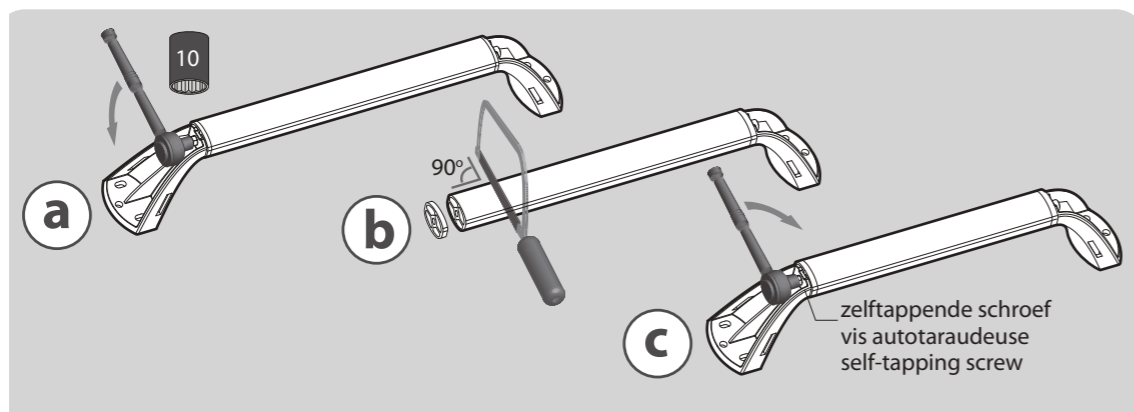
### Standard mounting hardware included:

6 Stainless steel chipboard screws 5x40mm and  
6 plugs ø8 (for stone)

### Surface Additional mounting hardware required:

Stone	Not Applicable
Wood	Not Applicable
Hollow wall	(plaster board) 6 Toggler plugs
Gypsum blocks	6 plugs tbv gypsum blocks
Aerated concrete	6 aerated concrete anchors

check the website for more information: [www.seccare.nl](http://www.seccare.nl)

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### Assembly instructions for application of adhesive stainless steel mounting plates for the SecuCare Aluminium wall bracket – length 40 cm. Art. Nr.: 8010.005.03 (Not suitable for longer lengths!)

#### Required for fitting:

- SecuCare Wall Bracket type 40 cm
- Adhesive stainless steel SecuCare mounting plates (supplied in the packaging)
- Bison PolyMax Express, not supplied (no other make or type)
- Open-end wrench no. 10
- Anti-lime scale
- All-purpose degreaser (e.g. washing-up liquid)
- Petroleum ether

**Important:** Also read the general disclaimer

#### General comment:

The wall bracket must be assembled carefully for maximum safety. The tile may be slightly rough (may also be smooth). Check that the wall tiles are properly attached to the wall surface.

1. Select the desired place to mount the bracket.
2. If necessary, shorten the SecuCare wall bracket (see assembly instructions for wall bracket).
3. First clean the surface with anti-lime scale, removing all lime scale residue.
4. Degrease using all-purpose degreaser (e.g. warm soapy water with washing-up liquid).
5. Dry and clean with petroleum ether.
6. Check that the mounting place is absolutely clean and free of grease.
7. Also degrease the bottom surface of the stainless steel mounting plates (with petroleum ether).
8. Remove the protective covers from the wall bracket.
9. Determine the correct position for the adhesive stainless steel mounting for the base of the wall bracket; on the inner surface of the wall bracket right at the base, on the other side protruding by approx. 2 mm – soon to be covered by the edge of the protective cover (see photo 1).
10. Place a small blob of PolyMax glue in the centre of adhesive stainless steel mounting (see photo 2).
11. Be sure to use enough PolyMax glue on the adhesive stainless steel mounting (see photo 3).
12. Place the adhesive stainless steel mounting on the bottom surface of the wall bracket bases and position them properly (see point 9).
13. Mount the wall bracket with the adhesive stainless steel bases in the desired position and press well.
14. Check the positions of the adhesive stainless steel bases in relation to the base of the wall bracket (step 9).
15. Now carefully remove the wall bracket from the adhesive stainless steel bases. NB: ensure that the adhesive bases remain in position.
16. Remove any remaining glue residue from the wall bracket base (using petroleum ether) (see photo 4).
17. Allow the adhesive stainless steel mounting to dry properly, removing any remaining PolyMax glue after around 4 hours.
18. After allowing to dry properly (2 x 24 hours), you can now mount the wall bracket onto the stainless steel adhesive plates using the nuts supplied (see photo 5). Attention, tighten the nuts hand-tight (max. 5 Nm).
19. Check before first-time use (and regularly thereafter) that the mounting is secure (see photo 6).
20. The maximum load is 90 kg.

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