

WARNING: Follow all instructions carefully and use only as directed. Always consult with your healthcare professional before treating any serious injury or if pain persists beyond treatment with the Kanjo Cushion.

WARNING: Risks are associated with any exercise program. Please consult with your healthcare professional before performing this or any physical activity.

Please visit
www.gokanjo.com/warranties for
information on our Limited Express
Product Warranty.

Questions? Concerns?

Please contact us at support@gokanjo.com www.gokanjo.com


Kanjo Memory Foam Acupressure CUSHION

Welcome to Kanjo, and thank you for your purchase. Kanjo reminds you to make time for you. Relieve pain. Destress. Relax and rejuvenate.

Kanjo Acupressure products have stimulators that target acupressure points to reduce pain throughout the body, much like ancient Eastern methodologies used in acupuncture, but non-invasive and easy to use at home and on the go.

You are on your way to experiencing the benefits of your **Kanjo Memory Foam Acupressure Cushion**, including neck, lower back, side and leg pain relief as well as headache and migraine relief and increased circulation.



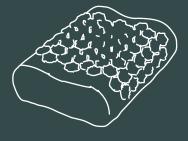

A few instructions:

- Place the Cushion as shown. The Cushion can be used in multiple positions.
- Adjust your position to maximize comfort.

A few notes:

The Kanjo Cushion's pain-relief effects are best facilitated by contacting the stimulators with your bare skin. For some, this may be overwhelming, and a shirt and long pants can be worn to reduce sensitivity to the stimulators.

If it's your first time using the Kanjo Cushion, we recommend a 15 minute session to start. First-time users may feel strange or even uncomfortable on the plastic stimulators. Any discomfort will subside as your body responds to the stimulators' effects and pain relief begins, usually after a few minutes of contact.

How do I clean this?:

To clean fabric cover, remove it from the memory foam core, spot clean with water or a mild detergent, and let dry completely before use.





