



**** IMPORTANT NOTE: MANUAL OPERATION OF AutoFold™ Powered Ramp In Case of Power Failure****

- ✚ In the event of power failure, ramp may be used in the manual operation mode by removing cotter pin from top of black motor. Then push out pin securing motor. (Motor Post View picture). Motor lays back into van and is disconnected from ramp. Simply bi-fold ramp up and down.
- ✚ Once problem is corrected simply re-insert pin into motor mount & re-insert clip for normal operation



**Contact the Roll-A-Ramp Distribution Center
for Technical Support 1.866.883.4722
We may request digital images**