



Lifeline Corporation Pte Ltd

745 Lorong 5 Toa Payoh #01-00 Singapore 319455

Tel: 62892062 • Fax: 62896096 • e-sales@lifeline.com.sg



User Manual

60/1110 Mobile Recliner Geriatric Chair with Tray



TABLE OF CONTENTS


Protecting your investment	2
Warnings	2
Special Notes	2
Specifications	2
Operating Precautions	3
General Warnings	3
Safety Inspection Checklist	4
Operation Instructions	5
Reclining the chair using the position lock level	5
Reclining the chair without using the position lock level	5
Positioning the articulating headrest	6
Locking and unlocking the rear wheels	6
Installing tray into position	6
Adjusting the tray position	7
Removing and tray storage	7
Cleaning the upholstery	7



Lifeline Corporation Pte Ltd

745 Lorong 5 Toa Payoh #01-00 Singapore 319455

Tel: 62892062 • Fax: 62896096 • e-sales@lifeline.com.sg

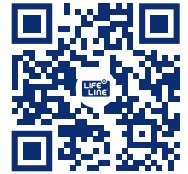

PROTECTING YOUR INVESTMENT

Please read these instructions to be sure all precautions are being taken to use this product safely and for the intended uses only.

WARNINGS!

Prior to operating this recliner you must review and understand this manual. If you are unable to understand the warnings, cautions and instructions, seek assistance before attempting to use this recliner – Injury or damage to property or this recliner may result.

SPECIAL NOTES

Warning / caution notices in this manual apply to hazards or unsafe practices, which could result in personal injury or property damage.

WARNINGS!

This recliner has not been designed and / or tested for seating surfaces or vehicle chairs for occupant use within any type of vehicle. At no time or for any reason is the recliner to be used in a transport vehicle.

Restraints and seat positioning straps – It is at the discretion of the recliner user and his / her healthcare professionals to determine the safest restraints to be used considering all limitations of the recliner user.

SAVE THESE INSTRUCTIONS

SPECIFICATIONS

Physical Dimensions

Total Width :	28.5" / 72 cm (With Tray)	Seat Depth :	19"
	26.5" / 67cm (Without Tray)	Seat :	35°
Total Length :	41" / 104 cm (Upright)	Back Support :	115° (Upright)
	68" / 173 cm (Reclined)		145° (Full Recline)
Total Height :	47" / 119 cm	Recliner Weight :	55 kg
Seat Width :	20" / 50 cm	Recliner Weight Limitation :	120 kg
Front Seat Height :	20" / 50 cm (Upright)		
Back Seat Height :	17" / 43 cm (Upright)		



Lifeline Corporation Pte Ltd

745 Lorong 5 Toa Payoh #01-00 Singapore 319455

Tel: 62892062 • Fax: 62896096 • e-sales@lifeline.com.sg



OPERATION PRECAUTIONS

GENERAL WARNING

DO NOT sit on the footrest when recliner is extended. The recliner WILL tip and bodily injury may occur.

DO NOT push or pull footrest to position the recliner.

DO NOT place hands or feet into any openings when adjusting the recliner. Attendant or care provider should always verify placement of user's hands and feet prior to adjusting the recliner. Failure to do so may result in serious bodily injury.

Operation of recliner with missing or broken footrest or mechanism guards may result in bodily injury. Periodic inspection of the footrest guard and scissor mechanism guards MUST be made 4 times a year. Torn, cracked or otherwise damaged guards MUST be replaced prior to use. Recliner MUST be removed from service until replacement guards are installed.

DO NOT lift the recliner by the footrest. Lifting by means of the footrest may result in injury to the user and/ or damage to the recliner.

DO NOT use near an open flame or heat source, as all upholstery materials, even though treated with flame retardant, can ignite when exposed to open flame.

Rear castor wheels MUST be locked during user transfer to or from recliner and while tilting and/or inclining (reverse recline).

DO NOT enter or exit the recliner when footrest is elevated.

DO NOT attempt to stop the recliner while in motion with the wheel locks.

WHEELS LOCKS ARE NOT BRAKES.

Before attempting to transfer in or out of the recliner, every precaution should be taken to reduce the gap distance. Turn the recliner toward the object you are transferring onto. When transferring to and from the recliner, ALWAYS ENGAGE BOTH WHEEL LOCKS. Special care MUST BE taken with people that have physical limitations, which may require an assistant.

DO NOT traverse, climb or go down ramps or slopes greater than 9 degrees.

DO NOT use parts, accessories, or adapters other than those authorized by Lifeline.

GENERAL WARNING

WARNINGS

The weight limitations for the recliner is 300 lbs (113.6 kg)

The tray has a weight limitation of 50 lbs (18.2 kg)

TILT AND/OR RECLINE OPTION OPERATED BY AN ATTENDANT

WARNINGS

ALWAYS make sure that the recliner is stable BEFORE using the tilt and/or recliner option.

Make sure the patient is properly positioned in the recliner before tilting, reclining or inclining (reverse recline) to maintain maximum stability and safety.

When returning the occupant of the recliner to the full upright position, more body strength will be required for approximately the last 20 degrees of the incline (reverse recline). Make sure to use proper body mechanics (use your legs) or seek assistance to avoid injury.



Lifeline Corporation Pte Ltd

745 Lorong 5 Toa Payoh #01-00 Singapore 319455

Tel: 62892062 • Fax: 62896096 • e-sales@lifeline.com.sg



Physical limitations of a user and larger occupants of the recliner can present conditions that can injure the assistant. When in doubt, assure that additional assistants are present before operating the tilt and/or recline feature.

SAFETY INSPECTION CHECKLIST

NOTE: To operate the recliner properly and safely, the mobile recliner must be cared for appropriately.

- Once a year or as necessary take the mobile recliner to a qualified technician for a thorough inspection and servicing.
- Routine maintenance will extend the life and efficiency of the recliner.
- Periodic inspection of the footrests and scissor mechanism guard **MUST** be made 4 times a year.
- Regular cleaning will reveal loose or worn parts and enhance the smooth operation of the mobile recliner.

ITEM CHECKLIST	Initially	Quarterly	Annually
GENERAL	•		
• Recliner rolls straight (no excessive drag or pull to one side).			
SEAT AND BACK	•	•	•
• Inspect for rips and sagging. Inspect seat upholstery to ensure it is fastened to seat frame. Check tension knobs on leverage bar for proper function.			
CASTERS	•	•	
• Inspect wheel / fork assembly for proper tension by spinning caster; Caster should come to gradual stop.	•	•	
• Tighten / loosen locknut if wheel wobbles noticeably or binds to stop. CAUTION: As with any vehicle, the casters should be checked periodically for cracks and wear, and should be replaced.	•	•	
CLEANING	•	•	
• Clean upholstery and armrest.			
FOOTREST GUARD / SCISSOR MECHANISM GUARD	•	•	
• Inspect for damage (torn cracked).			



Lifeline Corporation Pte Ltd

745 Lorong 5 Toa Payoh #01-00 Singapore 319455

Tel: 62892062 • Fax: 62896096 • e-sales@lifeline.com.sg



OPERATING INSTRUCTIONS

CAUTION

Ensure that there is adequate room to operate the recliner. There must be at least 3 feet clearance between the top of the recliner back and the surrounding objects, otherwise damage to the recliner and/or surrounding property may occur.

RECLINING THE CHAIR USING THE POSITION LOCK LEVER


NOTE: The position lock lever reclines the back and locks it into position. The user cannot recline the chair by pushing on the arms when the position lock lever is in use.

WARNINGS

Ensure that the position lock lever is in the locked position when occupying the recliner, otherwise bodily injury may occur to the occupant or to an individual standing behind the recliner.

NOTE: Right and left sides are determined by the user's seated position.

1. Locate the plunger release knob on the right side of the back frame.
2. Pull the knob outward and hold.
3. Pull the position lock lever out.
4. Align one of the notches on the position lock lever with the studs on the inner seat frames.
5. To re-adjust the position of lock lever perform the following:
 - A. Grasp position lock lever out.
 - B. Pull up to remove the lock lever from the studs.
 - C. Adjust the lock lever to a different position.
 - D. Secure lock lever in place by engaging it in the notches on the studs.

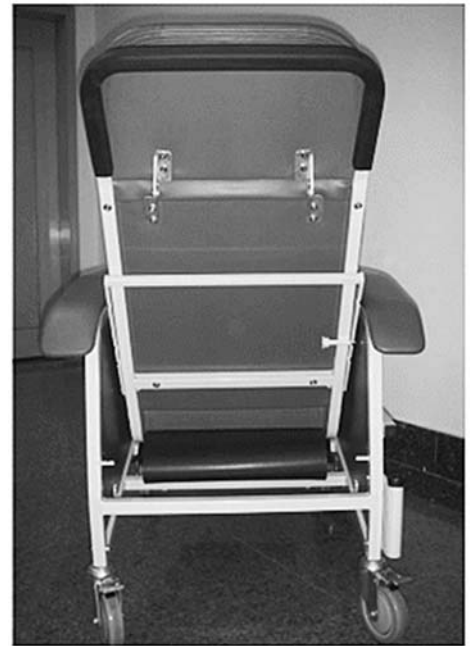

RECLINING THE CHAIR WITHOUT USING THE POSITION LOCK LEVER

1. Locate the plunger release knob on the back assembly.
2. Pull the knob outward and hold.
3. Push the position lock lever up against the back assembly frame and release the plunger release knob.

NOTE: The knob will hold the position lock lever in the stored position preventing it from locking into the studs on the frame.

4. While seated in recliner, grip the armrests and simultaneously lean body weight back and push forward with your arms.

NOTE: The footrest assembly will elevate automatically as the chair is reclined.

NOTE: The occupant should not be required to exert excessive force to change recline positions. If the recliner exhibits difficulty in reclining, the position lock lever may be engaged and an attendant should be called to put position lock lever in the stored position. If recliner still exhibits difficulty in reclining, call a qualified technician, Lifeline technical service listed on the bottom page.

5. An attendant can easily recline the chair from a standing position (beside the chair) by simply grasping the armrest with one hand; the padded push handle with the other and then pushing down the push handle.



POSITIONING THE ARTICULATING HEADREST

To position the articulating headrest, simply grasp the headrest and push or pull it into the desired position.

LOCKING / UNLOCKING THE REAR WHEELS

1. Push down on the wheel lock mechanism to engage the wheel locks.
2. Release the wheel locks by pushing the other side of the lock mechanism downward.

INSTALLING TRAY INTO POSITION



1. Remove the tray from its storage hooks on the side of the chair.
2. Pull down and twist the plunger release knobs under both armrests of the recliner to open the pathway for the table rods.
3. Insert the table rods into the openings. Slide the tray into the chair to the desired position. The push button will lock into place.
4. Twist the plunger release knobs to the original position to lock the table rod into place.



Lifeline Corporation Pte Ltd

745 Lorong 5 Toa Payoh #01-00 Singapore 319455

Tel: 62892062 • Fax: 62896096 • e-sales@lifeline.com.sg



ADJUSTING THE TRAY POSITION

There are 7 adjustable tray positions on the recliner. They range from 9" to 15" in 1 to 1½" increments.

1. Pull down on the plunger release knobs below the armrests of the chair and push/pull the tray to the desired position.
2. Ensure the tray locks into place.

REMOVING AND TRAY STORAGE

1. To remove the tray, stand in front of the chair and grasp the front edge of the tray with one hand. With the other hand, pull down on the plunger release knobs located below the armrests.
2. **STORING THE TRAY** – Slide the tray forward and away from the recliner. Once the tray is removed, store the tray on the hooks located on the right side of the recliner.

CLEANING UPHOLSTERY

NOTE: A variety of solvents may be used to clean the upholstery on the recliner dependant upon the severity of the stain. The following list should be helpful:

- Ordinary dirt stains: Clean upholstery with warm water and mild detergent to remove superficial soil.
- Ground-in dirt stains: Use soft bristle brush and powdered cleaner or similar detergent.
- Tar, asphalt or creosote: These will stain and should be removed as soon as possible. Area can be cleaned with kerosene or naphtha.
- Paint or shoe polish: Remove immediately. Use a white cloth dampened with kerosene, naphtha, or turpentine.

CAUTION

When kerosene or naphtha is being used, care should be exercised. Keep away from fire and flame and use only in a well-ventilated area.

Carefully follow cleaning instructions provided with cleaning products.

DO NOT use paint remover or any liquid brush cleaner.

Excrement: Remove immediately. Clean with 1:10 mix of bleach and water.

NOTE: The wear ability of your chair can be improved by using a quality paste wax periodically.