



# User Manual

20/2087

## Stainless Steel Commode with Front Tipper & PVC Seat Cushion



### WARNING

- Lifeline Corporation Pte Ltd shall not be held liable for any damages, injuries or accidents arising from mis-used, mis-handling and negligence.
- Lifeline personnel shall demonstrate the basic handling of the commode.
- The safety features for operating the commode must be strictly observed.
- Any defects must be reported immediately to Lifeline Corporation Pte Ltd.
- When transporting the commode, brakes should be in an unlocked position. The castors are designed for use in an indoor environment and on even surface.
- Castors may be damaged when moving the commode along any rough & uneven surface.



Lifeline Corporation Pte Ltd

130 Joo Seng Road #02-02 / 03. Singapore 368357

Tel: 6289 2062 • Fax: 6289 6096 • e-sales@lifeline.com.sg



## SAFETY INSTRUCTION

- Commode should not be use if found defective as it may injure the user or caregiver.
- Caregiver should be familiar with the functionality of the commode.
- Commode must not be overloaded (Maximum load capacity: 110kg).
- If the user is on the commode, castors must be locked. An unlocked commode poses a risk of falling to the user.
- Do not leave user with poor stability unattended on the commode.

## CLEANING

- Use a cloth to wipe the commode dry after use.
- Dry out in the sun if possible.
- Regularly clean out debris on the castors for easy manoeuvring.

Seat Width:	17"	Overall Width:	20.5"
Seat Depth:	17.5"	Overall Depth:	23" - 31"
Front Seat Height:	20"	Overall Height:	38"
Rear Seat Height:	20"	Seat to Arm Rest Height:	8.5"
Back Height:	15"	Weight:	10.3kg
Rear Wheel:	5"	Weight Capacity:	110kg
Front Wheel:	5"	Warranty:	1 year on frame
Floor to Bracket	17.5"		



### DETACHING THE ARMREST



Press the side button to release the lock. Pull the armrest upward to detach

### FLIP DOWN THE FRONT ANTI TIPPER



Press the side button to release the lock. Pull the front anti tipper downwards and lock into vertical position for use.

**\*front anti tipper is to prevent the user from falling forward.**

**DO NOT LEAVE USER UNATTENDED**