



TANGIBLE BOOST

Keep Your Coated Lenses
Performing Their Best!

What Is Tangible Boost?

Tangible Boost is a prescription conditioning solution designed to restore the Tangible Hydra-PEG layer on the surface of the lens and maintain its wettability. Normal wear and tear can happen over time, thinning the Hydra-PEG coating and decreasing its benefits to wettability and deposit resistance. Although the lens still performs better than an uncoated lens, patients may not experience the same “wow” factor of Tangible Hydra-PEG after the coating starts to thin.

Used once per month, Tangible Boost restores the Hydra-PEG coating, combating normal wear and tear. This ensures that patients experience the benefits of Hydra-PEG for the prescribed lifetime of their lens. Tangible Boost is not approved for use with hybrid contact lenses.



Frequently Asked Questions

What would cause the Hydra-PEG coating to thin?

Factors that may impact the coating over time include, but are not limited to:

- A patient's unique tear composition (heavy depositors)
- Vigorous rubbing while cleaning
- Use of tap water on the lenses
- Storing lenses dry

Who should use Tangible Boost?

Any patient with Tangible Hydra-PEG coated lenses will benefit from using Tangible Boost as a monthly preventative treatment to maintain the thickness of the Hydra-PEG coating. However, some specific indications include:

- Patients who notice their lenses do not feel the same, or do not feel “as slippery” as when they were brand new
- Known heavy depositors
- Patient with exposure issues such as incomplete blinking
- Patients noticing an increase in mid-day fogging

How is Tangible Boost sold?

Tangible Boost is a prescription only product that is sold directly to patients online. When ordering, patients can enter their Practitioner details so Tangible can verify their Rx information before processing the order.

Tangible Boost can also be sold directly by Eye Care Practitioners by wholesaling out of their practice.

What is included in a single box of Tangible Boost?

A box of Tangible Boost contains all the materials required for two monthly treatments, including two (2) foil pouches of Tangible Boost components (A and B), two (2) lens cases for soaking, and an instruction manual.

How does Tangible Boost fit into the Tangible Care System?

Tangible Boost is an essential part of taking care of Hydra-PEG coated contact lenses. Using Tangible Boost monthly with coated lenses makes sure that the coatings benefits maintain for the entire lifetime of the lens.

Can patients purchase Tangible Boost online without a prescription?

Patients can purchase Tangible Boost online at **tangiblescience.com** even if they do not have a prescription for Tangible Boost. Once an order is placed online, Tangible Science will reach out to the patient's practitioner to verify, or obtain a prescription. The order will not ship until an Rx is obtained. If an Rx is not obtained, the order will be canceled and refunded.



For more information visit
www.tangiblescience.com/products/tangible-boost