



Rev-5 Item-100398



## DreamPort™ Application Instructions

Visit [www.bleepsleep.com/instructions](http://www.bleepsleep.com/instructions) for step-by-step guide




1

## DreamPort™ Application Instructions

### Step 1

The first step to applying your DreamPorts™ is to thoroughly clean your nose area.

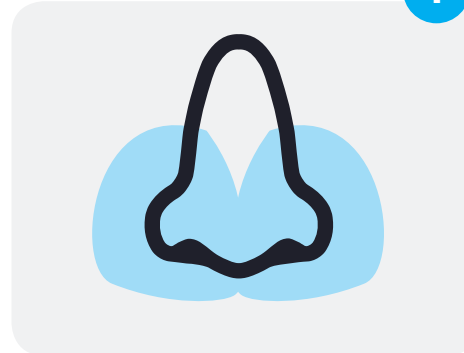

Clean areas around nostrils of makeup, oil or dirt with DreamPrep™ wipes or other mild astringent like alcohol or witch hazel wipes. Then allow your nose to air dry or pat dry with a towel.



2

### Step 2

Apply the DreamPort™ adhesives by applying pressure around the nostril - ensuring you have a good seal. Repeat the process to apply the second DreamPort™.



3

### Step 3

Click the applied DreamPorts™ into the DreamWay™ connector, then attach to your CPAP unit.



4

### Step 4

Settle in for the best bleeping night's sleep you've ever had!



Rev-5 Item-100398



## DreamPort™ Instrucciones de aplicación

Visite [www.bleepsleep.com/instructions](http://www.bleepsleep.com/instructions) para un video con instrucciones detalladas.




1

## DreamPort™ Instrucciones de aplicación

### Etapa 1

Limpie bien su nariz y el área alrededor de su nariz.

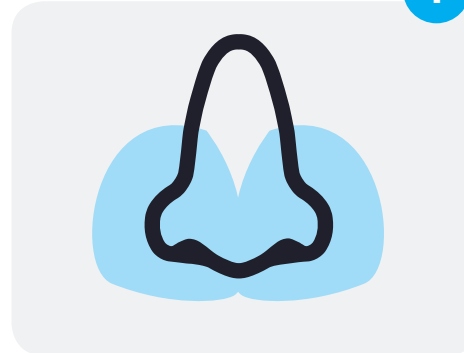

Limpie las fosas nasales de maquillaje, aceite o suciedad con la toallita DreamPrep™. Después, deja que la nariz se seque al aire o seque con una toalla.



2

### Etapa 2

Aplique el adhesivo DreamPort™ por aplicando presión sobre las fosas nasales - asegúrese que tiene una buena conexión.



3

### Etapa 3

Conecta los DreamPorts™ a la interfaz DreamWay™ con un clic audible, entonces conecta a su dispositivo para CPAP.



4

### Etapa 4

Prepárese para dormir bien!