



SureStep[™] Drug Screen Test. Multi-Drug Test panel with Integrated E-Z Split Key[™] Cup I & Cup II

A complete drug testing system in a sealed cup.

Offering a variety of simple and rapid urine screening tests for use in Occupational Health and Drug Rehabilitation.

Alere

Drugs and cut-offs

Name	Abbreviation	Urine cut off (ng/mL)
Amphetamine	AMP	300*
Barbiturates	BAR	300
Benzodiazepines	BZO	200*
Cocaine	COC	300*
Ecstasy	MDMA	500
Cannabis	THC	50*
Methadone	MTD	300
Methamphetamine	MET	300*
Morphine	MOP	300*
Phencyclidine	РСР	25
Tricyclic Antidepressants	TCA	1000

Multi-Drug Test Panel with Integrated E-Z Split Key™ Cup I & Cup II.

*The above cut-offs are compliant with AS/NZS 4308:2008.

Instructions for use

- 1. Collect urine specimen in the cup and secure cap
- 2. Check the temperature label (Temp Label) up to 4 minutes after specimen collection
- 3. Push the key into the socket of the cup to begin the drug screen test
- 4. Read results shown at 5 minutes

Refer to Instructions For Use provided within kit for full instructions.

Result interpretation

A line must appear in the control line region (C) to indicate that the test has worked correctly.

Please contact Alere for further information.

Alere

532 Seventeen Mile Rocks Rd Sinnamon Park, QLD, 4073 Australia +61 7 3363 7100 Office +61 7 3363 7199 Fax au.enquiries@alere.com.au Email

Alere Ltd

PO Box 105653 Auckland City, Auckland, 1143 New Zealand

alere.co.nz

0800 466 951 Office 0800 466 958 Fax nz.enquiries@alere.com Email

Kit size: 25 tests. Storage: 2–30 °C. Shelf life: 2 years after date of manufacture.


Drug combinations and other SureStep drug tests

The Multi-Drug test panel with Integrated E-Z Split Key™ Cup I & Cup II formats allows any combination of up to 6 drugs from an extensive menu of 20 different assays. A maximum of 12 assays are available, depending upon the exact combination required. Other formats available within the SureStep™ brand include: Single and Multi-Drug Test devices, Drug Test Panels, Adulteration Tests and Saliva Tests.

Features and Benefits

- Secure: Eliminates exposure to urine and provides temperature verification to confirm urine samples
- **Convenient**: Simple 3 step procedure collect the urine, push the key and read results at 5 minutes
- Flexible: Drug combinations from 6 drugs to 12 drugs
- Stable: Long shelf life stored at room temperature or refrigerated (2–30°C)
- Performance: Accurate and high level of correlation with reference methods GC /MS
- Verified to AS/NZS 4308:2008 Appendix B (Compliance certificate available)*
- * Only applicable to 6 panel test.

Security

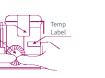



6888RE







For more information please contact:

7 – 12 Drugs

alere.com.au

© 2010 Alere. All rights reserved. A0680 Alere, SureStep™ and Integrated E-Z Split Key™ are trademarks of the Alere group of companies. Information not for distribution in the USA.