

Adobe White Vinyl Care Guide

Please be aware that while Adobe White is one of the most aesthetically pleasing colors, it can also be one of the most difficult to maintain in a busy office setting. It may require additional cleaning between each patient compared to other color choices.



Disclaimer: SunChiroTable is not responsible for Adobe White tables that stain or discolor through improper care and cleaning. Do not clean tables with commercial products containing any type of oil or harsh abrasive. This will damage the protective BeautyGuard on the fabric. If the cushions have been stained or discolored, they are NOT eligible for free replacements or refunds. However, we do offer replacement cushions for purchase if needed. Please contact us at sales@sunchirotable.com to place an order for replacement cushions.

General Cleaning & Disinfecting Guide: (via naugahyde.com)

- Tables can be cleaned and disinfected with a 10% bleach gel (1:10 dilution in water)
- After each application, disinfectants must be wiped off with clean water and dried with a clean towel to remove any chemical residue that could discolor or degrade the material surface.
- The surface of the vinyl has a special coating to protect the vinyl from staining. Improper cleaning can wear this coating away and will result in the table becoming more easily discolored. In a busy office it can be difficult to remember to wipe off disinfectants after every cleaning, but this is **vital** to the longevity of your table regardless of color!

Adobe White Special Care:

- To avoid the risk of staining, remove any pens or markers from the pockets of clients and doctors who are using the table.
- If the table has been marked with some kind of stain, it is important to remove it immediately. The best way we have found to remove most stains in our shop is to use a cloth sprayed with WD40. This will lift most stains, including ink-based ones. Wipe down the area with clean water afterwards to remove any residue.