JOHNNY LOCO MANUAL

This manual shows how to remove the metal ring when preparing an optical frame.

Please select your language:


[ENGLISH](#) | [NEDERLANDS](#) | [DEUTSCH](#) | [FRANÇAIS](#) | [ESPAÑOL](#)

Johnny Loco[®]

JOHNNY LOCO MANUAL (EN)

This manual shows how to remove the metal ring when preparing an optical frame.
If you need assistance, please contact Johnny Loco Eyewear.

eyewear@johnny-loco.com
[+31 20 3445928](tel:+31203445928)

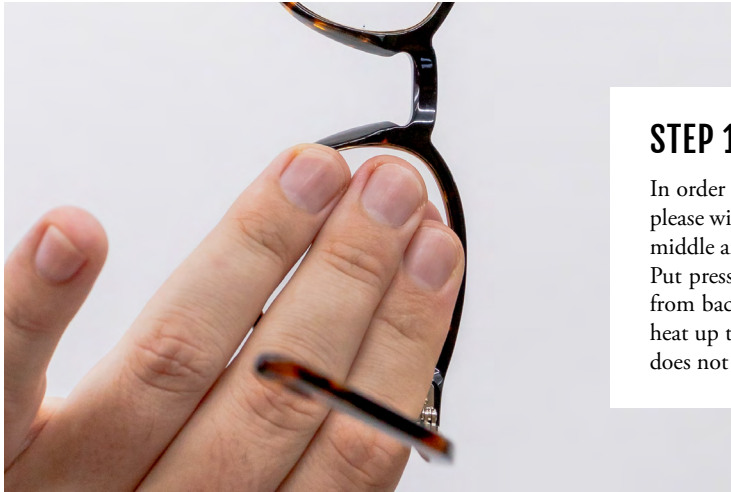STEP 1

In order to remove the demo lens and metal insert in one go, please widen your pressure point by using three fingers (index, middle and ring finger) as shown on the image instead of one. Put pressure on all three fingers removing the lens and insert from back to front until you hear a 'click'. You could slightly heat up the acetate to help, but please be aware that the metal does not get too hot while doing so.

STEP 2

Remove the metal insert from the frame and replace or adjust the lens as you would on a regular metal frame.

STEP 3

Replace the new lens and metal insert by using the spacing on the back of the acetate frame as reference to line up the lens correctly. Simply use two thumbs as shown on the image to push the lens and insert back (from front to back) in one go, until you hear a 'click'. No heating is required, this is a cold insert.

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