


30-0002.05.00 Syringe 3-Way
Installation Instructions

CAUTION: Turn Main Air and Water Supply Off Before Servicing Syringe.

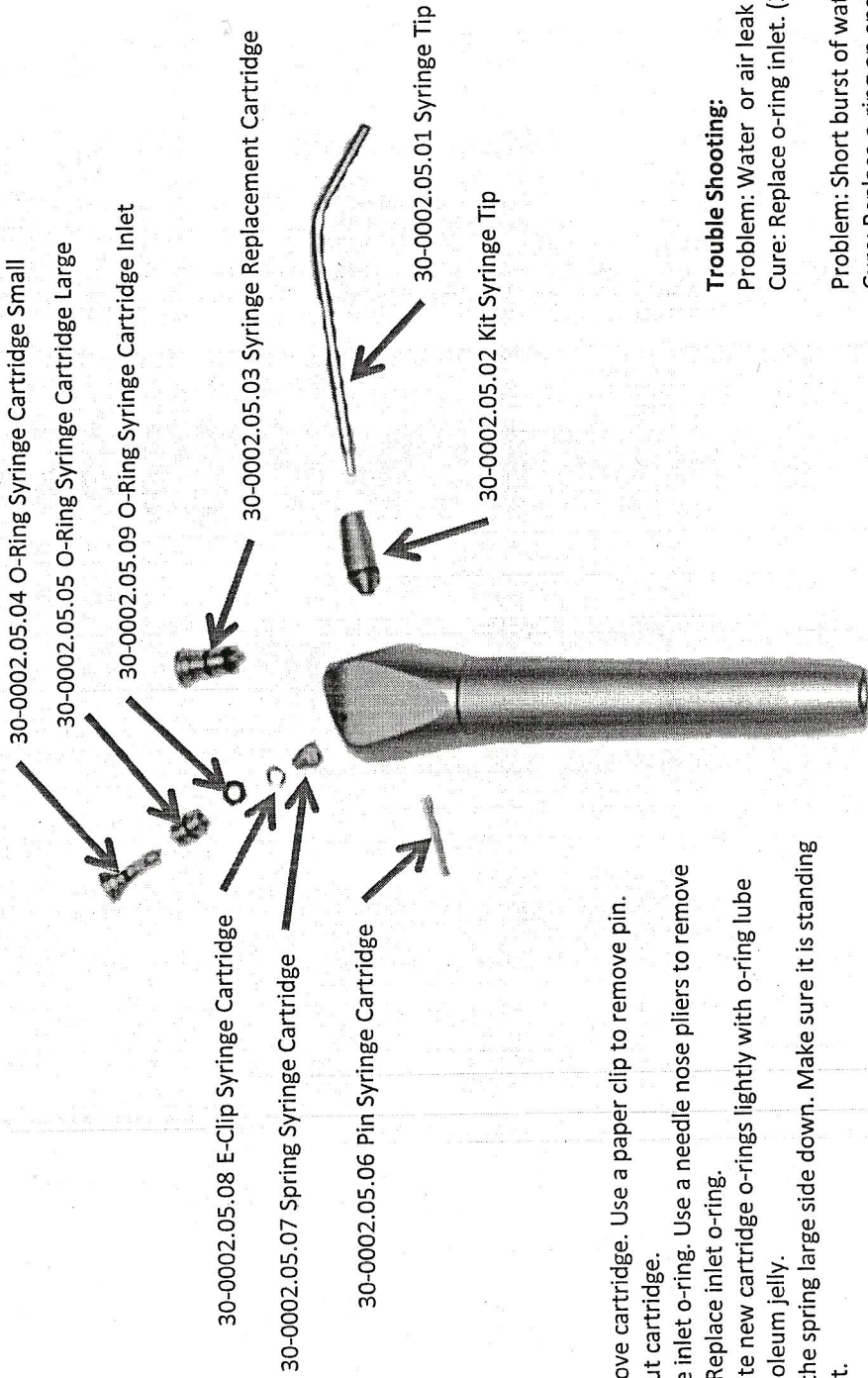

Service:

- 1) To remove cartridge. Use a paper clip to remove pin. Slide out cartridge.
- 2) Replace inlet o-ring. Use a needle nose pliers to remove e-clip. Replace inlet o-ring.
- 3) Lubricate new cartridge o-rings lightly with o-ring lube or petroleum jelly.
- 4) Install the spring large side down. Make sure it is standing straight.

30-0002.05.10 Syringe Repair Kit

- 30-0002.05.03 Syringe Replacement Cartridge 2 Each
- 30-0002.05.04 O-Ring Syringe Cartridge Small 2 Each
- 30-0002.05.05 O-Ring Syringe Cartridge Large 2 Each
- 30-0002.05.06 Pin Syringe 1 Each
- 30-0002.05.07 Spring Syringe Cartridge 2 Each
- 30-0002.05.08 E-Clip Syringe Cartridge 2 Each
- 30-0002.05.09 O-Ring Syringe Cartridge 10 Each

Trouble Shooting:

Problem: Water or air leak at tip.

Cure: Replace o-ring inlet. (30-0002.05.09)

Problem: Short burst of water when air is applied.

Cure: Replace o-ring on end of kit syringe tip.

Problem: Water or air leak around syringe cartridge.

Cure: Replace large o-ring. (30-0002.05.05)

Problem: Water or air leak on button when depressed.

Cure: Replace small o-ring. (30-0002.05.04)

Problem: Spray pattern not uniform.

Cure: Clogged tip, clean or replace. (30-0002.05.01)