
USER MANUAL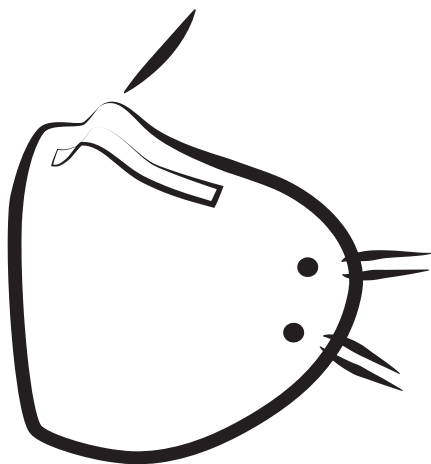

H910EN

Filtering half mask

Honeywell

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Please read the user manual before using the product.

APPLICATIONS

Particle filter half-masks classified FFP1, FFP2 or FFP3 against solid and liquid aerosols in concentrations up to Protection Factor X WEL (Workplace Exposure Limit): 4 x WEL: FFP1 class (low toxicity), 12 [10 in UK, FIN, D, I, S] x WEL: FFP2 class (average toxicity), 50 [20 in FIN, S, UK & 30 D, I] x WEL: FFP3 class (high toxicity). «D» marked for passing the dolomite clogging test. «NR» (single -shift use) or «R» (re-useable) marked according to the amended standard EN 149:2001 +A1:2009.

WEARING AND ADJUSTMENT

- 01 Follow these instructions each time respirator is worn.
- 02 Cup the respirator in the hand, allowing the head straps to hang freely.
- 03 Hold the respirator under the chin with the nose-piece facing outwards. Place the lower head strap around the neck below the ears.
- 04 Holding the respirator against the face with one hand, place the top head strap above the ears, around the crown of the head.
- 05 Place hands on each side of the respirator and move slightly right, left, up and down, to adjust the position of the respirator and the nose-piece to achieve the most optimal fit on the face.

FIT-CHECK THE FACE-SEAL AS FOLLOWS

- 01 Place both hands over the respirator - do not disturb the position of the respirator.
- 02 Inhale sharply (a negative pressure should be felt inside the respirator).
- 03 Exhale sharply (a positive pressure should be felt inside the respirator).
- 04 If you detect air-leaks, readjust the head straps and/or the nose-piece.

WARNING: If you cannot achieve satisfactory fit, do NOT enter the contaminated area. See your Supervisor.

USE INSTRUCTION

- 01 Failure to follow all instructions and limitations concerning the use of the respirator and/or failure to wear the respirator for the whole duration of exposure to contaminants can seriously reduce the performance characteristics of the respirator and lead to illness, injury or death.
- 02 Before Occupational Use of this respirator, a written respiratory protection program must be implemented meeting all the local government requirements.
- 03 Taking a new mask from its box.
- 04 Check the condition of the mask before use.
- 05 Fit the mask and check leak-tightness before entering the contaminated area.
- 06 Wear it for the whole duration of exposure to contaminants
- 07 Discard the respirator and replace with a new one if:
 - it is removed while in a contaminated area,
 - excessive clogging of the respirator causes breathing difficulty or discomfort,
 - the respirator becomes damaged,
 - for masks offering relief against nuisance odors, the smell becomes detectable.
- 08 Do not alter the mask.
- 09 Leave the contaminated area if dizziness, irritation or other distress occurs.

LIMITATIONS

- 01 DO NOT use this mask to enter or stay in an area where:
 - oxygen concentration is less than 17% (oxygen-deficient atmospheres),
 - contaminants and/or their concentrations are unknown or immediately dangerous to life or health,
 - aerosol concentration exceeds levels fixed by applicable health and safety regulations or Protection Factor x WEL,
 - gases and/or vapors are present.
- 02 DO NOT use this mask in explosive atmospheres.
- 03 DO NOT use before receiving training.

CHECKING BEFORE USE

The user shall ensure that the mask is suitable for the intended application before entering the contaminated area.

STORAGE CONDITION

- 01 The respirator should be stored at the area which is well-ventilated, avoids direct sun exposure and dry area. Keep away from fire and pollution.
- 02 Storage temperature range is -20°C (-4°F) to 38°C (100.4°F). Maximum Relative Humidity < 80%.
- 03 Best before: Check on package.

WARNING

This respirator helps protect against certain contaminants. It may not eliminate the risk of contracting disease or infection. Misuse may result in sickness or death.

This mask is marked "NR", the particle filtering half mask shall not be used for more than one shift. Keep unused masks in their closed box and store in a non-contaminated area.

CLEANING

Applicable only on the re-usable masks («R»). If the mask is to be used for more than one shift, the face seal must be cleaned with a wipe impregnated with a cleaning and disinfecting solution, without alcohol.

Sort out your waste by respecting the regulations in force.

FOR MORE INFORMATION or assistance on Honeywell products, contact your local Honeywell distributor.

Particle filtering half mask meet the requirements of Regulation (EU) 2016/425. They have been tested according to EN 149:2001+A1:2009. Module B (EU type-Examination Certification) and Module C2 or Module D Certification have been issued by CCQS Certification Services Limited, Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin15, D15 AKK1, Ireland (NB 2834).

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Honeywell Safety Products (Shanghai) Co., Ltd.