

### MEASURING CHART

**Class 1: Mediven®Elegance, Mediven®Plus, Mediven®Comfort, Mediven®for Men, Mediven®Active**  
**Class 2: Mediven®Elegance, Mediven®Plus, Mediven®Comfort, Mediven®for Men, Mediven®Active**  
**Class 3: Mediven®Plus**

CIRCUMFERENCE		MEASUREMENTS IN CM						
Sizes		I	II	III	IV	V	VI	VII
Thigh - X-Wide Silicone Band	g	49-57	53-62	57-67	61-72	65-77	69-81	73-85
Thigh - Silicone Band	g	43-48	45-52	49-56	53-60	56-64	60-68	64-72
Thigh, w/w attach, Pantyhose	g	43-57	45-62	49-67	53-72	56-77	60-81	64-85
	d	27-33	29-36	32-39	34-42	36-45	38-48	40-50
	c	28-34	30-37	33-40	35-43	37-46	39-49	41-51
	b	18-20	20-22	22-24	24-26	26-28	28-30	30-32
<b>Extra wide A-D stockings available Mediven®Plus: Class 1, 2 &amp; 3</b>								
	c	34-38	37-41	40-44	42-46	44-48	46-50	48-52

LENGTH for Mediven®Elegance,Plus,Comfort,Men					
A-D	Petite	34-38	A-G	Petite	62-71
A-D	Regular	39-44	A-G	Regular	72-83

LENGTH for Mediven®Active ONLY					
A-D	Petite			37-42	shoe sz 38-42
A-D	Regular			43-49	shoe sz 43-48

### MEASURING CHART Duomed Class 1 & 2


CIRCUMFERENCE		MEASUREMENTS IN CM				
Sizes		S	M	L	XL	XXL
Thigh - Silicone Band	g	47-54	52-60	56-65	62-72	65-76
Thigh NO Silicone Band	g	42-57	48-64	54-71	60-78	65-85
	c	28-34	32-38	36-42	40-46	42-50
	b	19-21	22-24	25-27	28-30	30-32

### LENGTH for Duomed

A-D	Petite	34-38	A-G	Petite	62-71
A-D	Regular	39-44	A-G	Regular	72-83

### MEASURING CHART Medi Travel

CIRCUMFERENCE Ankle only		MEASUREMENTS IN CM				
Sizes		S	M	L	XL	XXL
	b	18-20	20-23	23-26	26-29	29-32
<b>LENGTH A-D Petite</b>	34-38	<b>LENGTH A-D Regular</b>			38-44	

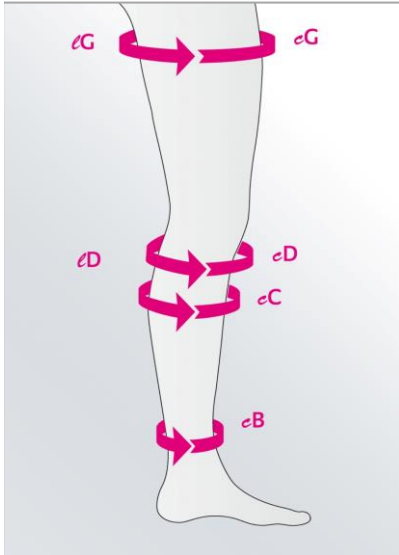


Lengths are taken from  
 A=floor to D (below knee)  
 to G (Thigh,w/w attach,P/hose)

Circumference skin measurement  
 ONLY

Length taken standing position  
 ONLY

Gusset Variants refer to  
 Export Catalogue

## MEASURING CHART Mediven® Forte Class 2 & 3

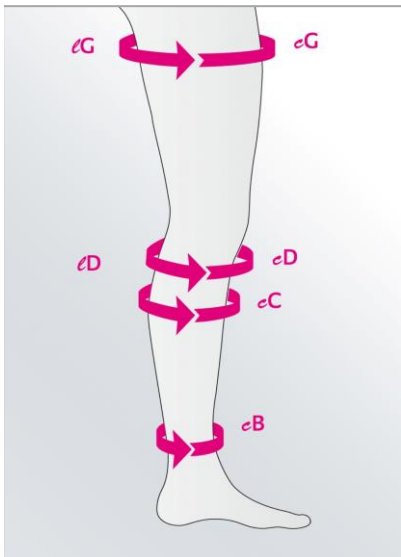

CIRCUMFERENCE		MEASUREMENTS IN CM						
Sizes		I	II	III	IV	V	VI	VII
Thigh - X-Wide Silicone Band	g	48-53	51-57	55-61	59-65	62-70	66-74	71-80
Thigh - Silicone Band	g	44-47	46-50	50-54	53-58	56-61	60-65	64-70
Thigh, w/w attach, Pantyhose	g	44-53	46-57	50-61	53-65	56-70	60-74	64-80
	c	29-34	31-36	33-38	35-40	37-42	39-44	41-46
	b	18-20	20-22	22-24	24-26	26-28	28-30	30-32
<b>Extra wide A-D stockings available Mediven®Forte: Class 2 &amp; 3</b>								
	c	33-38	35-40	37-42	39-44	41-46	43-48	45-50

### LENGTH for Mediven®Forte

A-D	Petite	33-37	A-G	Petite	62-71
A-D	Regular	38-43	A-G	Regular	72-83

Compression	mm Hg	Indications
Class 1 Light	18-21 mm Hg	Mild varicosis without a significant predisposition of oedema, early varicosis in pregnancy
Class 2 Medium	23-32 mm Hg	For more noticeable symptoms, pronounced varicosis with formation of oedema, post traumatic swelling, after healing of minor ulcers, after thrombophlebitis, to stabilise the results of sclerotherapy and varicotomy, more severe varicosis in pregnancy.
Class 3 Strong	34-46 mm Hg	For all late complications of constitutional or post thrombotic venous insufficiency, significant predisposition to oedema, secondary varicosis, atrophie blanche, dermatosclerosis and after healing of severe ulcers, particularly recurrent ulcers.
Class 4 Very Strong	over 49 mm Hg	Lymphoedema and elephantiasis

\*Absolute contraindications: advanced, peripheral arterial occlusive disease, decompensated heart disease, septic phlebitis, Phlegmasia coerulea dolens.

\*Relative contraindications: supportive dermatoses, intolerance of compression stocking fabric, sensory disturbances of the limb, advanced peripheral neuropathy, primary chronic arthritis.

**Medi assumes no liability as a result of any contraindicated use of this product**

