

SRC Patented Anatomical Support Panels deliver targeted compression, ideal for treating multiple conditions. The SRC Rehab garment is lightweight, breathable, moisture wicking with flatlock seam construction that will not irritate the skin. Recommended by health care professionals for its ability to improve mobility and speed up post surgery recovery.

Abdominal Panel

Abdominal surgery, e.g. laparotomy, appendectomy, abdominoplasty, hernia repairs

**Double layer panels*


Gusset Panel

Gynaecological procedures and pelvic floor repair surgeries

**Double layer panels*

Double Layer Panels*

Provide true cross compression so that during activity no compression is lost at any time, delivering increased circulation and muscle healing

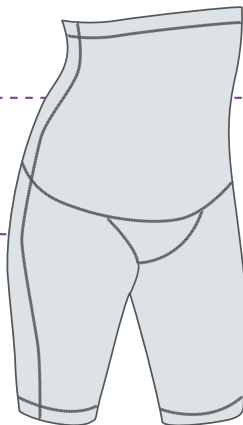

Back Panel

Provides back support from T12 to L5
Lower back surgical procedures between and around these vertebrae

**Double layer panels*

Leg Panel

Encourages good venous return
decreasing swelling

Wear & Care Instructions

- 1** Sit while putting the garment on over your feet and legs. Pull the garment over one foot and leg at a time.



- 2** Once on both legs, pull the garment up to your knees.



- 3** Pull garment up to your waist. Ensure the gusset panel is in contact with the crotch area.



- 4** Pull the top band up until it sits under your bra. You may need assistance after surgery. Lift over any surgical wounds. To remove fold elastic over top and roll down



- 5** The garment should feel supportive, not restrictive. It does NOT need to be tight to work. Wear for a minimum of 6 weeks post surgery or as instructed by your health care professional.



- 6** Many women continue to wear their rehab shorts for exercise or for aesthetic reasons under fitted clothing.



Caring For Your Garment



Unlike hosiery our garments are pretty tough. Below are some tips to maintain its performance and extend its lifespan.

Splintered / sharp fingernails, sharp jewellery and watches can snag and damage the fabric.

Washing Instructions



Wash your garment in cold water with a small amount of detergent.



Line dry inside. Do not dry in the direct sunlight.



Do not use softeners or bleach.

Endorsed by:

