



SRC
restore™

Restore Incontinence/Prolapse Leggings

For Women

SRC Patented Anatomical Support Panels deliver targeted compression. The SRC Restore garment is lightweight, breathable, moisture wicking, with flatlock seam construction that will not irritate the skin. Recommended by health care practitioners for its ability to help with symptoms of mild urinary incontinence and mild prolapse. Wear day and night for optimal results.

Abdominal area*

Support from the umbilicus to pubic bone

Support for Back Area*

Lumbo-sacral area

Leg Area**


Graduate compression encourages good venous return

Gusset Area*

Triple layer even compression to the entire perineal area

**Triple layer panels provide true cross compression so that during activity no compression is lost at any time, delivering increased circulation and support*

*** Single layer*

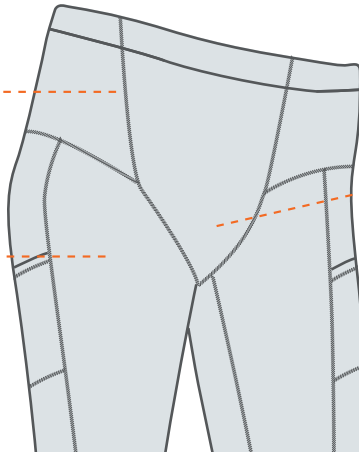

Wear & Care Instructions

- 1 Sit while putting the garment on over your feet and legs. Pull the garment over one foot and leg at a time.



- 2 Once on both legs, pull the garment up to your knees.



- 3 Pull the top of the garment and especially the gusset piece, well into the perineal area so that it is in full contact with that area. If this panel is not in continuous contact with the perineum it significantly reduces its ability to work.



- 4 The garment should feel supportive, not restrictive. It does NOT need to be tight to work.



Caring For Your Garment



Unlike hosiery our garments are pretty tough. Below are some tips to maintain its performance and extend its lifespan.

Splintered / sharp fingernails, sharp jewellery and watches can snag and damage the fabric.

Washing Instructions



Wash your garment in cold water with a small amount of detergent.



Line dry inside. Do not dry in the direct sunlight.



Do not use softeners or bleach.

Warning If you have any medical condition that makes you question the safety of wearing any garment please always consult with your health care practitioner before wearing the garment.