



Enhancing Lives,
Delivering Comfort

OPERA® SIGNATURE SIDE
WALL PROTECTOR

Installation and User Guide



VISIT [OPERABEDS.COM](https://www.operabeds.com) | CALL 0333 222 8584



Before you begin! We recommend that you conduct a risk assessment to ensure that the Opera Signature Side Wall Protector meets your requirements and does not become a trip hazard.

1. General Information and Safety

1.1 Intended & Non-Intended Uses

The Opera Signature Side Wall Protector is designed to reduce the risk of an Opera Signature Profiling Bed being placed too close to a wall, plug socket, skirting board or fixture, which may be damaged by the bed. The wall protector gives a 40mm clearance from the bed surrounds to the wall or fixture.

The Opera Signature Wall Protector is only designed to be used with the Opera Signature Profiling beds. Opera will not be liable for any hazards, injury or damage arising from using the wall protector incorrectly or with a different bed.

Any incidents or issues must be reported to Opera at, support@operabeds.com.

2. User Instructions



Always read the instruction manual before using the wall protectors, pay particular attention to the instructions below.



Always carry out a risk assessment to ensure that the wall protector is suitable for the required application. Ensure that the installation of wall protectors does not create risks for both staff and users, in particular, trip hazards.




The wall protectors should only be fitted to the bed at points where the bed is against a wall, socket, fixture or skirting board. Non compliance with this may result in creating trip hazards, this can result in serious injury.

If the bed is moved away from the wall without the wall protectors being removed, this can result in creating trip hazards, and may result in serious injury. Always remove the wall protectors before moving and relocating the bed.

The Opera Signature Side Wall Protector (A) is designed to fit into the Opera Signature Profiling Bed's tubing above the castors (B).

To install the Opera Signature Side Wall Protector, remove the tube cap (C) on the Opera Signature Profiling Bed. Align the cutting on the wall protector with the screw (D) inside the tube of the bed. Push the wall protector into the tube and once it reaches the end, twist clockwise slightly. Push the rubber ring inside the bed's tube to stabilise the wall protector.

1 | Align the cutting with the screw

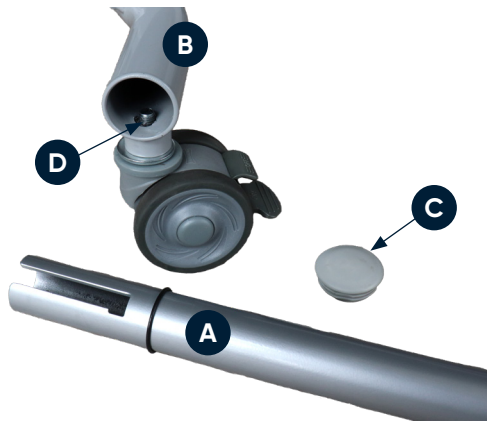

2 | Insert the Wall Protector into the Tube and Twist



3 | Place the Rubber Ring into the Tube



Check the wall protector monthly to ensure that it has not become loose.



Azure House, Connaught Road, Kingswood, Hull, HU7 3AP

0333 222 8584 | support@operabeds.com | [operabeds.com](https://www.operabeds.com)

Version 1 | Published in July 2022