

Instructions for use Max Cleaning

[Cleaning, Disinfection and Sterilization]:


The product is provided in a non-sterile state; it is single use device.

It is required to be processed before each use by following Step-by-Step instructions, with steam sterilization at 126°C for 30 minutes.

Step-by-Step instructions:

- 1) Steam sterilization at 126°C for 30 minutes;
- 2) Rinsethe device with Normal saline (0.9% NaCl) for 10 seconds;
- 3) Dry the device with a dental air gun;

- Minimal invasive file with maximum cleaning.
- Minimal stress applied on dentinal walls.
- Super elasticity with extreme flexibility.
- Maximum cleaning of the canal with smear layer removal.

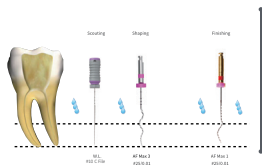

- 1.Irrigate the canal.
- 2.With the use of apex locator, negotiate the canal with C File 10 in a watch-wind motion to full working length, to get a patent canal pathway.
- 3.Irrigate the canal.
- 4.Use Max 3 in a pecking motion (Pecking motion: in and out motion for a depth of 3 mm) till reaching the working length. The motion of the file is on rotation movement(speed 800RPM, Torque 1N).
- 5.Irrigate the canal.
- 6.To ensure better cleaning of the apical third and agitation of irrigation, use Max 1 with the same manner and settings mentioned in step 4.
- 7.Irrigate the canal.

SHANGHAI FANTA DENTAL MATERIALS CO.,LTD
Room 1302, Building 2, No. 6965 Pu Xing Rd., Qingcun
Town, Fengxian Dist, 201414 Shanghai China
+0086-2164033133
WWW.FANTA-DENTAL.COM



△ NFTA Allergy is Prohibited