

Instruction for C-Handle

[Product Name]

C-HANDLE

[Description]

C-Handle is a new designed grip attachment for Endodontic manual instruments, to simplify scouting the canals in the most distal teeth and in limited mouth opening, where accessibility is difficult.

[Characteristics]

- Compatible with Endodontic manual instruments (K files, C files).
- Easy and quick snap ring attachment.
- Long handle design allowing unobstructed visual access.
- Frosted design aiming for non-slippery and comfortable "pen-grasping" sense of touch.
- Distal end notch design and high internal conductivity allowing C-Handle to be used with Apex Locators.
- Autoclave sterilization to reduce cross-infection.

[Clinical uses]

- Compatible with C files in locating canal orifices.
- Glide path and scouting of the canals with improved filling movement.
- Simplified working length determination in the most distal teeth and limited mouth opening.
- Effective for recapitulation between files.

[Precautions]

- C-Handle is intended to be used by dentists only.
- Use C-Handle in accordance with the descriptions.
- It is the dentist responsibility to check the hand files prior to use.
- Ensure the file is well-engaged in the handle before filing action.

[Sterilization]

Autoclave the product before use according to related International Standard (ISO17664: 2017); Irrigate with alcohol and dry up after each time using.

[Storage]

Store in a dry place under the room circumstance. Keep corrosion gas away.

Instruction for C-Handle

	Manufacturer		Date of manufacture
	Authorized Representative in the EuropeanCommunity		Use-bydate
	Caution		Batch Number
	Sterilizable in a steam sterilizer (autoclave) at the specified temperature		Keep dry
	Consult instructions for use		Keep away from sunlight
	Manufacturer Logo		

[Registered Company] Shanghai Fanta Dental Materials Co., Ltd

[Registration Address] Room 1302, building 2, No. 6965, Guangming Puxing Road, Qingcun Town, Fengxian District, Shanghai, China

[Tel:] +021-64033133

[After-sales Service Company] Shanghai Fanta Dental Materials Co., Ltd

[Europe Authorised Representative Address] SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Contact: Yana Zhang (Ms.)

[Tel:] +31 (0) 2021 11106

[Prepared Date for Instruction] 20200325