

# Instruction for plugger

## [Product Name]

Plugger

## [SPECIFICATION]

Refer to specification sheet

## [COMPOSITION]

The instrument can be divided into two parts: handpiece and working part.

## [INSTRUCTIONS FOR USE]

Extrude the canal in three or four times, first to the apical 1/3 with #35-70 plugger, then change to #40-80, #50-100 during the filling of gutta percha point.

If there is a large canal, try to use #60-120 plugger.

## [CONTRAINDICATION]

Do not use in patients allergic to NiTi and stainless steel.

## [STERILIZATION]

Products must be sterilized before use: Wipe with alcohol

## [STORAGE]

At dry and ventilated places

## [PACKAGE]

Individual package

## [PRODUCTION DATE]

Refer to seal or label on the package.

## [EXPIRATION DATE]

Allow for 5 years' guaranteed under the normal conditions since the date of delivering the goods.

# Instruction for plugger



[Registered Company] Shanghai Fanta Dental Materials Co., Ltd

[Registration Address] Room 1302, building 2, No. 6965, Guangming Puxing Road, Qingcun Town, Fengxian District, Shanghai, China

[Tel:] +021-64033133

[After-sales Service Company] Shanghai Fanta Dental Materials Co., Ltd






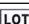




[Europe Authorized Representative Address] SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Contact: Yana Zhang (Ms.)

[Tel:] +31 (0) 2021 11106

[Prepared Date for Instruction] 20200325

	Manufacturer		Date of manufacture
	Caution		Use-bydate
	Sterilizable in a steam sterilizer (autoclave) at the specified temperature		Batch Number
	Consult instructions for use		Keep dry
	Manufacturer Logo		Keep away from sunlight