





Re-Order #	Description	Size	Unit
556069	Skil-Care Slide Sheet	48"L x 40"W	Ea.

Easy to Use



Resident Lateral Transfer

1. Place the Skil-Care Slide Sheet under the resident utilizing the “Log Roll” or similar technique (Fig. 1).
2. Position the Skil-Care Slide Sheet under the resident so that its edge is parallel and just past the resident’s shoulders (Fig. 2).
3. The bed/gurney which the resident is transferring to should not have a gap greater than 5cm or 2 inches from the bed. Make sure the bed/gurney casters should be locked during transfer and is slightly lower from the surface where the resident is being transferred from. Grasp the handles on both sides of the sheet and pull the resident towards the surface you are transferring to (Fig. 3). Make sure when doing the transferring the bed rails are in the up position except the ones between the two surfaces.

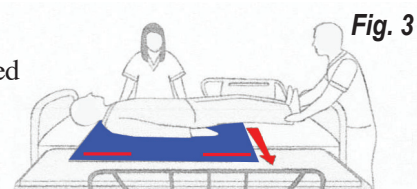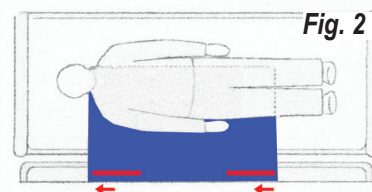

Resident Repositioning

1. Place the Skil-Care Slide Sheet under the resident utilizing the “Log Roll” or similar technique (Fig. 1).
2. Position the Skil-Care Slide Sheet completely under resident and approximately 12 inches above residents head (Fig. 2).
3. Always make sure the casters on the bed/gurney are locked during the repositioning of the resident. One caregiver on each side will grasp the handles of the underside of the sheet and gently pull the bottom sheet from the head towards the feet, unrolling the Skil-Care Slide Sheet under the resident until it is in position.

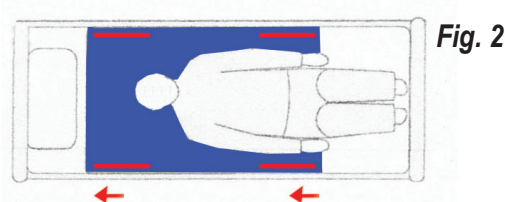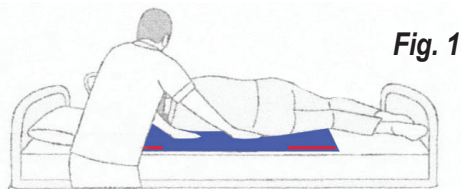

Caution

- DO NOT leave the Skil-Care Slide Sheet unattended under a resident or on their bed at any time.
- DO NOT use the slide sheets to lift patients. Only use to slide, reposition or transfer patients.
- Patient positioning should be done according to your facility’s patient handling procedures.
- Before each use check the system for signs of wear and replace when necessary.

Easy Care

Skil-Care Slide Sheet can be laundered at temperatures below 82°C, 180°F or wiped clean using a damp cloth.